

JAY[®]

GO BACK

JAY[®] GO Back

IMPORTANT CONSUMER INFORMATION

NOTICE: *This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.*

SUPPLIER: *This manual must be given to the user of this product.*

USER: *Before using this product, read this entire manual and save for future reference.*

WARRANTY REGISTRATION: *To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.*

Owner's Manual



Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a GO Back is appropriate. Backs should only be installed by an authorized Sunrise Medical supplier.

THE JAY GO BACK

The JAY GO Back is designed to provide the important benefits of a proper back support for a wide variety of users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

Maximum user weight:

300 lbs. (136 kg) for sizes 12" to 21" widths

500 lbs. (227 kg) for sizes 22" to 26" widths

⚠ WARNING

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.

Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

⚠ WARNING

Do not use JAY GO Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

⚠ WARNING

Do not use the JAY GO back cover or straps to lift wheelchair.

GO BACK



MAINTENANCE

Sunrise recommends that the JAY Go Strap Assembly be checked for wear and tear every 6 months. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose or broken components.

EASY CLEANING

NOTE- When disassembling the back for cleaning, note the position and orientation of the Strap Assembly and cover for correct reassembly.

Outer Cover

Wipe down outer cover using a soft cloth and warm soap and water. The outer cover can be machine washed in warm water and light detergent as needed.

NOTE- Remove foam from inside of cover prior to machine washing.

The foam core may be washed by hand with light soap and water.

Strap Assembly

Wipe down strap assembly as needed using light soap and warm water.

ATTENTION- Do not machine wash the strap assembly.

If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

Laundry Care Symbols	
	Machine wash in warm water (50° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean


INSTALLATION

To install the JAY GO Back, first remove the outer cover from the strap assembly. Then install the strap assembly onto the back canes of the wheelchair by wrapping the Velcro sleeves around the back canes and engage the Velcro fully. (A)

IMPORTANT– Note the height of the back. Make sure the strap assembly is appropriate to the top of the back canes and leaves enough room underneath the strap assembly for the cushion.

NOTE– Align grommet holes of the JAY Go back with the grommet holes (if present) on the wheelchair. If provided, install the wheelchair back cane screws as shown in Figure B.

Ensure the horizontal Velcro adjustment straps are facing rearward. Then install the cover making sure the padded side of the cover is next to the back of the client. Lastly, fold the back flap of the cover over the back of the strap assembly.

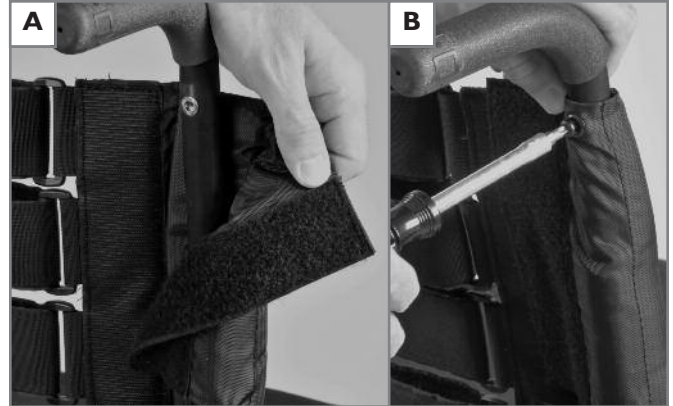

ADJUSTMENT

To adjust the tension of the JAY GO Back, first lift up the back cover flap to expose the horizontal Velcro straps (C). Then adjust each strap individually to the desired tension (D).

ATTENTION– AVOID over-tightening of the strap assembly as this may cause damage to the integrity of the back canes.

Lastly, fold down the back cover flap over the horizontal Velcro adjustment straps to complete your adjustment.

WARRANTY

24 MONTH LIMITED WARRANTY

Each JAY GO Back is carefully inspected and tested to provide peak performance. Every JAY GO Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.