

Invacare® Floor Base and Fixed Trapeze

en **Model Nos. 7714P and 7740A**
User Manual

I General

en

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage.



DANGER!

– Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.



WARNING!

– Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.



CAUTION!

– Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.



Gives useful tips, recommendations and information of efficient, trouble-free use.

2 Safety

2.1 General Guidelines



DANGER!

Risk Of Death, Injury Or Damage

Bed accessories designed by other manufacturers have NOT been tested by Invacare. Use of NON-Invacare bed accessories may result in injury or death.

– Use ONLY Invacare rails, mattresses, bed extenders and other accessories with Invacare bed products.



DANGER!

Risk Of Death, Injury Or Damage

Improper use of this product may cause injury or damage.

– If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment

– Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment.

Continued use of the bed with damaged parts could lead to the bed malfunctioning, causing injury to the user and/or caregiver.

– Check ALL bed components and carton for damage, and test components before use. In case of damage or if the bed is not working properly, contact a technician or Invacare for repair.

To avoid injury or product damage

– Set-up and Assembly instructions MUST be performed by qualified technicians only. "Qualified Technician Only" sections are marked accordingly.



WARNING!

Risk Of Injury Or Damage

The Fixed Trapeze has a safe working load of 168 lb.

The Fixed Trapeze is designed to provide support, increase stability and assist the user when repositioning himself/herself in bed.

– The trapeze is not designed to support the total body weight of an individual or to be used as an assist device when transferring into or out of a bed. Use the trapeze as a positioning device only.

– The Fixed Trapeze MUST ONLY be installed onto beds with metal framed headboards.

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.



Yes, you can.®



DANGER!


Risk Of Death or Injury

To avoid risk of falling or other injuries:

- After ANY adjustments, repair or service and BEFORE use, make sure that all attaching component parts are secure.
- DO NOT INSTALL OR ADJUST TRAPEZE OF AN OCCUPIED BED.
- DO NOT attach swivel trapezes to the floor base. This may cause the floor base to tip over.
- Floor Base MUST be positioned under the bed at the headboard position.
- Ensure that the snap buttons fully protrude through the adjustment hole of the floor leg assembly. This ensures that the Floor Base is properly installed and securely locked in position.

3 Setup

3.1 Assembling the Floor Base

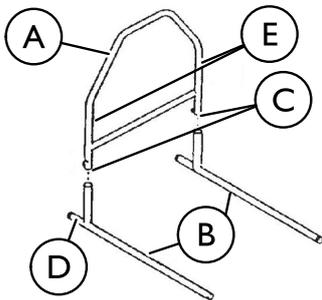

1. Install the "A" frame **A** into the floor base assembly **B** with the snap buttons **C** facing the short length **D** of the floor base.
2. Depress the snap button on one leg **E** of the "A" frame and slide it into the floor base assembly until an audible "click" is heard.




Make sure that the snap button fully protrudes through the adjustment hole of the floor base assembly. This ensures that the Floor Base is properly installed and securely locked in place.


3. Repeat STEPS 1 and 2 for the opposite side.
4. Attach brackets to the floor base. Refer to Attaching Brackets to Floor Base, page 2.

3.2 Attaching Brackets to Floor Base/Bed End


Attaching Brackets to Floor Base

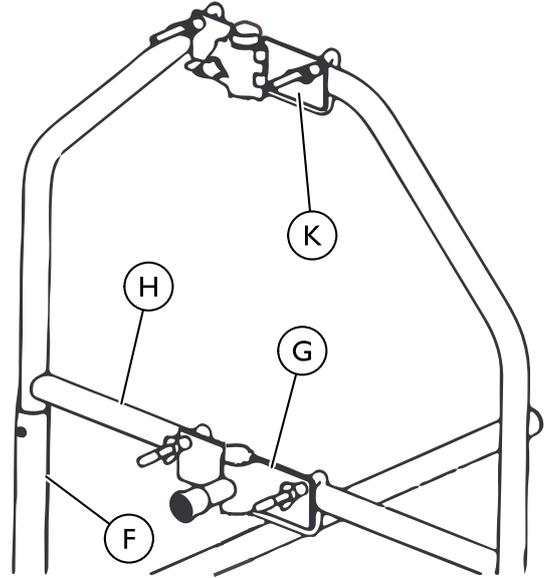

Upper Bracket

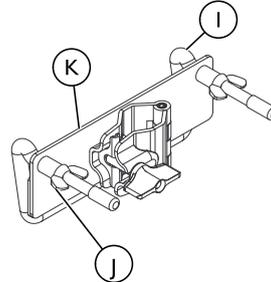


Lower Bracket

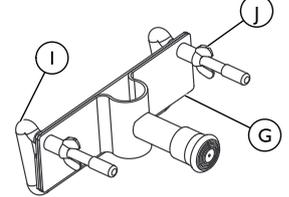


1. Place the floor base **F** on a flat surface.
2. Position the lower bed bracket **G** in the center of the bottom rail **H** of the floor base with the tubular hook **I** facing down.
3. Securely tighten the wing nuts **J** to secure the lower bed bracket to the floor base.
4. Position the upper bed bracket **K** in the center of the top rail of the floor base with the tubular hook facing down.
5. Securely tighten the wing nuts to secure the upper bed bracket to the floor base.


Attaching Brackets to the Bed End

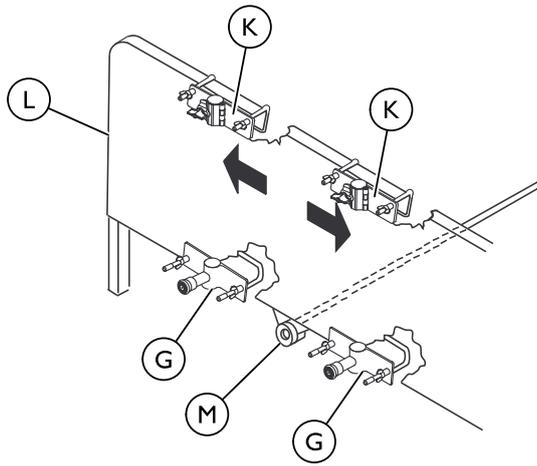

Upper Bracket

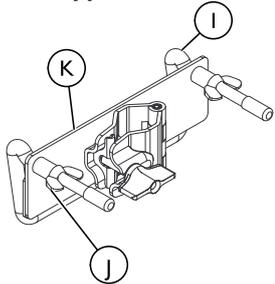

Lower Bracket

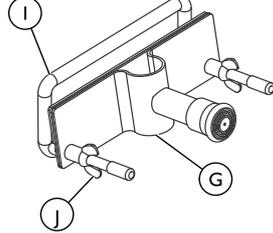

i Position the lower bed bracket **G** in the center of the bottom bed frame **L**. The lower bed and upper bed brackets **K** **MUST** be positioned as close to the center of the bed as possible without interfering with the bed drive shaft **M**.

1. Position the lower bed bracket **G** in the center of the bottom bed frame with the tubular hook **I** facing up. Make sure that there is no interference between the lower bed bracket and the bed drive shaft **M**.
2. Tighten wing nuts **J** to secure lower bed bracket onto bottom bed frame.
3. Position the tubular hook of the upper bed bracket **K** so it rests on the top rail of bed end with the tubular hook facing down.
4. Securely tighten the wing nuts to secure the upper bed bracket to the top rail of the bed end.

3.3 Installing the Trapeze




DANGER! Risk Of Death or Injury

To avoid risk of falling or other injuries:


- DO NOT INSTALL OR ADJUST TRAPEZE OF AN OCCUPIED BED.
- DO NOT attach swivel trapezes to the floor base. This may cause the floor base to tip over.
- Floor Base **MUST** be positioned under the bed at the headboard position.

i When using the floor base **F** in conjunction with a bed **L**, the long legs of the floor base should slide under the bed.

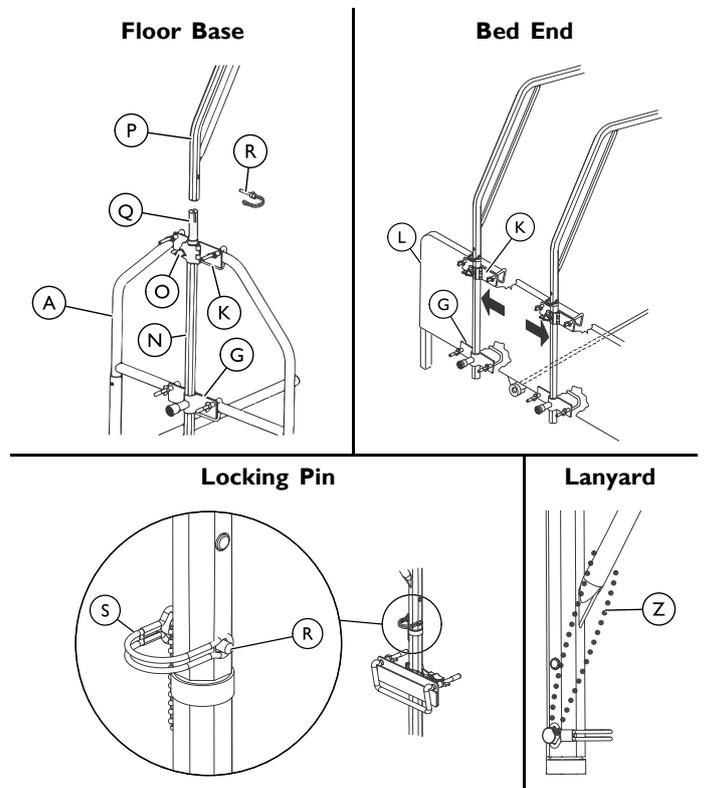

1. Install the upper bracket **K** and lower bracket **G** onto the floor base **F** or bed end **L**. Refer to Attaching Brackets to Floor Base, page 2 or Attaching Brackets to the Bed End, page 3.
2. Put the lower offset bar **N** through the clamp on the upper bed bracket and through the socket in the lower bed bracket.
3. Securely tighten the wing nut **O** on the clamp on the upper bed bracket to hold the lower offset bar in place.
4. Put the upper offset bar **P** onto the insert bar **Q** that is welded onto the lower offset bar.
5. Make sure the upper offset bar is fully seated.
6. Place the lock pin **R** through the holes in the upper offset bar and insert.
7. Secure the D-loop **S** around the pin.
8. Secure the lanyard **Z** through the lock pin loop and through the upper offset bar.
9. Adjust the height and/or position of the trapeze. Refer to 3.4 Installing/Adjusting the Grab Bar, page 3.

3.4 Installing/Adjusting the Grab Bar

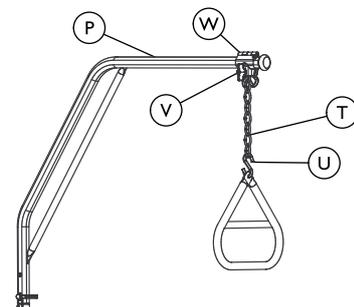


Height

1. Position any of the lower chain links **T** onto the "S" hook **U**.

Position

1. Loosen trapeze key **V** by turning counterclockwise.
2. Slide the clamp **W** to desired position on the upper offset bar **P**.
3. Secure the trapeze clamp to the offset bar by turning the key clockwise.

3.5 Storing the Floor Base

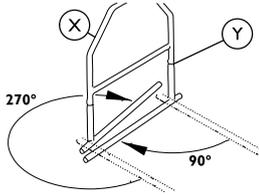

1. Remove the trapeze from the floor base.
2. Depress one snap button X on floor leg assembly and lift up on the "A" frame Y until the snap button clears the top of the floor leg assembly.
3. Rotate one leg of the floor base assembly 270° clockwise away from the other leg.
4. Rotate the remaining leg of the floor base assembly 90° clockwise towards the other leg.

4 Warranty

4.1 Warranty

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for a period of one year of the product for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare corporation will issue a return authorization. the defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. c.o.d. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

Invacare Corporation.

USA

One Invacare Way
Elyria, Ohio USA
44036-2125
800-333-6900

www.invacare.com

Canada

570 Matheson Blvd E. Unit 8
Mississauga, Ontario
L4Z 4G4
800-668-5324

© 2014 Invacare Corporation All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

1180736-B-01 2014-11-03



Making Life's Experiences Possible™



Yes, you can.®