

MAINTENANCE

- Check elastic cord before every use. Do not use cane if cord is cut, frayed, or damaged.
- Periodically check the screws on the feet to be sure they are tight.
- Replace the feet every six months. See HurryCane.com to purchase new feet.
- Enhance your mobility at HurryCane.com for official HurryCane® accessories.

SPECIFICATIONS

FDA-listed
Eight adjustable heights:
From 30 ½" to 37 ½"
Folded size: 13 ¾"

Comments, Feedback, or Questions

Toll-Free Number: 1-800-933-1136
E-mail: succes@hurrycane.com
HurryCane.com
Like us on Facebook: facebook.com/AllTerrainCane

RESET THE FLEX

The pivoting base of a new HurryCane® is designed to have less flexibility initially but will adapt to the user's movement over time. As the cane adapts to your movements, your HurryCane® may need to have its flex reset when you want the cane to stand beside you without holding on to it. Simply 'reset the flex' by following these 3 easy steps:

1 Pivot



Keeping base firmly on ground, move handle in a full circle.

2 Center



Move handle directly above base.

3 Release



Carefully let go.

NOTE: If the cane falls over, you can raise it easily by putting your foot on the base and pressing down.

STORAGE INSTRUCTIONS

