Heated Lumbar Support Soporte lumbar térmico Support lombaire chauffé

RTL2017CTL

Drive DeVilbiss Healthcare
99 Seaview Blvd.
Port Washington, NY 11050
1-877-224-0946
www.drivemedical.com

Heated Lumbar Support

COMFORT TOUCH™ Cushions by Drive™ are designed to help satisfy your therapeutic needs while immersing you in comfort with an ergonomic design, high-quality foam and luxurious look and feel.

Beneficial Uses

The Comfort Touch Heated Lumbar Support cushion delivers postural support while providing heated therapy to target areas of pain and tension in the lower back. The super soft luxurious cover enhances relaxation whether you're at home or at the office.

How to Use

Remove the Comfort Touch Heated Lumbar Support cushion from the retail box and check for any damage on velour cover, cushion, heating pad, control and cord. If the product is defective, contact Drive Customer Service 888-546-4998.

When ready to use check to make sure the heating pad is lying flat inside the cover and not folded as this can cause partial overheating. Secure the cushion to any chair using the strap on the back of the cushion. Plug into an outlet that is an AC 110V. Push the control button to the ON position; the indicator light will turn on and the lumbar cushion will begin heating. When not in use, push the control to the OFF position and unplug the cord. When storing product follow storage guidelines below. Product is intended for indoor use only.

CAUTION & WARNINGS

1. During consistent heating, if there are any feelings of discomfort or too much heat, push the control to the OFF position immediately and discontinue use.
2. If numbness or burning occurs, discontinue use and consult your physician immediately.
3. Refrain from using the cushion with heat therapy on any area that has been injured for at least 48 hours.
4. Do not allow children to use the product unsupervised.
5. Keep away from pets.
6. Never use heating pad without the Lumbar Cushion and cover.
7. Do not use in places with flammable vapors or gasses, high oxygen concentrations or other oxidizers. There is an increased risk of explosion if used in the presence of flammable anesthetic.
8. Intended for indoor use attached to an electrical source.

Electrical Precautions

Keep connections dry. Always keep the electrical components away from water. If the electrical components fall into water when plugged in, DO NOT touch any wet part of the product. Unplug only at dry electrical connections.

- Do not handle the electrical cord with wet hands.

- Connect to an 110V AC outlet only.
- Unplug product when not in use.
- Never leave unattended when plugged in.
- Do not operate — or unplug immediately — if plug, or cord, is damaged.
- Keep the cord and product away from heated surfaces.

Contraindications

- Consult your physician before using if you have a heat-related injury or adverse reactions to heat.
- Monitor your skin condition during therapy every 20 minutes; special care monitoring must be provided when using with children or those with adverse reactions to heat.
- Consult with your physician before use if you are experiencing wound healing problems, or have had multiple surgical procedures.

Drive's Comfort Touch Heated Lumbar Support cushion is intended to provide postural support and heat therapy for targeted lower back muscle pain and tension. **Seek medical attention immediately if symptoms persist or condition worsens.**

Storage Instructions

Allow the Comfort Touch Heated Lumbar Support cushion to cool down before storing away; ensure storage space is at room temperature. Do not stack other articles on top of the cushion as this may damage or distort the product. Never expose the cushion to chemical repellents while in storage, as this could accelerate deterioration of the cord.

Care and Cleaning Instructions

Cover: Unplug cushion. Unzip velour cover and open hook and loop fastener side panel. Remove the heating pad component fully from the cushion by pulling the cord and control unit through the side panel and out from the cover. Next remove the cushion by folding in half and pulling out from the cover. Wash the cover on a cold gentle cycle with mild soap as needed. Do not bleach. Do not wash with other articles. Tumble dry on low heat or hang to air dry.

Foam and Heating Pad: If the foam or heating pad has any odor or stains, surface clean it with a moist cloth and lay flat to air out until completely dry. No industrial solvents should be used at any time. Any lingering odor will dissipate over time. Never put foam or heating element in the washer or dryer. Never iron or air dry in the sun, as this may lead to deterioration of the heating wire and cause risk for potential danger.

**Note- foam is a porous material. Small holes seen on the foam itself are not a defect in the product but rather the texture and composition of foam.*

To Reassemble Lumbar Cushion: Insert folded foam into cover and position in place first. Using plug end, insert plug through cover opening and then through hook and loop fastener side panel. Pull the cord all the

Heated Lumbar Support Soporte lumbar térmico Support lombaire chauffé

RTL2017CTL

Drive DeVilbiss Healthcare
99 Seaview Blvd.
Port Washington, NY 11050
1-877-224-0946
www.drivemedical.com

way through including the control unit. Then insert heating pad over foam cushion ensuring it is properly in place and lying flat, not folded. Close side panel and zip.

One Year Limited Warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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