

vivehealth.com

#### WHAT'S INCLUDED

- 1x Shampoo Basin
- 1x Hand/Foot Pump
- 1x Shower Hose/Sprayer
- 1x Drain Pipe
- 1x Patch Kit
- 1x Shower Bag
- 1x S-Hook

#### **OVERVIEW**

We are constantly answering questions and recording helpful videos to make using your Vive Shampoo Basin as easy as possible. Check out the included links and QR codes to help you through the process.



To see all FAQ's in one place visit **vhealth.link/487** 

### **INSTRUCTIONS**



Want to watch us do this? Check out **vhealth.link/gydv** for an easy instructional video.

## Inflating the Shampoo Basin

 Remove the deflation attachment from the Hand/Foot Pump nozzle.



- Locate the inflation valves.
  Two are located on the front-right corner of the Shampoo Basin and the third is underneath.
- \*\* The two corner valves inflate the main chambers of the Shampoo Basin and the bottom valve inflates the headrest.





Insert the inflation attachment into any of the air valves to begin inflating.



- You can use the Hand/Foot Pump with either your hands or foot to inflate the Shampoo Basin.
- \*\* The hose must be securely attached to the blue knob on the Hand/Foot Pump for proper inflation.



5. Ensure that the product is inflated to your satisfaction, but do not overinflate.

### Filling the Shower Bag

When full, the Shower Bag can be heavy. It is recommended to hang your Shower Bag first then fill it with water.

1. To hang the Shower Bag, you can use the provided S-Hook to attach to a door.



2. Before filling, ensure that your Sprayer is set to the off position.



### Using the Sprayer

 Place the individual's head on the Shampoo Basin pillow comfortably.

**Note:** Before you begin applying water, ensure that there are no leaks and the air pressure is not declining.



2. Adjust the Sprayer from the off position to the on position.





You can adjust the water pressure as needed by moving the Sprayer lever between the on and off positions.

## **Emptying the Shampoo Basin**

When full, the Shampoo Basin can be heavy. You can use any of the following methods to empty the Shampoo Basin.

- Using the optional Drain Pipe (Recommended).
- a. Before attempting this method, ensure that you are close enough to a drain so that half of the Pipe will reach the drain.
- b. Submerge the Pipe completely in the Shampoo Basin.
- c. With most of the Pipe submerged, remove one end and direct toward a drain.

d. Allow the water to flow from the Drain Pipe into the drain.



- 2. Using the provided drain hole.
- a. This method is best used if the Shampoo Basin is set on a movable cart, or is easily transported without difficulty or spilling its contents.
- b. At a sink or bathtub, reach underneath the Shampoo Basin to find the drain hole.



c. Remove the plug to release the water.



- 3. Using a container
- a. When full, the Shampoo Basin can be heavy; this method allows you to remove some of the water with a smaller container to make the Shampoo Basin easier to carry.
- Use a measuring cup or other container to begin emptying the water.



- c. When light enough, you can carry the Shampoo Basin to a drain to empty the remaining water using the provided drain hole.
  - You can use the provided drain hole on the Shower Bag to empty any remaining water.



After you have removed most of the Shampoo Basin contents, it is recommended that you wash the inside with detergent or soap and then rinse. Then, dry the inside with a towel or paper towel to prevent any build-up.



## Deflating the Shampoo Basin

If you do not plan on using the Shampoo Basin for a long period of time, it is recommended that it is dried completely and deflated.

To deflate the Shampoo Basin, insert the deflation nozzle into the air valves and press down.



# Using the Patch Kit

If you notice a leak, please use the included patch kit.

 Completely dry the area to be patched, use a dry paper towel.



2. Using the adhesive and apply a small amount around the hole.



3. Then, quickly apply a patch to cover the hole.



4. Apply slight pressure to the patch to ensure it binds to the adhesive.



5. Wait a few hours for the patch to properly attach to the Shampoo Basin.

# **GOT MORE QUESTIONS?**

Check out our list of Frequently Asked Questions at **vhealth.link/487** for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



service@vivehealth.com



1-800-487-3808



vivehealth.com

Distributed by



8955 Fontana Del Sol Way Naples, FL 34109