ECG MONITOR

Owner's Manual

OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive ECG Monitor as easy as possible.



To see all FAQ's in one place visit **vhealth.link/spv**

WHAT'S INCLUDED

- ECG Device
- USB Charging Cable
- Storage Case

QUICK START GUIDE

- 1. Press and hold the button until the device powers on.
- 2. Hold both ends of the device so that the metal probes are firmly on your skin. You may follow any of the four methods illustrated below:









3. Hold the device in place until your results show on the screen.


Note: You will hear a few beeps that indicate heart beats, this does not mean that the reading is over. do not move the device until your final results show on the screen.

4. To see more detailed information about your results, please download our app.

To view between previous results on the device screen, press the button to toggle through old readings.


Pre-sample Interface

When the waveform becomes stable, the device will start formal sampling automatically, sample time countdown on the bottom right corner begins until finished one time sample and the color of sample time turns red.

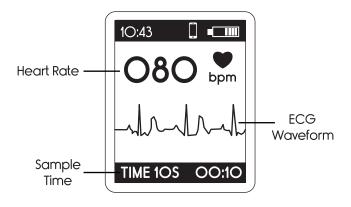

Formal Sample Interface

The device will enter into case review interface after completed sampling. Case review interface displays the sampling start time and heart rate.

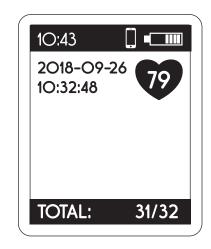

DOWNLOADING THE APP

You can easily access your data through our convenient App available on Apple and Android. Search for the Vive Precision app on the iOS and Android stores.



HOW TO USE THE APP

For downloadable app instructions and other valuable information on using your ECG visit www.vivehealth.com/apps

CHARGING THE DEVICE

At the top of the screen, there is a battery level indicator. Use the included USB charging cable to recharge the device.



WARNINGS /

- This device is not to be used by patients who have a pacemaker. If you have an internal electronic device, consult your physician before using an ECG monitor.
- This device is not designed or intended for complete diagnosis of cardiac conditions. This device should never be used as a basis for starting or modifying treatment without independent confirmation by professional medical examination.
- Always consult your physician before you begin or modify any exercise program
- This device does not detect or measure all heart rate, heart rhythm and heart waveform changes, especially those related to ischemic heart conditions.

- Do not attempt self-diagnosis or self-treatment based on the recording results and analysis.
- Interpretations made by this device are potential findings, not a complete diagnosis of cardiac conditions. All interpretations should be reviewed by a medical professional.
- Users should notify their doctor of a possible change in health: a labeling of the ECG as normal should not be relied on as a guarantee of absence of arrhythmias or other health conditions.
- The heart rate analysis is only valid if there is a valid rhythm (QRS complex visible).
- Do not use this device during an MRI scan
- Our device will not be performing any measurements that require validation through ANSI/AAMI EC 57.
- ECG reports viewed or printed at any magnification other than 100% may appear distorted and could lead to misdiagnosis.
- Do not use on humans less than 20kg.

QUESTIONS?

Our friendly customer service team is standing by to assist you. Feel free to connect on the phone, email or chat.



service@vivehealth.com



1-800-487-3808



vivehealth.com

