

STETHOSCOPE

QUICK START GUIDE


OVERVIEW

We are constantly answering questions and recording helpful videos to make using your Vive Precision Stethoscope as easy as possible. Check out the included links and QR codes to help you through the process.



To see all of the FAQs in one place vhealth.link/sgv

HOW TO SWITCH BETWEEN THE DIAPHRAGM AND BELL IN THE CHESTPIECE

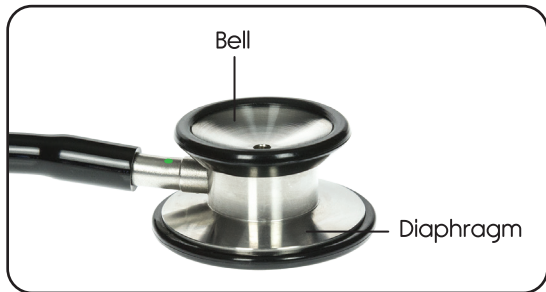

Want to watch us do this? Check out vhealth.link/34m for an easy instructional video.

1. Rotate the chestpiece clockwise or counter-clockwise to open and close the diaphragm or bell. A green dot on the metal tubing will indicate which side of the chestpiece is open for sound.



HOW TO WEAR THE HEADSET


1. Align the headset by making sure the silicone eartips point slightly forward. This ensures the silicone eartips align with your ear canals for comfort and maximum sound transmission.

2. Holding the headset in front of you with one ear tube in each hand, pull the headset open and place the silicone eartips into your ears.

Correct Positioning



Incorrect Positioning

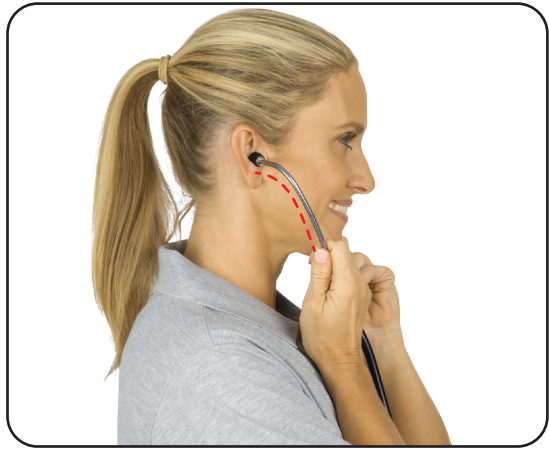

3. If the fit isn't comfortable, or acoustic performance is lacking, adjust the headset tension. Gently squeeze the ear tubes together to increase headset tension; pull them apart to decrease tension.

vive[®]
PRECISION

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com
Made in China