

THE LEADERS IN RECREATION, FITNESS, AND ABILITY!

9889 Garrymore Ln

Missoula, MT 59808

888-687-3552 | +1-406-549-0769

www.aquacreek.com



Aqua Creek Legrest Instructions

PART #: F-LRB (F-LRT / F-LRW / F-LRG)

MANDATORY

LEAVE THIS MANUAL WITH LIFT OWNER

Read and follow all instructions. Lift safety can only be ensured if the lift is installed and operated according to these instructions.

- NEVER permit children to play on or around the lift
- Do not allow children to use the lift without adult supervision
- NEVER apply direct water pressure to the electronics
- NEVER use the lift with a dry pool


READ CAREFULLY

Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.

Parts List

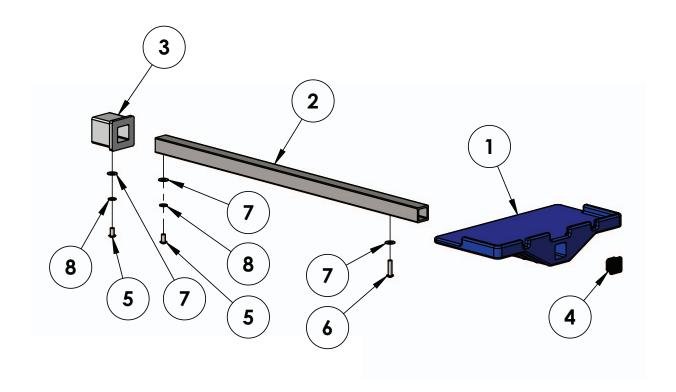

ITEM#	QTY	PART #	DESCRIPTION
1	1	P-ACFRB	PLASTIC FOOTREST, BLUE
2	1	P-505LRT-02	LEGREST TUBE
3	1	P-503LRPI-A	INSERT, PLASTIC, FOR LEGREST
4	1	SQR-1-14-20	CAP, PLASTIC, 1 X 1, BLACK
5	2	BB 1/4 X 1/2	BOLT, 316 SS, 1/4-20 X 1/2 BHSCS
6	1	BB 1/4 X 1	BOLT, 316 SS, 1/4-20 X 1 BHSCS
7	3	WF 1/4	WASHER, FLAT, 1/4, 316 SS
8	2	WL 1/4	LOCK WASHER, 1/4, 316 SS

Installation Instructions

The Aqua Creek legrest assembly is designed to fit under the chair and is installed inside the 2" square tubing that supports the chair from underneath. To install the legrest;

- 1. Insert the legrest assembly into the 2 x 2 square tubing that supports the chair until the plastic insert of the legrest assembly sits flush against the end of the 2 x 2 tube.
- 2. Using the supplied 1/4" hardware, fasten the legrest assembly to the 2 x 2 tubing using a 5/32" allen wrench.

