




Configure Your PL400EF for First Use

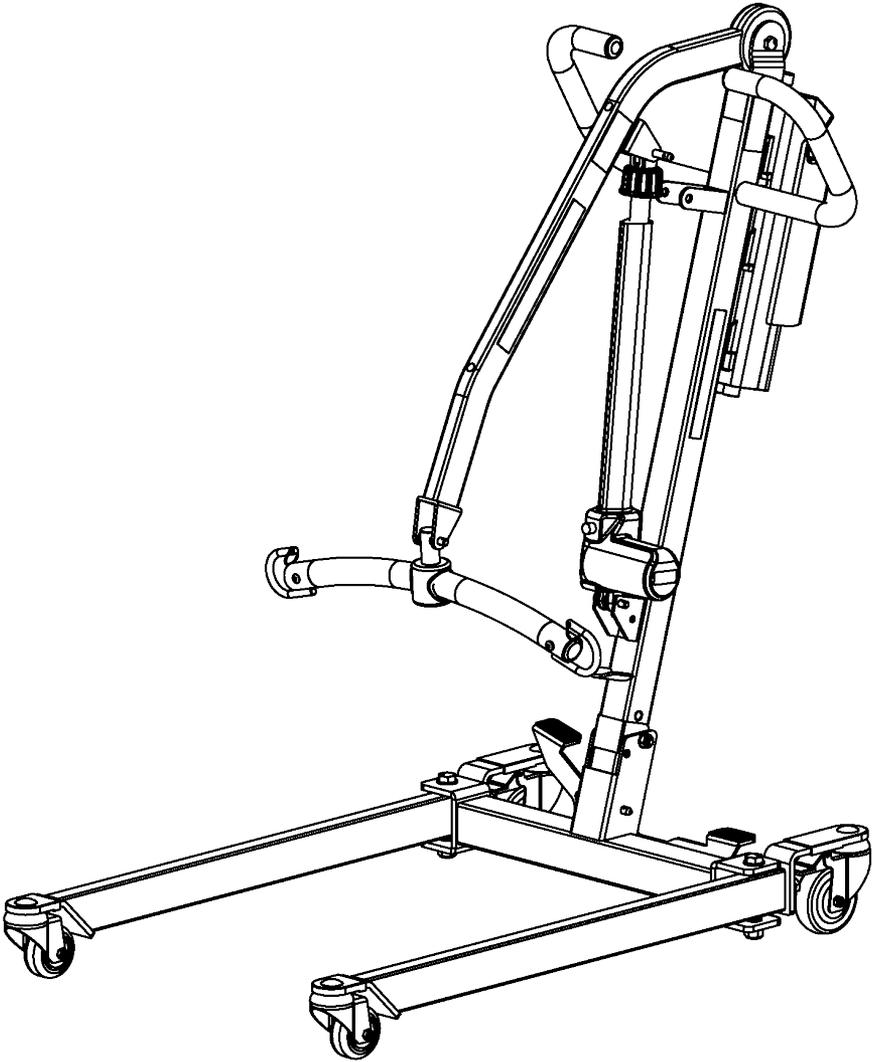


Step 1: Remove folded lift from box

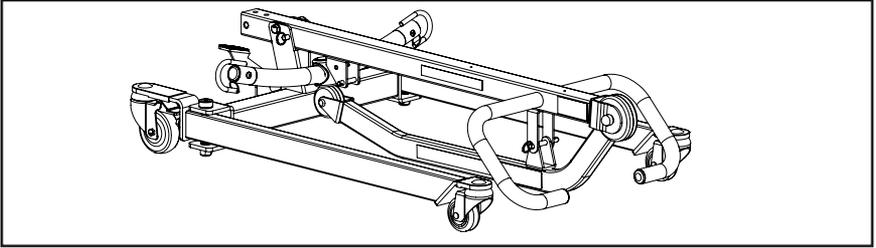


Step 2: Secure the stand to the mast. Hold the nut on its side with two fingers. Align the nut and the hole on the stand bracket with the lower hole on the mast. Make sure the tab on the stand bracket sit in the top hole on the mast, and insert the screw to the nut as follows. Use the EZ-Tools to tighten the screws.

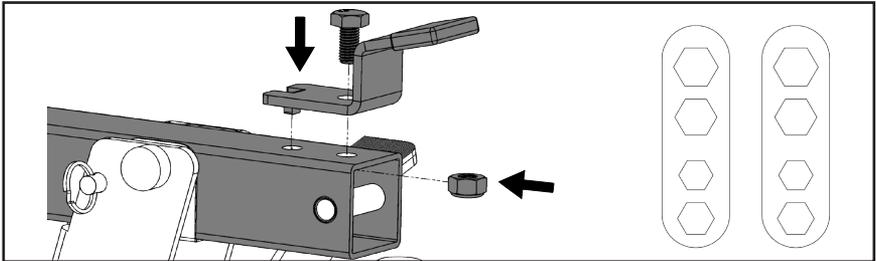


Step 3: Remove the quick release pin that is securing the mast

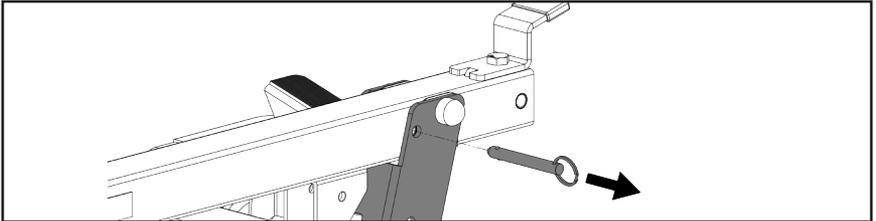


Step 4: Lift the mast upright

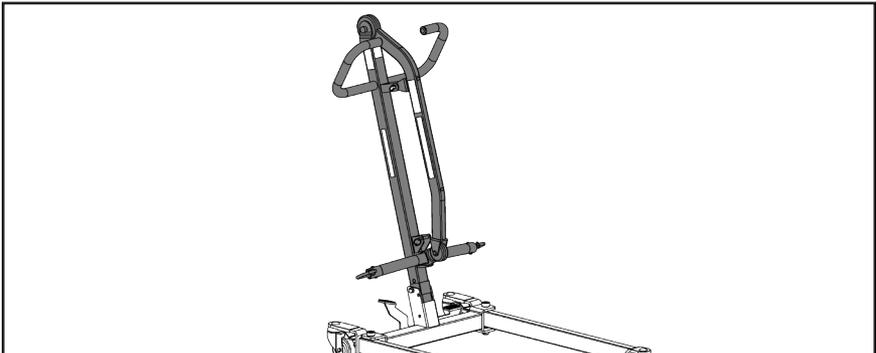


Step 5: Insert the pin to secure the mast

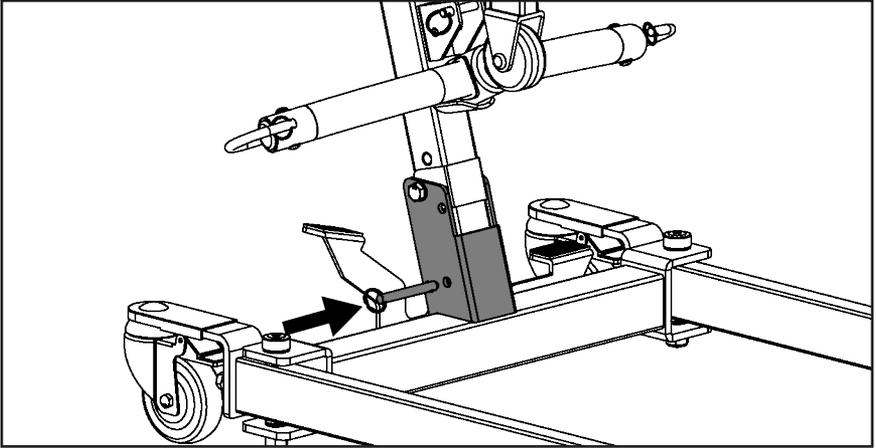


Step 6: Remove the pin that is securing the boom

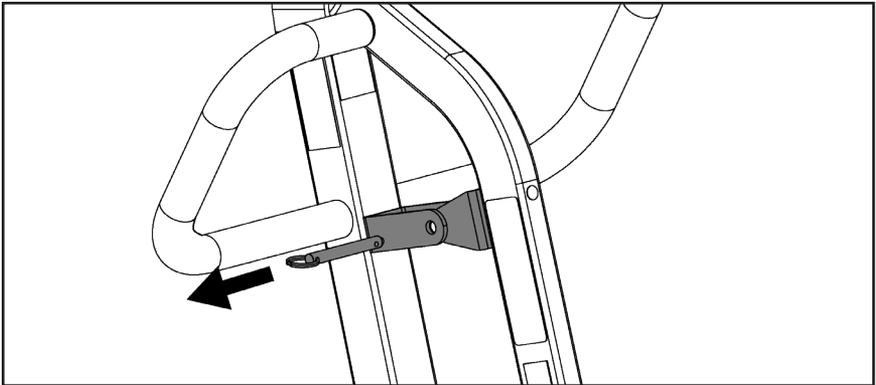


Step 7: Remove lower actuator pin

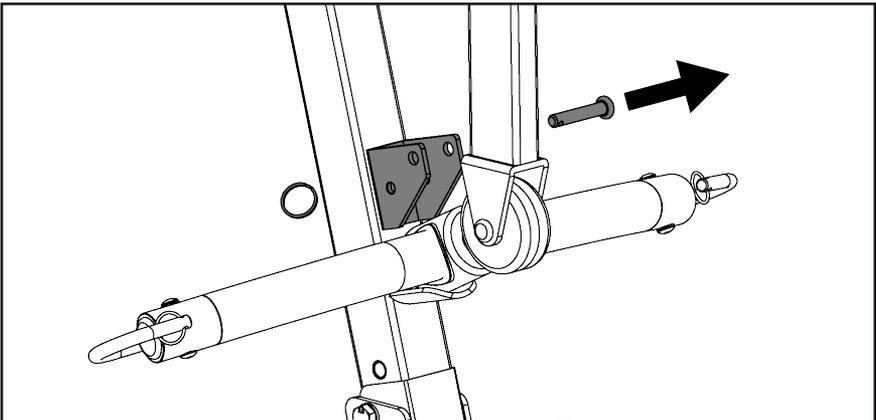

Step 8: Move the boom out of the way, and install the lower end of the actuator as follows

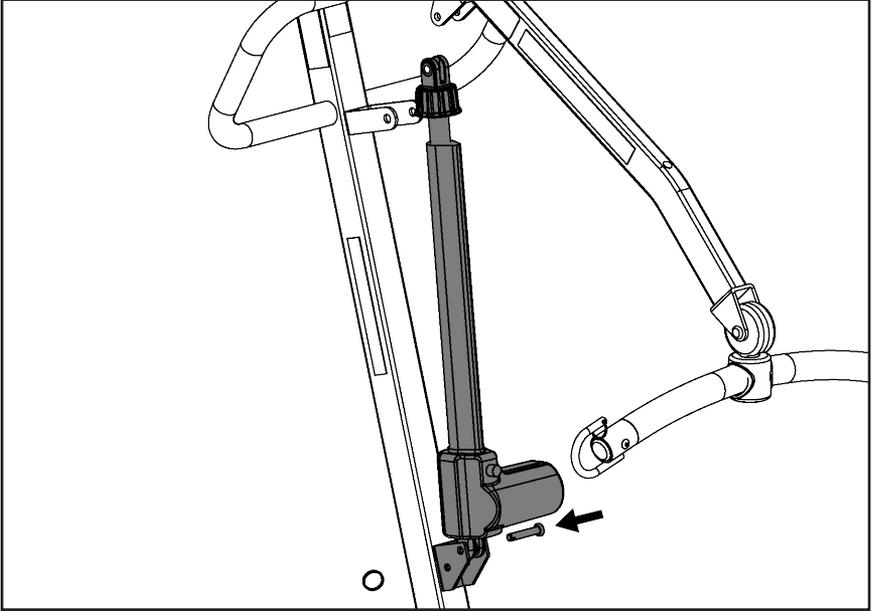

Step 9: Install the upper end of actuator and reinsert the quick release pin to secure it.

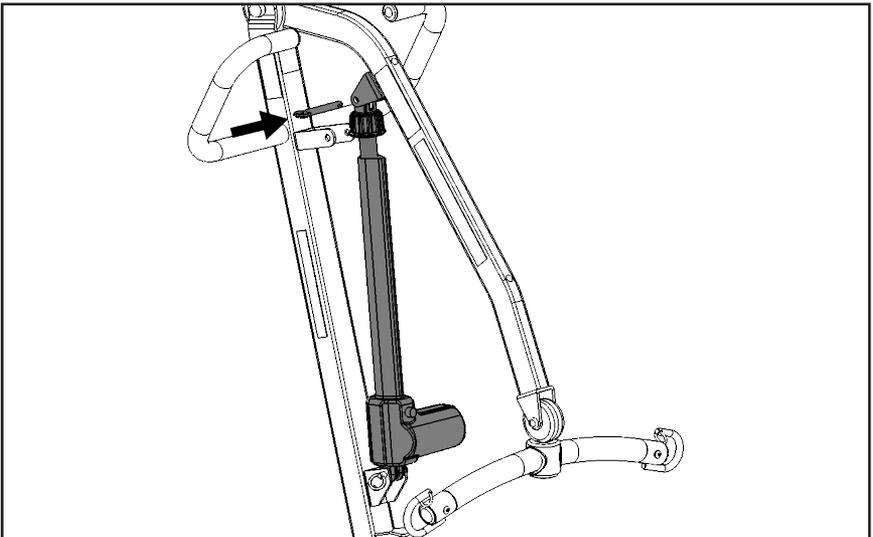

Step 10: Refer to “Quick Setup Guide to TiMotion Electronics” to install the controller.