THE EMPOWER ROLLATOR LE DÉAMBULATEUR EMPOWER ANDADOR RODANTE EMPOWER

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Table of Contents

Safety Instructions	3
Assembly Instructions	4-5
Rear Wheel Assembly	6
Front Wheel Assembly.....	7
Brake Handle Assembly.....	8-10
Adjustment of Brakes	11
Operating Instructions	12
Maintenance	12

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Safety Instructions

To ensure your safety in using the Medline rollator, the following safety information and all instructions must be followed.

GENERAL WARNINGS AND CAUTIONS:

- ⚠ **DO NOT use this product without first reading and understanding the instructions contained in this manual.** If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer, or technician before use. Failure to do this may result in bodily injury or damage to the device.
- ⚠ **DO NOT** self-propel the rollator while seated.
- ⚠ Rollators are intended for individual use only and are **NOT TO BE USED AS A WHEELCHAIR**. Serious injury to the user and/or damage to the rollator's frame or wheels may result from improper use.
- ⚠ **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
- ✓ **CAUTION:** The backrest is not intended to support full body weight. Avoid leaning while sitting as this may result in a fall. Use **EXTREME** caution when shifting weight while seated against the backrest. This may cause the rollator to tip.
- ✓ All wheels must be in contact with the floor and/or ground at **ALL TIMES**. This will ensure the rollator is properly balanced.
- ⚠ **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result:
Maximum weight capacity: 300 lbs (136 kg)
 - Items should be placed completely in the tote bag and should not protrude. Excess weight may cause the rollator to tip, resulting in bodily injury or rollator damage.
 - The tote bag located under the seat has a weight limitation of 10 lbs (4.5 kg).
- ✓ Before each use make sure the rollator frame is fully and securely opened.
- ✓ Periodically wipe down the rollator with a soft cloth and non abrasive cleaner.

SEAT WARNINGS:

- ⚠ The brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- ⚠ **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- ⚠ **DO NOT** use the seat to transport people or objects.
- ⚠ **DO NOT** sit on the seat when the rollator is on an incline.
- ⚠ The rollator was designed to be used as a walking aid. **DO NOT** use the rollator in a manner that could cause personal injury; such as a scooter, a knee walker, or a jumping/hopping aid.
- ⚠ **DO NOT** stick fingers or hands near the seat slide.

BRAKE WARNINGS:

- ⚠ Check the brakes for proper operation. Make sure that all parts are secure and that the moving parts are in good working order.
- ⚠ Wheels should roll easily and not wobble.
- ⚠ When rising from a lower position, **DO NOT** use the rollator to provide assistance to the user. If the rollator is used to pull the user up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
- ⚠ **DO NOT** push the rollator while brakes are enabled. Brakes are to be used to keep the wheels from rolling. However, it may not prevent the rollator from moving if it is pushed.

Assembly Instructions

Assembled Rollator

- A. Handgrips
- B. Adjustable Handlebar
- C. Handlebar Height Adjust Lock
- D. Back Rest
- E. Padded Seat
- F. Handlebar Folding Mechanism
- G. Adjustment Knob
- H. Brake Handle/Lock
- I. Brake Cable
- J. Triangular Knob Screw
- K. 8" (203 mm) Front Wheels
- L. 8" (203 mm) Rear Wheels
- M. Brake Shoe

Figure 1
Assembled Rollator

Rollator Components


- 1 Rollator Frame
- 2 Front Wheels
- 2 Handlebars
- 2 Rear Wheels
- 4 Triangular Knob Screws
- 1 Backrest
- 1 Seat
- 1 Bag

Assembly Instructions

1. Carefully remove the rollator components from the shipping carton.
2. Refer to diagram (Figure 1) as needed.
 - 1 Rollator Frame
 - 4 Triangular Hand Screws
 - 2 Front Wheels
 - 2 Rear Wheels (attached to brake cables)
3. Examine all parts for shipping damage. **DO NOT** use the rollator if there is apparent damage or if any parts (other than the bag) are missing. If there are missing or damaged parts, please call Medline Customer Care toll free at 1-800-MEDLINE.
4. Remove plastic cover from the wheels before use.
5. Check all of the wheels to see that they are in good working order and roll freely.
6. Turn the frame over on its side.
7. Unfold the rollator by grabbing a front leg and rear leg and pull them apart.

When inserting and adjusting the legs and handles, use the following guidance and the height measurements below.

1. Measure from the ground to the crease in the back of the knee. (Dimension A in figure above)
2. Adjust the rollator legs accordingly so the seat is at same height as measurement taken above.
3. Keeping arms at your side with a natural bend at the elbow, measure from the ground to the center of the wrist. (See Dimension B in figure above)

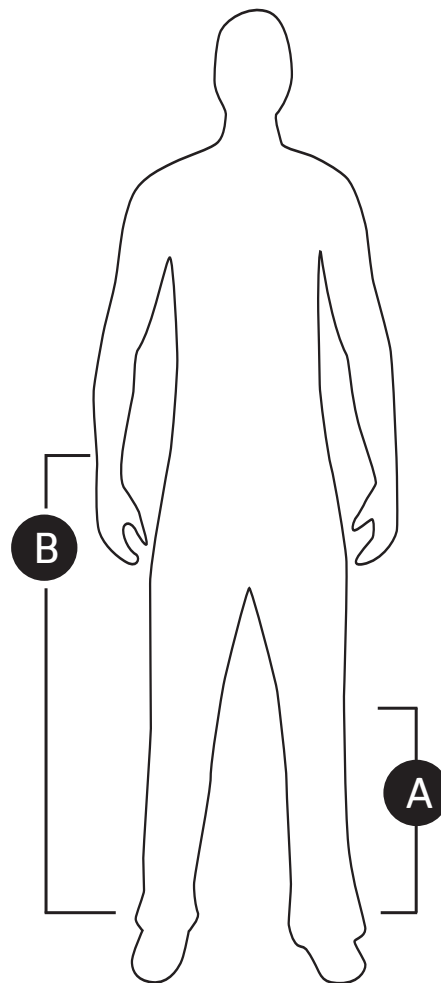Rear Wheel Assembly

NOTE: The rear wheels are the ones with the brake shoes on them.

- Ensure these are mounted so that the wheel/brake is directly below the brake handle as shown. **Figure 1.**
- Ensure that the bend of the wheel leg piece bends away from the center of the rollator shown. **Figure 2.**

Figure 1

Figure 2

1. Insert wheel assembly to desired height and use chart 1 (page 7) for recommended height settings. **Figure 3.**
2. Insert triangular hand screw into hole on same side as the wheel is on so that the triangular hand screw stays outside of walker frame. **Figure 4.**
3. Tighten triangular hand screw until secure (**Hand Tighten Only!**). **Figure 5.**
4. Repeat steps 1-3 for other rear wheel.

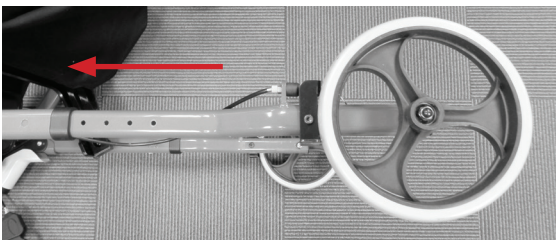
Figure 3

Figure 4

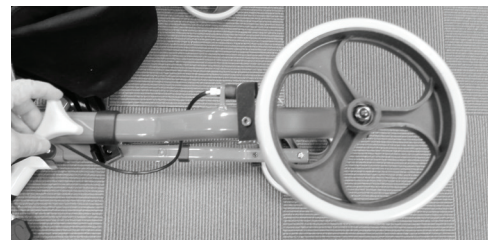
Figure 5

NOTE: Make sure the brake shoe and the triangle knob are to the outside. The leg should bend away from the center of the rollator.

Front Wheel Assembly

NOTE: The front wheels are the ones that swivel.

- Ensure these are mounted so that the bend turns toward the floor. **Figure 6.**

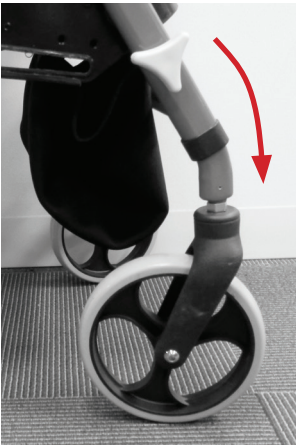
Figure 6

1. Insert wheel to desired height depicted by numbered holes (set to the same number as rear wheels). **Figure 7.**
2. Place triangular hand screw into hole. Make sure triangular hand screw stays on the outside of rollator frame. **Figure 8.**
3. Tighten triangular screw until secure (**Important: Hand Tighten Only!**). **Figure 8.**
4. Repeat steps 1-3 for other front wheel.

Figure 7

Figure 8

Chart 1

Seat Heights provided by leg adjustments (set to Dimension A above):

Seat Height	Legs				
	Hole 1	Hole 2	Hole 3	Hole 4	Hole 5
	25" (63.5 cm)	24" (61 cm)	23" (58.5 cm)	22" (56 cm)	21" (53.3 cm)

Note: 1 is the highest, 5 is the shortest setting.

Brake Handle Assembly

- 1. Press and hold the yellow button in (item C in Figure 1 above and Figure 9 below).
- 2. Insert hand grip into appropriate hole (see Figure 10).
- Note:** Keep brake cable to the outside of the walker.
- 3. Repeat steps 1-2 for other hand grip.

Figure 9

Figure 10

Chart 2

Handle Height adjustments (set to Dimension B above):


		Legs				
Arms		Hole 1	Hole 2	Hole 3	Hole 4	Hole 5
	Hole 1	39" (99.1 cm)	38" (96.5 cm)	37" (94 cm)	36" (91.4 cm)	35" (88.9 cm)
	Hole 2	38.25" (97.2 cm)	37.25" (94.6 cm)	36.25" (92.1 cm)	35.25" (89.5 cm)	34.25" (87 cm)
	Hole 3	37.5" (95.3 cm)	36.5" (92.7 cm)	35.5" (90.2 cm)	34.5" (87.6 cm)	33.5" (85.1 cm)
	Hole 4	36.75" (93.3 cm)	35.75" (90.8 cm)	34.75" (88.3 cm)	33.75" (85.7 cm)	32.75" (83.2 cm)
	Hole 5	36" (91.4 cm)	35" (88.9 cm)	34" (86.4 cm)	33" (83.8 cm)	32" (81.3 cm)
	Hole 6	35.25" (89.5 cm)	34.25" (87 cm)	33.25" (84.5 cm)	32.25" (81.9 cm)	31.25" (79.4 cm)

Note: 1 is the highest, 5 is the shortest setting.

Adjust the backrest height for comfort:


- 1. Pull the Backrest Height Adjust Lock (item G in Figure 1 above).
Raise or lower the backrest to desired height.
- 2. Release the Backrest Height Adjustment Knob. Move backrest to desired height. Place knob firmly into appropriate hole.
Ensure backrest is firmly locked in place.
- 3. Repeat steps 1-2 for the other side.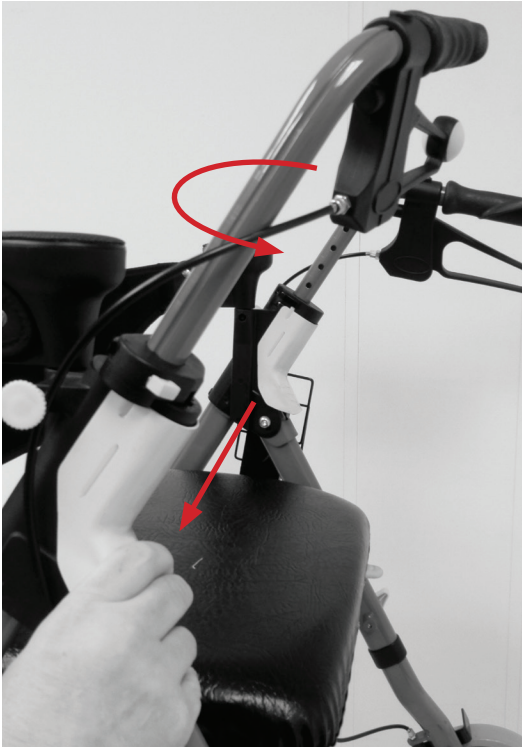


Folding the Rollator

1. Push down on the Handle Bar Folding Mechanism and hold (**Item F in Parts Guide, Page 4**).
2. Rotate the handle to the center of the rollator.
3. Release the Handle Bar Swivel Lock.
4. Repeat steps 1-3 for the other side.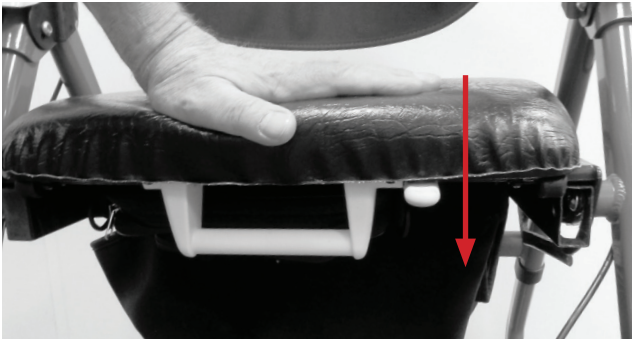

5. Ensure the seat is latched closed by pushing down on the seat.

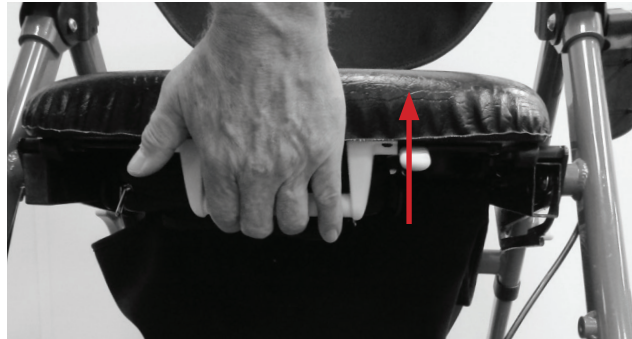6. Pick up on the large handle.

7. The Rollator should fold and lay flat.

Checking the Brakes

Before each use, check to make sure the brakes are working properly.

1. Applying pressure on the brake handles using a pulling up motion the brakes will stop the wheels until the handle is released.

2. Pushing down on the brake handles will lock the wheels in place permanently. (To lock the brakes, push down on the brake handles until you feel and hear the brakes click into the locked position.)

3. Pull up to release brake lock.

Adjustment of Brakes:

(There are 2 adjustments for each brake - one at the handle and one at the brake)

When adjusting the brakes on the rollator, keep in mind if you adjust it too far one way or the other, the lock will not function properly. Please test thoroughly and tighten all parts completely prior to using rollator.

1. At the top of the rollator, find the adjustment nut which is located where the cable goes into the brake handle. This part is usually silver.
 - If the brake is too tight, at the handles of the rollator, check to see if there is a gap between the adjustment nut and thumbscrew. If there is, turn the adjustment nut counter-clockwise until it is flush up against the thumbscrew (You may need pliers to get started). Then rotate both together clockwise until it is tight against the handle. This should loosen the brake. If there is no gap and it is already tight against the handle, no further adjustment can be made. (Refer to Step 3 below.)
 - If the brake is too loose (not locking), standing at the handles of the rollator, turn the adjustment nut and thumbscrew counter-clockwise (You may need pliers to get started). The thumbscrew and adjustment nut will push away from the handle and this will tighten the brake. Once the brake is adjusted properly, rotate just the adjustment nut clockwise until snug against the handle. There should now be a gap between the adjustment nut and the thumbscrew.
2. At the bottom of the rollator, find the adjustment nut similar to the one located at the top. Hold the brake pad against the wheel. This will take the tension off of the cable.
 - If the brake is too tight, check to see if there is a gap between the adjustment nut and thumb screw. If there is, pull up on the cable, and looking at the adjustment nut from the top, turn the adjustment nut counter-clockwise, screwing the adjustment nut to the thumb screw. Adjust to correct tension. If there is no gap, no further adjustment can be made here. (Refer to Step 3 below.)
 - If the brake is too loose (not locking), pull up on the cable, then, looking at the adjustment nut from the top, turn the adjustment nut clockwise, screwing the adjustment nut to the bracket. There will now be a gap between the thumbscrew and the adjustment nut. Adjust to correct tension.

Operating Instructions

Using the rollator as a walker:

For proper positioning and use of the rollator, imagine a line drawn across the back of the rear wheels. Part of one foot should be slightly ahead of that line as you walk.

Using the rollator as a seat:

- Push the rollator to the desired location.
- **LOCK** the brakes **BEFORE** sitting.

To Operate the Brakes:

- To slow down, squeeze the brake handles up.
- To lock the brakes, push down on the brake handles until you feel and hear the brakes click into the locked position.
- **CAUTION: If the brakes have been adjusted but the rear wheels on the rollator move while the brakes are in the locked position, DO NOT use the rollator. Please call our toll-free Customer Service number at 1-800-MEDLINE for turns or repairs.**
- To release the brakes from the locked position, simply pull up on the brake handles.

To Access the Storage Bag under the Seat:

- Lift the release lever on the front edge of the seat.

Maintenance

The Medline rollator should be checked periodically to ensure the brakes are functioning properly and that all nuts, screws and bolts are secure.

If the brakes do not lock the wheels, the brakes need to be adjusted. If adjustment does not tighten the brakes, **DO NOT** use the rollator. Call Customer Service 1-800-MEDLINE.

Keep your receipt!

Contact us at 1-800-MEDLINE, Monday – Friday 8am to 5pm CST if you encounter any problems or for information regarding repairs or maintenance.

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LIFETIME WARRANTY	6 MONTH WARRANTY
FRAME	HAND BRAKES BRAKE CABLES PUSH-BUTTONS BACKREST BRAKES UPHOLSTERY WHEELS (FRONT AND REAR) MEDLINE BAG TRIANGULAR HAND SCREWS ALL OTHER PARTS

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Lifetime Limited Warranty

Your Medline rollator frame is warranted to be free of defects in material and workmanship for a lifetime for the original consumer. This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components such as plastic accessories, brakes and grips, which are subject to normal wear and replacement. Non-durable components are warranted for 6 months. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

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Medline Industries, Inc.

Three Lakes Drive, Northfield, IL 60093 USA

1-800-MEDLINE (633-5463)

medline.com | info@medline.com

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Medline United States

1-800-MEDLINE (633-5463)

www.medline.com | info@medline.com

Medline Canada

1-800-396-6996

www.medline.ca | canada@medline.com

Medline México

01-800-831-0898

www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information.

We reserve the right to correct any errors that may occur within this manual.

©2018 Medline Industries, Inc. Medline is a registered trademark of Medline Industries, Inc.

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© Medline Industries, Inc. 2018 Medline est une marque déposée de Medline Industries, Inc.
©2018 Medline Industries, Inc. Medline es una marca registrada de Medline Industries, Inc.