
CARE AND MAINTENANCE

Your Zip'r3 requires a minimal amount of maintenance. The following are the maintenance items that should be followed to ensure that your scooter is safe and reliable.

Daily

- Charge your battery daily if your scooter is used for more than twenty minutes.
- Always store your scooter in a warm and dry environment.

Cleaning

- The scooter shroud is made from durable ABS plastic with a high gloss type finish. Therefore, the body can be cleaned like a car (cool water and mild soap to remove dirt/oils). DO NOT SPRAY THE ENTIRE SCOOTER WITH WATER AS THIS WILL DAMAGE ELECTRICAL PARTS AND UPHOLSTERY. A light application of car wax will help the shroud retain its high gloss.
- Use a rubber conditioner on your scooter's tire sidewalls to help preserve them.
- Bumpers and trim benefit from an occasional application of rubber or vinyl conditioner.

HAZARD WARNING

- DO NOT APPLY RUBBER CONDITIONER ON TIRE TREAD; THIS MAY CAUSE THE TIRES TO BECOME DANGEROUSLY SLIPPERY. ALSO DO NOT USE A RUBBER OR VINYL CONDITIONER ON THE SCOOTER'S VINYL SEAT OR FLOORBOARD. THEY WILL BECOME DANGEROUSLY SLIPPERY AND RESULT IN PERSONAL INJURY AND/OR DAMAGE YOUR SCOOTER.
- ALWAYS PROTECT BATTERIES FROM FREEZING TEMPERATURES AND NEVER CHARGE A FROZEN BATTERY. THIS DAMAGES THE BATTERY AND CAN CAUSE PERSONAL INJURY.

Maintenance

- Regularly check your scooter tires for wear.
- Regularly check you scooter seat posts for wear.
- All axle bearings and motor/transaxle assembly are prelubricated, sealed and require no subsequent lubrication.
- Keep the console, charger, battery, and electronics free of moisture.
- For prolonged storage, you may wish to place several boards under the frame of your scooter to raise off of the ground. This reduces the possibility of flat spots developing on the flat free tires.