


# *ScootaTrailer®*

*Manufactured in the U.S.A.*

## Top Instructions

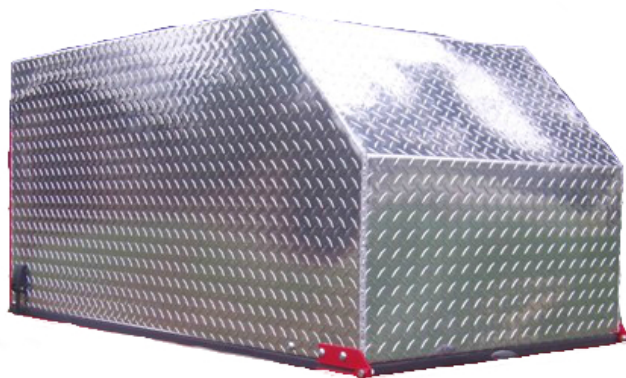

ScootaTrailer, LLC

Any car. Any Scooter. Anywhere.™



## **Tools Needed:**

1. Two 9/16" wrenches
2. Knife or scissors

## **Step 1:**

- Remove the bolts, nylon washer and lock nut from the red top hinge.
- Set the aluminum top onto the trailer with the top hinges to the inside of the trailer hinge (Illustration 1)
- Hold the nylon washer in between the hinges and put the bolt through, proceed to screw on the nylon lock nut. Push down on the top to compress the top to align the holes

## **Step 2:**

- Tighten the nuts and bolts with the two (2) 9/16" wrenches until the hinges and nylon washer has no space between them. Be sure to not over tighten, so that the top may freely move up and down

## **Step 3:**

- Lift the aluminum top after the bolts have been correctly assembled and cut the zip ties attached to the top
- Attach top ball stud end to the trailer ball stud until it snaps (Illustration 2)

### **\*Tip\***

To make assembling the aluminum top easier, it would help out for someone to lift up on the top at the rear of the trailer, helping to slide the 3/8" bolt through the hinges



Illustration 1

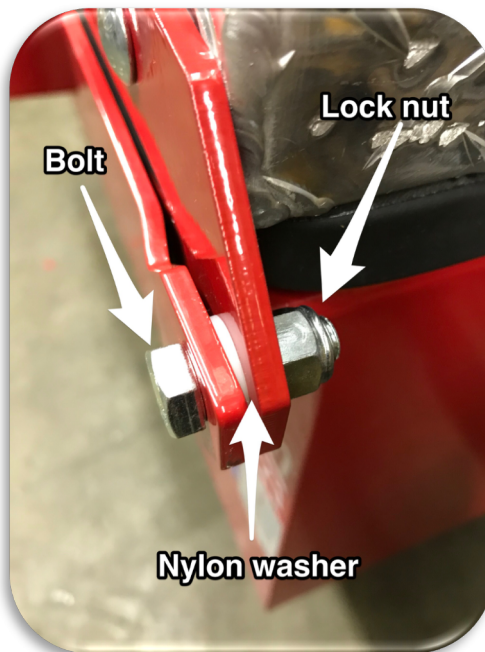


Illustration 2

