



Supplying Hospitals
for Over 90 Years

User Guide

MDS6800BA

Bed Assist Bar with Nylon Storage Pocket




Contents

Assembly Instructions	3-5
General Warnings	5
Warranty	7

ASSEMBLY INSTRUCTIONS

DO NOT install this product without first reading and understanding this instruction sheet. If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to install this equipment. Failure to follow instructions may result in serious injury.

1. Open the box and remove the contents.

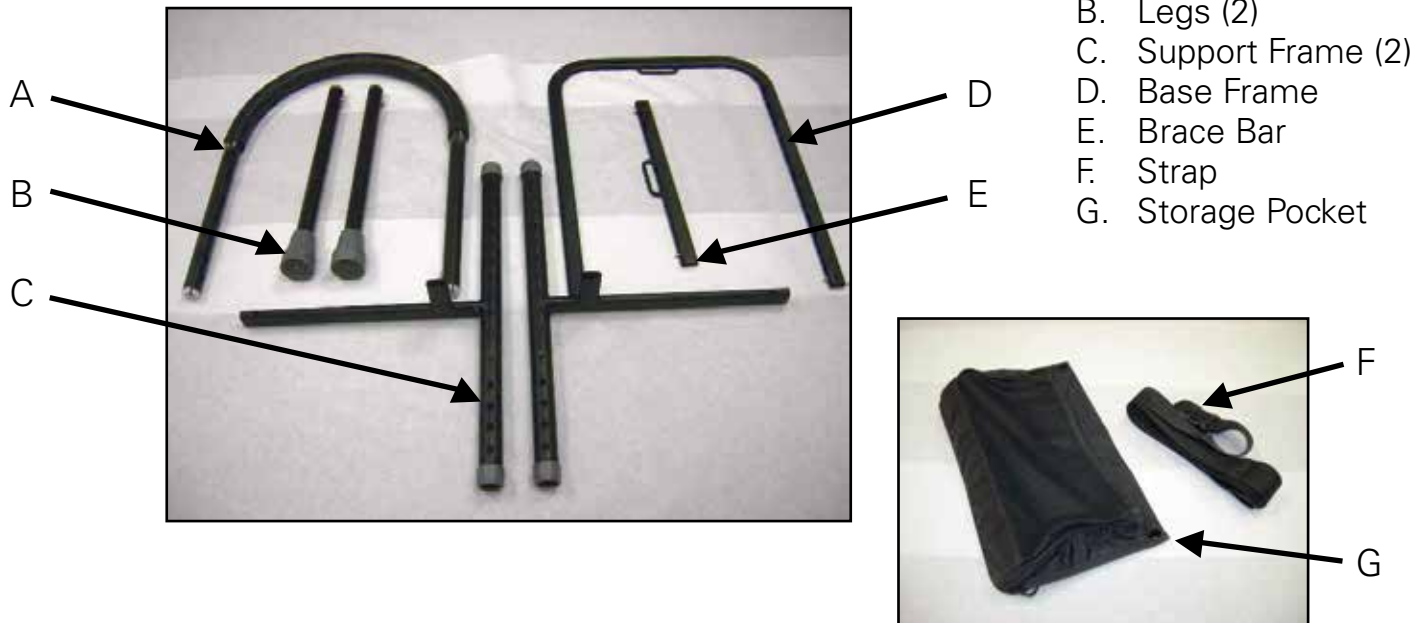

2. Gather parts C, D, and E.



3. Assemble the brace bar (E) to the 2 Support Frame pieces (C). Make sure that the arrow on the two support frame pieces is pointing up.



4. Attach the Base Frame (D) to the Support Frame pieces (C).



5. Attach the Legs (B) to the Support Frame pieces (C).



6. Attach the Assist Bar (A) to the other side of the Support Frame pieces (C). Notice the clips on the Assist Bar just below the foam padding. The clips should face outward as these are what the storage pocket hangs on.



7. Attach the Storage Pocket (G) to the outside of the Assist Bar (A).



8. Take the mattress off the bed. Place the assembled bed assist at a desired location on your bed. Bed assist will sit between the bed frame and the mattress, or between box spring and the mattress. The legs will need adjusting depending on the height of your bed frame (Fig.1). Make sure that the legs are firmly touching the ground. Once in place, take the strap (F) and thread it through the 2 loops in the base (Fig.2). Depending on the size of your bed you might be able to only loop it through one loop in order to reach the opposite side of the bed (on king size beds).

Fig.1


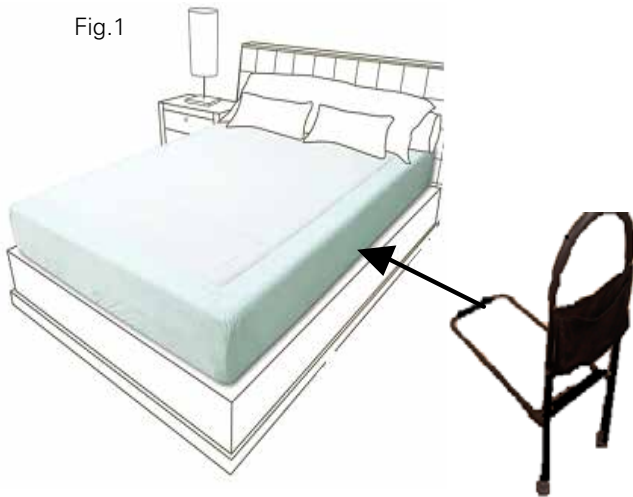


Fig.2

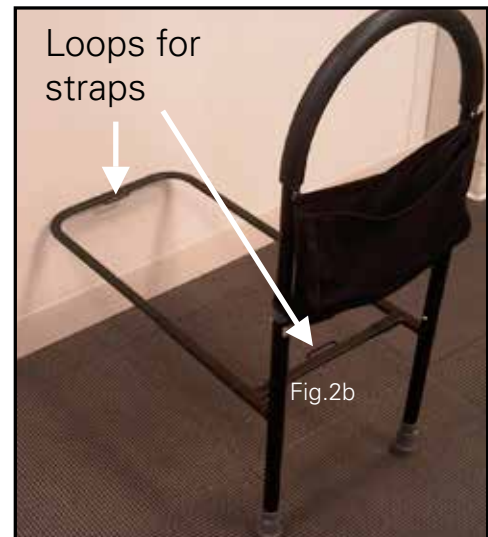

9. Secure the strap around the bed frame on the opposite side of the bed and tighten.
10. Place the mattress back on the bed.
11. Depending on the width of your mattress the top of the assist bar might need height adjustment, adjust to desired height by pulling out the e-clip (Fig.2b) and pushing in the push button.

GENERAL WARNINGS

- Before use ensure that all push buttons and clips are securely in place and that the legs are leveled and are securely touching the ground.
- This unit is not meant to support full body weight but to assist with stability when getting in or out of the bed.
- This unit is not meant to be used as a restraint for preventing people from rolling out of bed.

Lifetime Limited Warranty

Your Medline bed assist bar is warranted to be free of defects in material and workmanship for a lifetime for the original consumer. This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the material and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse, negligence, or normal wear and tear. The warranty does not extend to non-durable components such as plastic accessories, brakes and grips, which are subject to normal wear and replacement. Non-durable components are warranted for 6 months. If you have a question about your Medline product or this warranty, please call us at 1-800-MEDLINE. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

LIFETIME WARRANTY	6 MONTH WARRANTY
FRAME	ASSIST BAR GRIP EXTENSION LEG TIPS PLASTIC COMPONENTS STRAP STORAGE POCKET



Medline Industries, Inc.

One Medline Place
Mundelein, IL 60060

Medline United States
1-800-MEDLINE (633-5463)
www.medline.com | info@medline.com

Medline Canada
1-800-396-6996
www.medline.ca | canada@medline.com

Medline México
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information.
We reserve the right to correct any errors that may occur within this manual. ©2009 Medline Industries, Inc. Medline is a registered trademark of Medline Industries, Inc.