© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

universal cup holder
portavasos universal
porte-gobelet universel



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation instructions

1. Remove contents from box.
2. Place U-shaped section (A) around pole at desired height.
3. Remove rubber insert (B) if necessary to fit tubing diameter.
4. Thread scw through holes and fasten using tightening knob (C). Swivel cup to the desired angle then tighten the knob.
5. DO NOT over tighten.

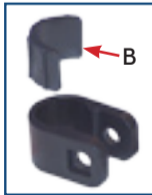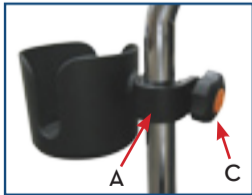1 year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This 1 Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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