


THE CLEANCUT

ULTRA-LOW

SIZING & MEASUREMENT GUIDE


TUB REQUIREMENTS:

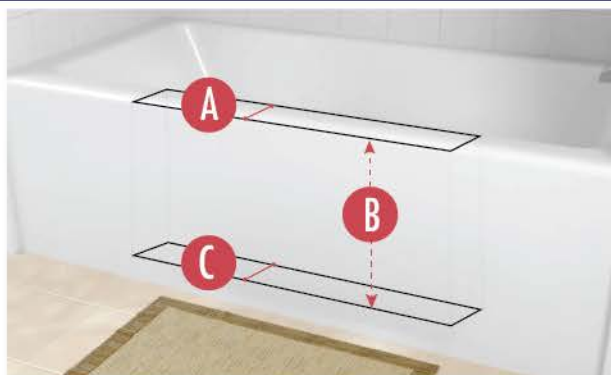

- A** **MAX* TUB-WALL WIDTH AT TOP:** 6 Inches
- B** **BOTTOM WIDTH MEASURE DEPTH:** 14.75 Inches
Distance to measure down from top of tub to establish "Max Tub-Wall Width at Bottom"
- C** **MAX* TUB-WALL WIDTH AT BOTTOM :** 7.25 Inches
Note: Bottom tub width is typically wider than top width.

**If your tub exceeds these MAX size measurements, this product will not work with your tub.*

TEMPLATE (CUT LINES) AND PRODUCT OVERLAP MEASUREMENTS:

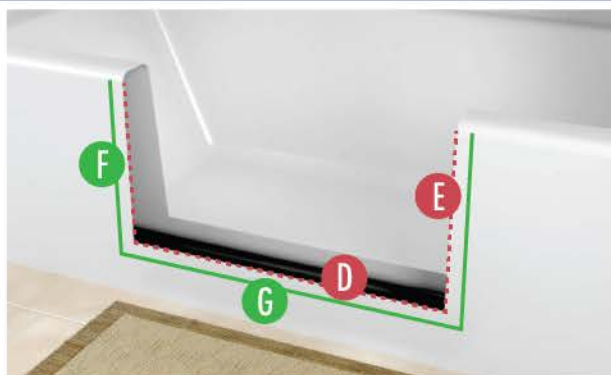


- D** **TEMPLATE LENGTH:** 25.5 Inches
- E** **TEMPLATE HEIGHT:** 14 Inches

AMOUNT PRODUCT OVERLAPS PAST CUT LINES:

- F** **LEFT AND RIGHT SIDE OVERLAP:** 1.5 Inches
- G** **BOTTOM OVERLAP:** .50 Inch

ACTUAL PRODUCT MEASUREMENTS:

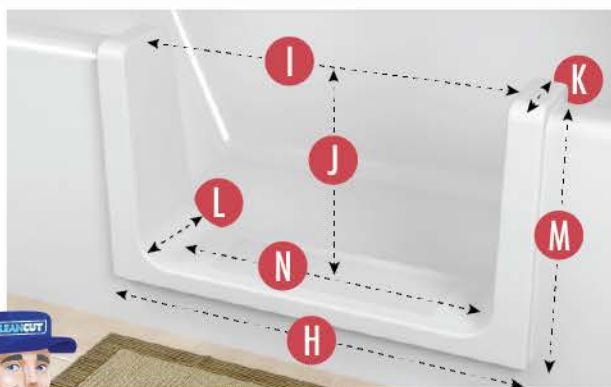

- H** **PRODUCT LENGTH:** 28 Inches
- I** **OPENING LEFT TO RIGHT AT TOP:** 24.25 Inches
- J** **PRODUCT OPENING TOP TO BOTTOM:** 13.75 Inches
- K** **PRODUCT WIDTH AT TOP:** 7 Inches
- L** **PRODUCT WIDTH AT BOTTOM:** 8.25 Inches
- M** **PRODUCT HEIGHT:** 15.75 Inches
- N** **OPENING LEFT TO RIGHT AT BOTTOM:** 20.75 Inches



Watch a video of how to install all the CleanCut products by visiting www.CleanCutBath.com/install