

Thank you for your purchase of The CleanCut Step™/Ultra-Low™. CleanCut has prepared these standard care and use instructions for the The CleanCut Step™ and Ultra-Low™. Please feel free to contact the installer of this product or CleanCut with any questions you may have. Again, thank you for your purchase!

- Do not use the tub or apply weight to The CleanCut Step™/Ultra-Low™ for two (2) days after installation to allow for the adhesive to completely dry.
- Prior to first use, please inspect the bead of caulk around the perimeter of The CleanCut Step™/Ultra-Low™ to ensure that a continuous caulk bead has been applied and that there are no cracks or areas where caulk is absent. Conduct a similar inspection of the perimeter adhesive every six (6) months thereafter.
- Prior to first use, please clean and rinse the bathtub to rid the area of any dust and/or particulates that may remain after installation.
- Do not fill the tub with water over the opening of The CleanCut Step™/Ultra-Low™.
- It is recommended that the unit be cleaned regularly with a gentle household cleaning agent. Do not use harsh cleansers or abrasive tools to clean CleanCut Step™/Ultra-Low™.
- It is recommended to completely close the shower curtain prior to turning on water to prevent water from splashing outside the bathtub.
- As with any bathtub, children should not be left unattended in the bathtub.

MSDS Sheet for Enclosed Adhesive available upon request