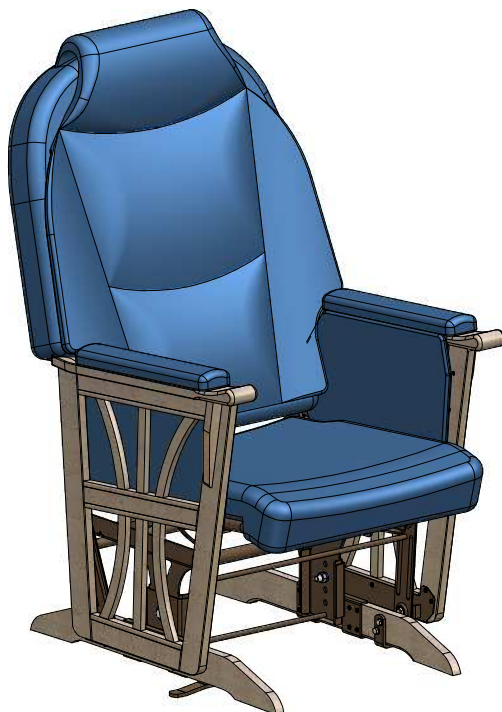
Thera-Glide™

Safety Glider

Wood Model
*U.S. Patent # 6 406 095
**Can. Patent # 2 323 398

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Modèle en bois
*Brevet U.S. # 6 406 095
**Brevet Can. # 2 323 398

*DECO style illustrated

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User's Guide

Your Thera-Glide™ Safety Glider is built for years of comfort and trouble free service. Thoroughly read and follow the instructions step by step to find out how to adjust, operate and care for your Thera-Glide™ Safety Glider.

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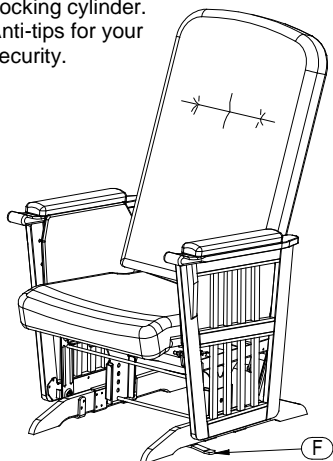
Using the Thera-Glide™ Chair

For safest operation, please familiarize yourself with the following components:


- A. Back cushion.
- B. Backrest mechanism.
- C. On/Off activator of locking system.
- D. Locking system cable.
- E. Locking cylinder.
- F. Anti-tips for your security.

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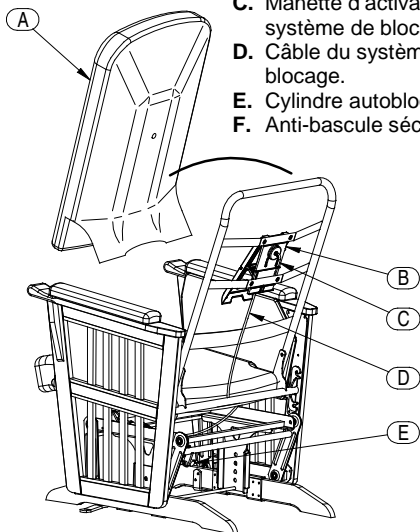
Available accessories

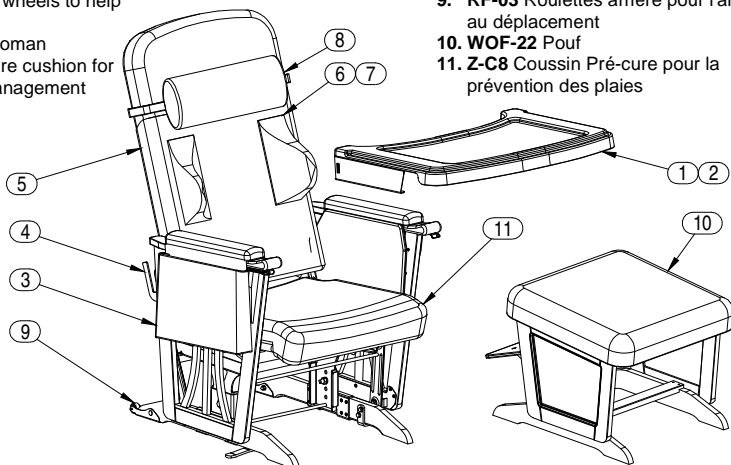- 1. RA-01 Opaque activity tray
- 2. RA-02 Clear activity tray
- 3. RA-09 Side storage bag (not compatible with RA-01/02)
- 4. RA-15 Tray storage support
- 5. RA-20 Backrest canvas
- 6. RA-23 4" Lateral trunk support (includes RA-20)
- 7. RA-24 6" Lateral trunk support (includes RA-20)
- 8. RB-18 Cervical support
- 9. RF-03 Rear wheels to help moving
- 10. WOF-22 Ottoman
- 11. Z-C8 Pre-cure cushion for pressure management

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Consult our website for a complete and up-to-date list of available options and accessories.

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Models and options

The Thera-Glide™ chair is offered in 3 models with a variety of options for a better comfort.

W-20 11-20-02

STYLE

W-20 = MISSION

W-21 = DECO

W-22 = UPHOLSTERED /
CAISSON

STAIN | FINI

11 = Natural / Naturel

12 = Mahogany / Acajou

13 = Walnut / Noyer

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17 = Small / Petit

20 = Standard / Régulier

24 = Large

BACKREST STYLE | DOSSIER

01 = Simplicity / Simplicité

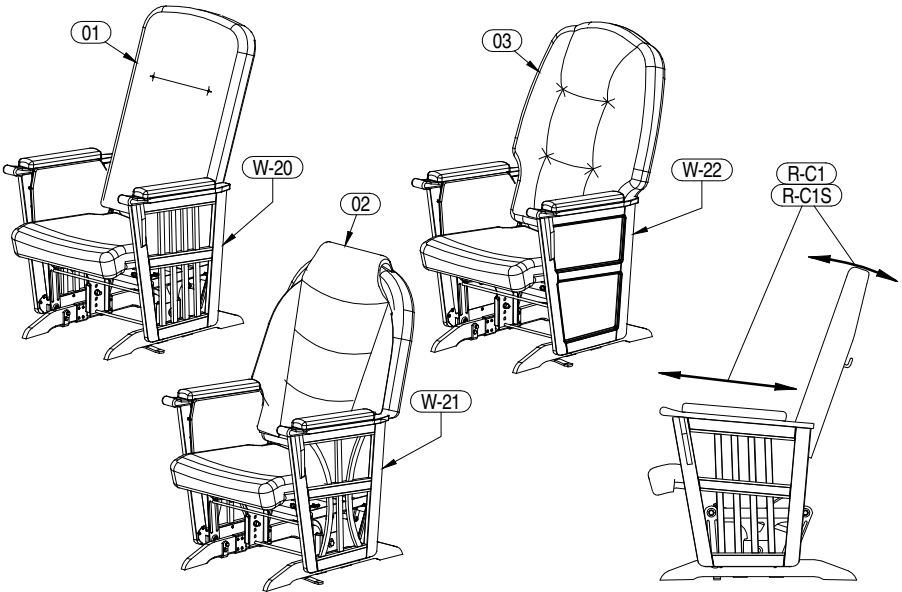
02 = Classic / Classique

03 = Comfort+ / Confort+

OPTIONS

R-C1: Seat depth adjustment 15" to 18" (38cm to 46cm) and backrest angle adjustment 95° to 115°.

R-C1S: Seat depth adjustment 13½" to 16½" (34cm to 42cm) and backrest angle adjustment 95° to 115°.

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WARNING

Be sure all adjustments are correct and bolts and nuts are tightened before sitting patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment under supervision.

Wood Thera-Glide™'s maintenance

For best performance, it is necessary that the chair is kept clean and all parts adjusted properly.

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When needed, the upholstered surfaces must be washed with a mild detergent diluted or dissolved in water. If necessary, a mild clear non-chlorinated disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as above.

The wood Thera-Glide™ is shipped in 2 separate boxes of different sizes. Check that you have both boxes before starting assembly.

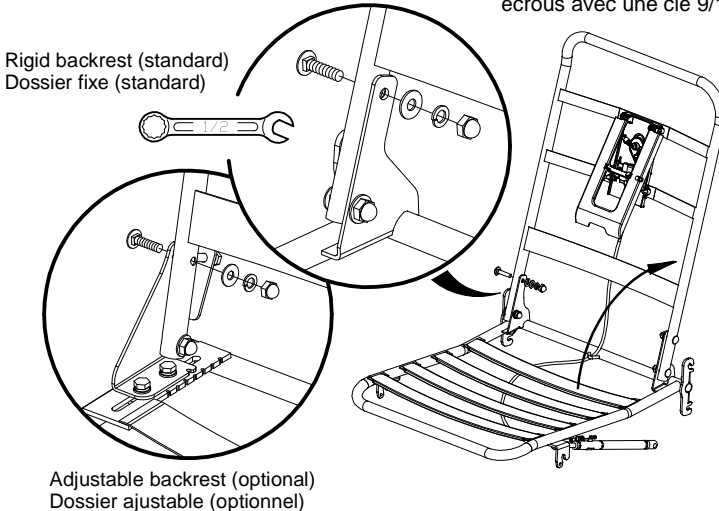
Carefully remove the wood base with the metal frame from the large box. Unscrew the two cross pieces that secure the wood side frames together.

Remove the folded metal seat/backrest frame from the small box. Be careful with the cylinder and the cable mechanism. We recommend keeping the cylinder tied to the frame until the seat frame is attached to the wood frame.

A wood base, a metal seat/backrest frame and upholstered cushions are required for each chair.

1. Back metal structure assembly

- 1.1. Raise the backrest frame upward.
- 1.2. Use the 2 carriage bolts 5/16-18 x 1 1/4", 4 washers, 2 lock washers and 2 cap nuts to attach the metal backrest frame to the seat metal frame. FOLLOW THE ORDER AS SHOWN. Tighten all bolts using 9/16 wrench (not included).

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2. Chair assembly

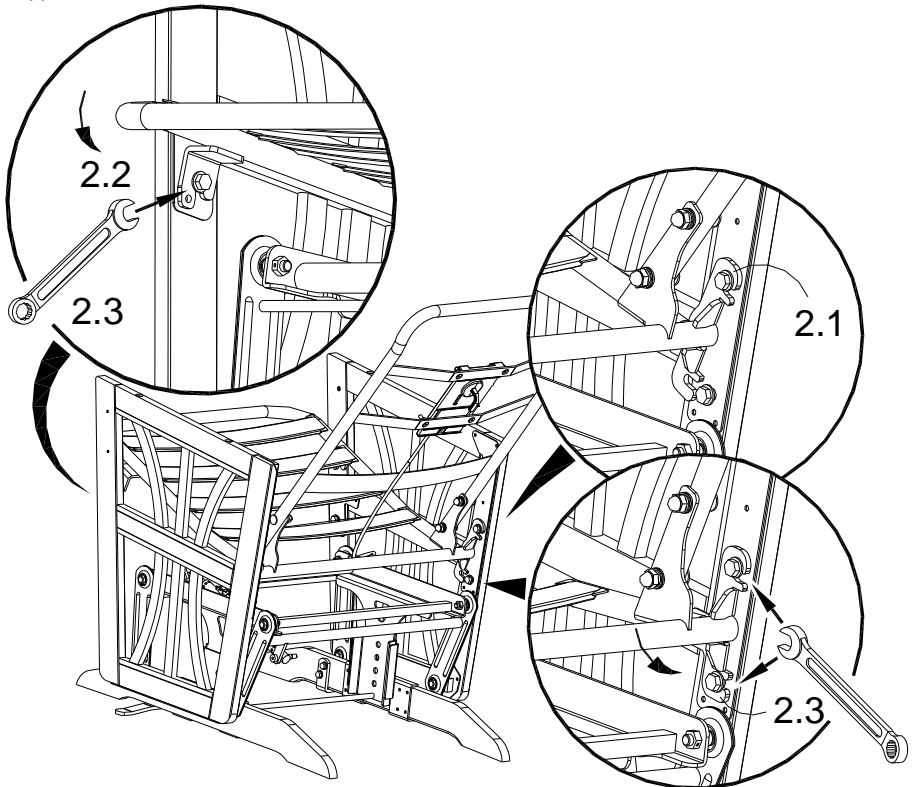
- 2.1. Insert the metal seat frame within the wooden side panels. Insert the slots at the rear of the seat frame onto the rear flange head bolts attached to the wood side panels.
- 2.2. When the rear of the metal frame is correctly positioned, carefully lower the front of the metal frame and align the front slots onto the front flange head bolts attached to the wood side panels.
- 2.3. Be sure all 6 slots of the metal frame are well engaged on their respective flange head bolts. Tighten the 6 flange head bolts with a ½ wrench (not included).

Note: the flange-head bolts are glued to avoid loss during transport. They will require a little more torque to tighten than an ordinary unglued bolt. Do not over-tighten the six flange-head bolts or the metal insert imbedded in the wood frame may be stripped.

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
3. Cable installation / adjustment

Note: for Thera-Glide™ chairs delivered fully assembled, the cable tension is factory adjusted with the correct amount of slack.

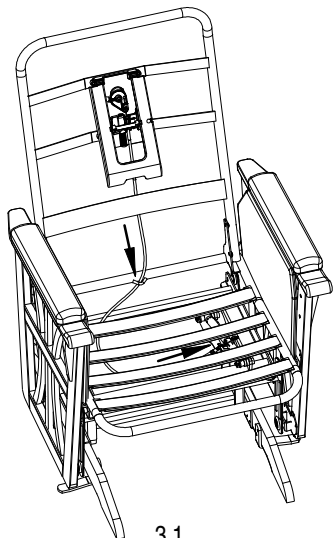
- 3.1. Attach the activator control cable to the cylinder. But first make sure the control cable is threaded through the welded eyelet located on the backrest cross tube.
- 3.2. Place the threaded end of the conduit cable in its holder on the locking cylinder and fasten the loose hexagonal nut to secure it. Initially set the hexagonal nut on the conduit side so 2 to 3 threads are showing. Adjust later as needed. Slide the steel wire with the ball end through the slot of the locking lever. While pulling on the steel wire, adjust the hexagonal nuts so that the ball end sticks out by half from the tip of the locking lever slot.
Note: make sure the backrest mechanism plate is not depressed.
- 3.3. Insert the metal cable closure on the locking lever (A-7). When the metal cable closure is inserted, push it in so that it completely covers the lever's slot.
- 3.4. Manually operate the mechanism to ensure that all is working correctly. If needed, adjust the adjustment nuts. When it's correctly adjusted, use a flat screwdriver to secure the set nut on the end side of the cable conduit, and use a 10mm wrench to tighten the adjustment nut on the opposite side. **Be sure that the nuts are tightened enough.**

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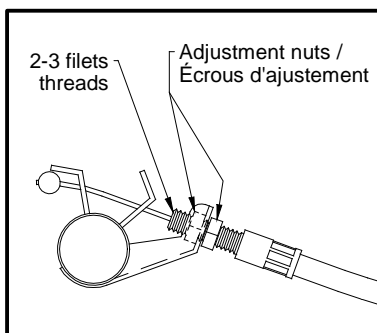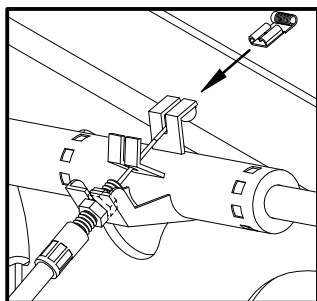
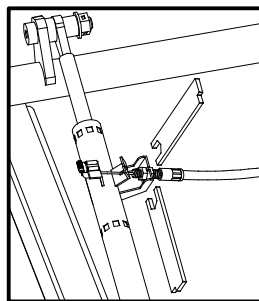
3.3

3.4

- 3.5 Put the backrest cushion back on the chair and manually operate the mechanism again to ensure it is working correctly. Readjust if needed.
- 3.6 To finish, apply a very small dab of general usage grease on the steel wire in the slot of the locking lever.

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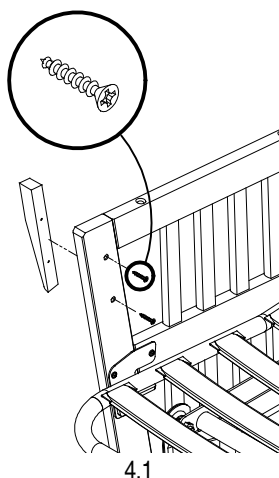
4. Wooden Armrest Assembly

- 4.1. Use #8 x 1" wood screws to attach the wooden support wedges to the wooden side frame at each side of the chair. There is a right and left wedge. The correct wedge should be flush with the top of the wooden side frame.
- 4.2. Determine the right and left Armrest Caps as shown in illustration 4.2. The Armrest Cap handgrip will curve up and the bevelled end will curve toward the seat back as shown.
- 4.3. Use the 1/4-20 x 1" Phillips/square machine screws (4 included) to attach the wooden armrest to the side frames on each side of the chair.

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
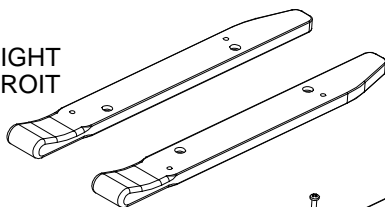
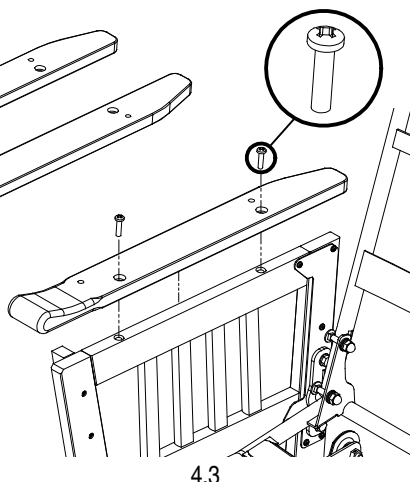
4.3

5. Armrest Pads Assembly

- 5.1. Find the left and right armrest pads as shown. LEFT is on the left side when sitting in the chair.
- 5.2. Use the 1/4-20 x 1" Phillips/square machine screws (4 included) to attach the left armrest pad on the left side of the chair. Repeat on the other side.

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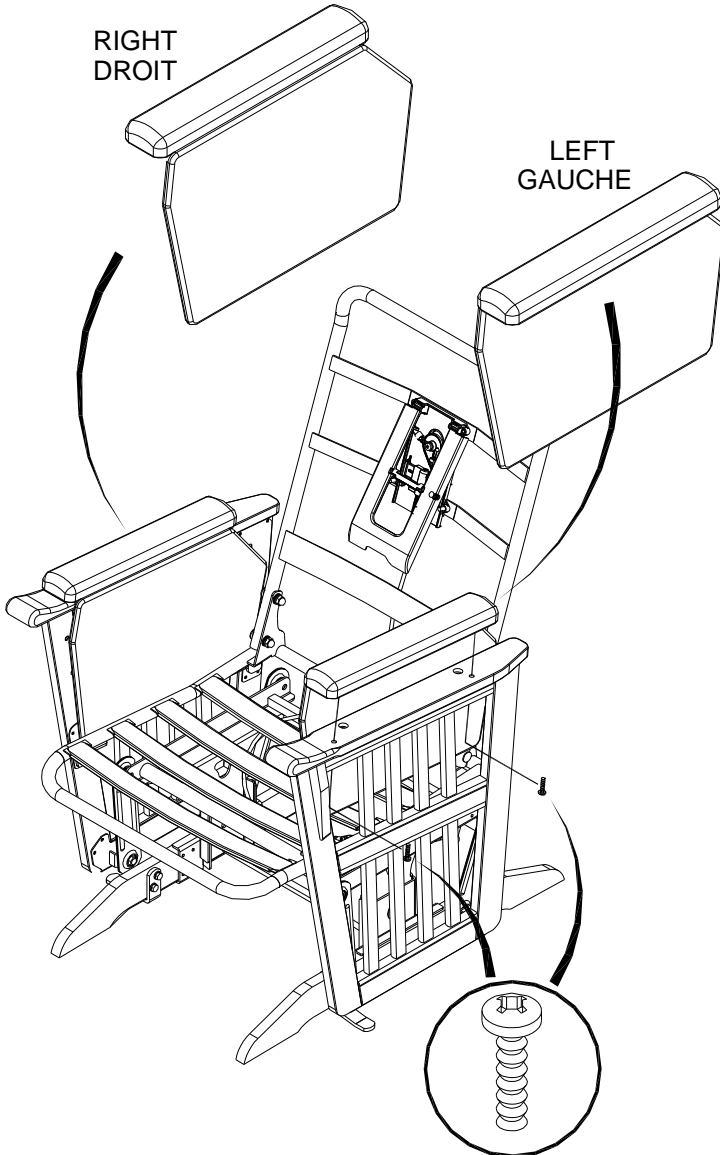
6. Seat assembly.

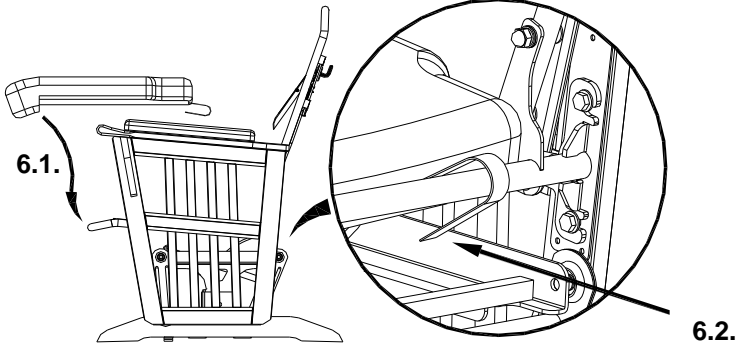- 6.1. Slide in the seat cushion so that the seat-apron's pocket located under the front of the cushion slips over the front tube of the seat deck. Push the cushion toward the back of the seat deck.
- 6.2. Secure it by wrapping the Velcro strips around the rear tube. Seal Velcro.


7. Back cushion assembly.

- 7.1. Slip the backrest cushion over the backrest.
- 7.2. Allow the Glider locking lever to pass through the hole in the back cushion, and slide the plastic washer and the rubber o-ring over the lever.
- 7.3. Secure the backrest cushion to the chair with the Velcro strips.

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
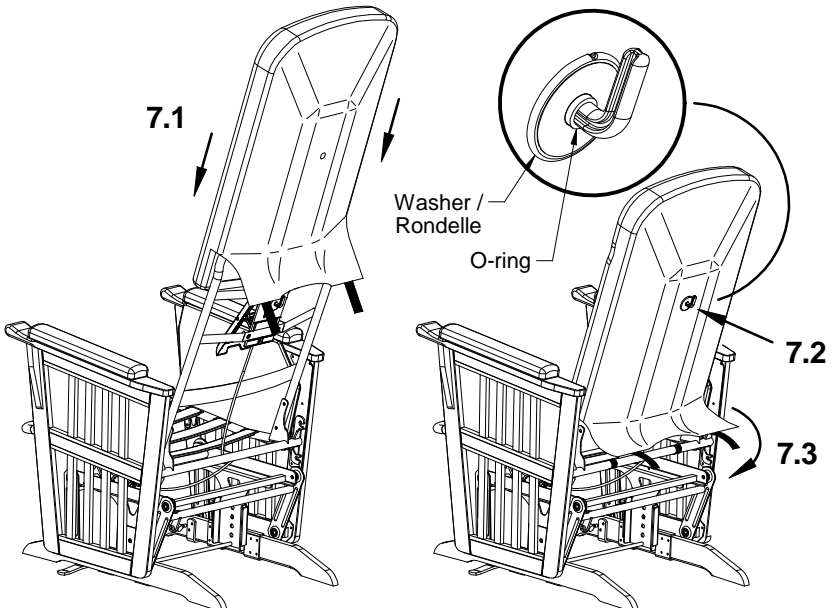
8. Lock System.

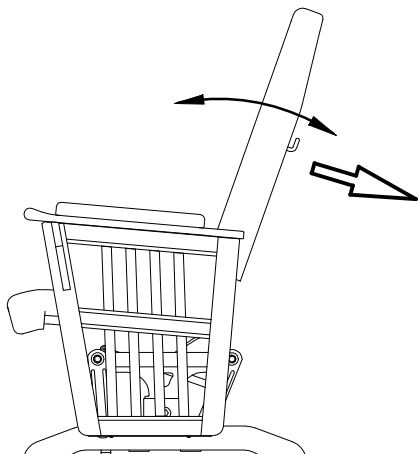- 8.1. To unlock the chair's glide motion, rotate locking lever located on the back of the chair upward and clockwise.
- 8.2. To lock the chair's glide motion, rotate the locking lever downward and counter-clockwise.

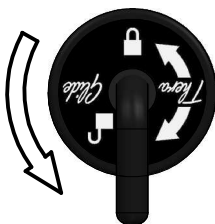
2.1
Glides / Berce

2.2
Does not glide /
Ne berce pas

9. Seat Depth Adjustment

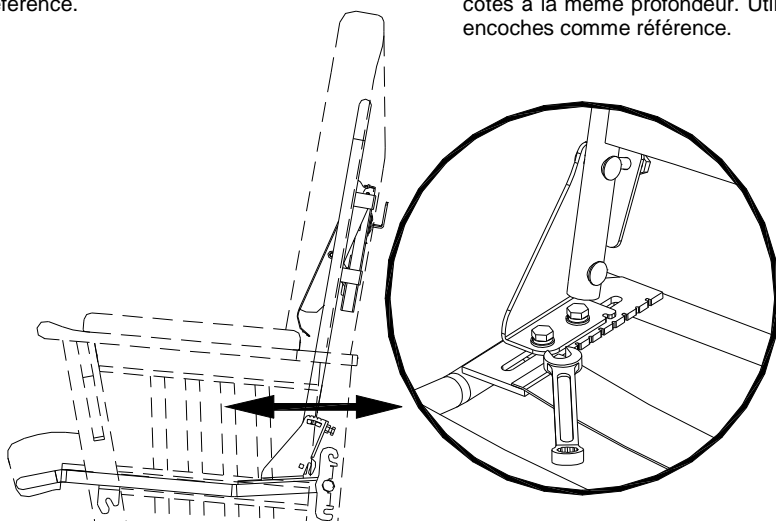- 9.1. Loosen (DO NOT REMOVE) the 2 sliding bolts on the bracket on each side at rear of the seat, as shown.
- 9.2. Adjust the seat depth as desired and re-tighten the bolts. Adjust each side to the same depth. Use the notches as reference.

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
10. Backrest Angle Adjustment

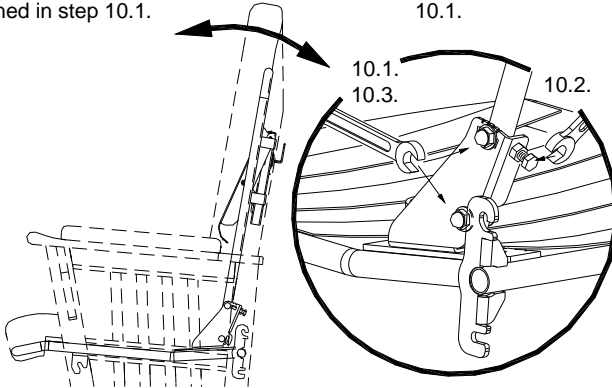- 10.1. Loosen (DO NOT REMOVE) the 2 bolts on the bracket at each rear side of the seat, as shown.
- 10.2. To adjust the backrest angle, screw or unscrew the pressure bolt on the rear of the bracket. Adjust the backrest angle and tighten the bolts.
- 10.3. Re-tighten at each side the bolts loosened in step 10.1.

11. Height adjustment

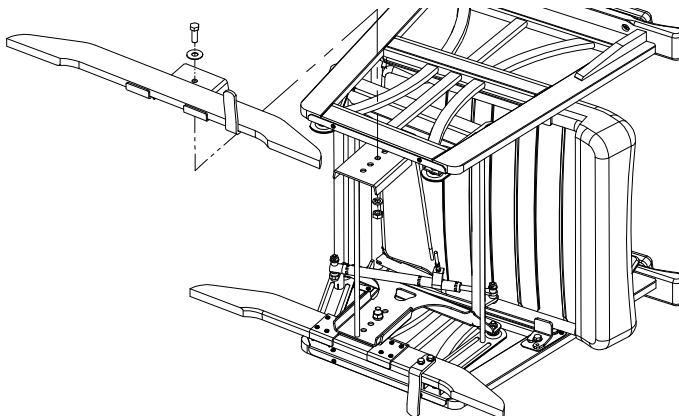
- 11.1. Tip the chair on its side. Using one 5/8" wrench and one 11/16" wrench, unfasten the hexagonal nut on the leg stand inner side, and remove the washers and the bolt. If the leg stand seems stuck, carefully twist it to set it free. Move it to the desired height, then replace the bolt, washers and hex nut and tighten firmly.
- 11.2. Repeat on the chair's other side.

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Available only if you have the R-C1 or R-C1S backrest adjustment option.

Important: Note "C"

Always specify fabric color and pattern, wood stain and the Thera-Glide™'s serial number for all replacement parts orders. Replacement parts may exhibit minor color/stain variances from the original part. LPA Medical will not be responsible for any possible variances.

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TERMS & CONDITIONS

GENERAL:

- All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

- LPA MEDICAL can not accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsided damage.

RETURN GOODS

- Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request most include the following information:
Reason for requested return

LPA MEDICAL invoice or acknowledgement number

LPA MEDICAL model or part number and number of cartons and items being returned.

Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY

- LPA MEDICAL warranty covers all defects in workmanship, except the exclusions listed in the not covered section below, for the following periods of time from date of purchase:

Lifetime for metal parts (frame): R-505, R-605, R-705, T-40, U-110, U-120, W-2, Z-15, Z-25 chairs

5 years for metal parts (frame): C-207, Optimum, O-07, R-505/605/705-SHD, S-150, S-250, Z-45 chairs

3 years only for auto lock glider mechanism

1 year for all other mechanism, gas spring cylinder or mechanical lock, control cable, wheels & casters, tray, upholstery.

If service is required under this warranty, contact our Customer Service Department in writing, giving the following information

- 1- Model or part number involved
- 2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number
- 3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

- LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric facing and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, you may also have other rights which vary from Province to Province or State to State.

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Adresse

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FOR YOUR RECORDS

**Purchaser's record for
LPA MEDICAL WARRANTY**

Date of purchase

Purchased from

Address

Model No, serial No. and quantity