HARMONIE ***HARMONY***

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Care-Recliner
Model C-207

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User's guide

Your HARMONY care-recliner is built for years of trouble-free service. Carefully read and follow the instructions step-by-step to assemble, operate and care for your recliner.

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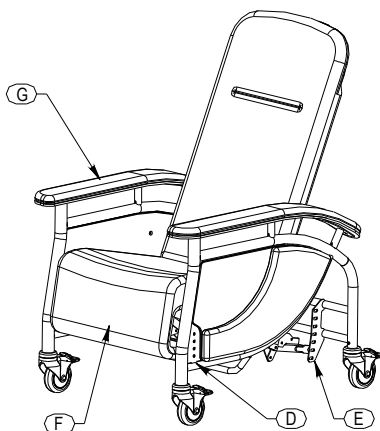
Using the HARMONY Chair


For the safest operation, please familiarize yourself with the following components:

- A**: Swivel wheels with brake.
- B**: Seat depth and backrest angle adjustment.
- C**: Push bar.
- D**: Front seat height adjustment.
- E**: Rear seat height adjustment.
- F**: One piece seat and legrest cushion.
- G**: Wide armrest.

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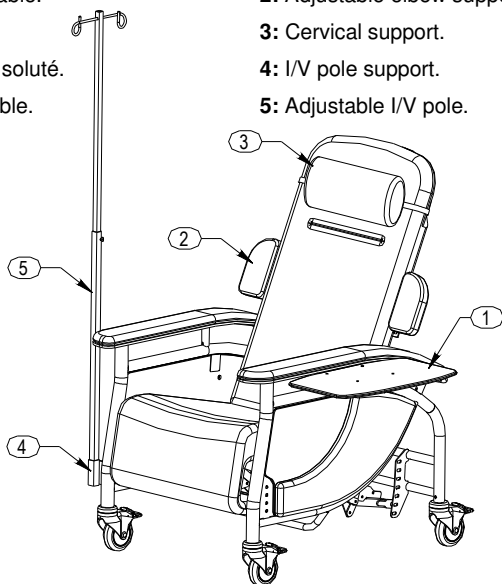Available Accessories

- 1**: Fold-down side tray (left or right).
- 2**: Adjustable elbow supports.
- 3**: Cervical support.
- 4**: I/V pole support.
- 5**: Adjustable I/V pole.

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WARNING

All brakes **MUST BE ENGAGED AT ALL TIMES** except during transportation.

This product is equipped with a mechanized legrest. **DO NOT** let children sit on this chair without supervision. **DO NOT** let children operate any mechanism. A child could be seriously injured when declining the legrest.

ALWAYS leave an empty chair in the upright position.

Keep hands and feet clear of all mechanisms.

ONLY the occupant should operate the chair.

Be sure all adjustments are correct and bolts and nuts are tightened before sitting patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment and under constant supervision.

Maintenance

For optimum performance, it is necessary that the chair be kept clean and all parts adjusted properly.

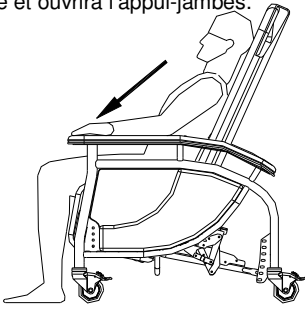
When needed, the upholstered surfaces must be washed with a mild detergent diluted or dissolved in water. If necessary, a mild, clear disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as described above.

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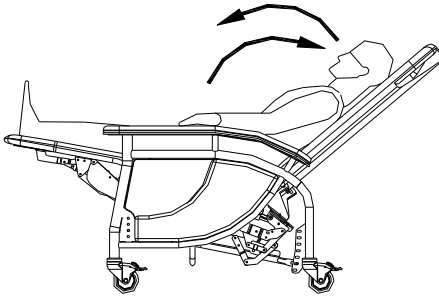
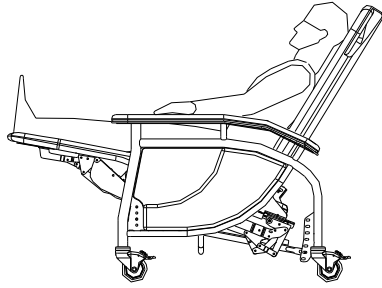
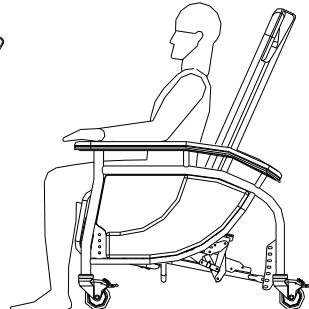
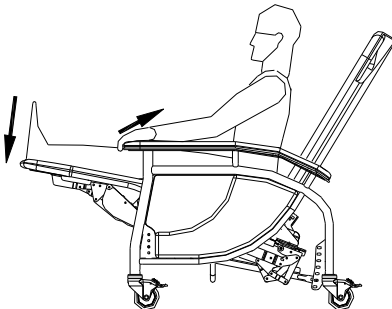
How to use of your HARMONY

All brakes **MUST BE ENGAGED AT ALL TIMES**, except during transportation.

1. To open the legrest, grip the end of the armrests and push your body back. This will activate the mechanism and open the legrest.


2. To incline the backrest, lie back into the chair.
3. To recline the backrest, move back into the upright position.

4. To decline the leg rest, push it down with your feet. At the same time, grip the armrests and pull forward.

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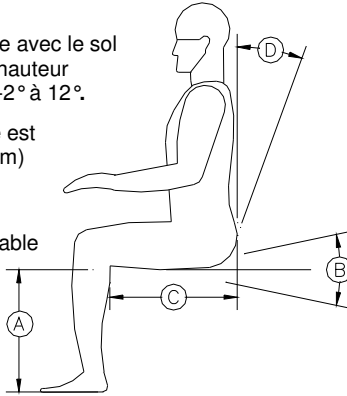Chair Adjustments

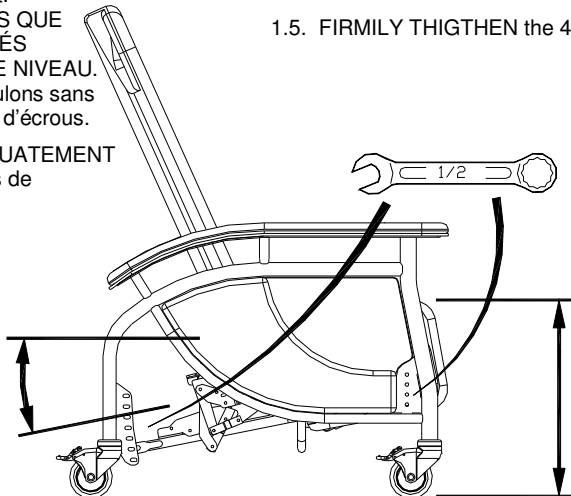
- A.** The seat height from floor is adjustable from 17 ½" to 20 ½", by 1" increments.
- B.** The seat angle with the floor is adjustable from -2° to 12°.
- C.** The seat depth is adjustable from 17" to 18 ½".
- D.** The seat and backrest angle of the starting position between is adjustable from 5° to 18°.

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

1. Seat height (A) and Seat Angle Adjustments (B).

- 1.1. Determine if the seat height should be adjusted up or down.
- 1.2. Loosen the 4 seat structure screws (2 at front and 2 rear). **DO NOT REMOVE.**
- 1.3. Remove the 2 front screws and move the seat structure to the desired position. **BE SURE EACH SIDE IS AT THE SAME LEVEL.** Replace the bolts with the lock washers.
- 1.4. Remove the 2 rear screws and move the seat structure to the desired position to obtain the correct seat angle. **BE SURE EACH SIDE IS AT THE SAME LEVEL.** Replace the bolts with the lock washers.
- 1.5. **FIRMLY THIGHTEN** the 4 screws.

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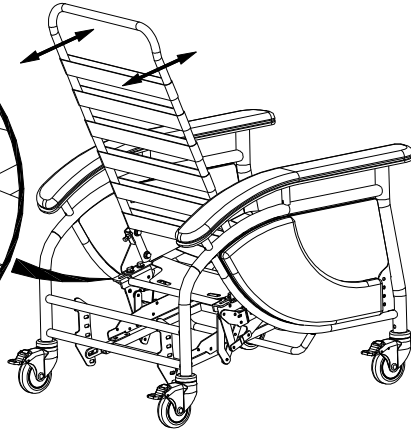
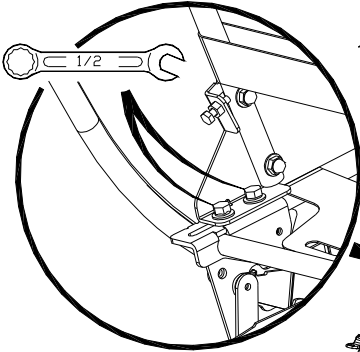
2. Seat Depth Adjustment (C)

- 2.1. On each rear side of the seat, loosen (DO NOT REMOVE) the 2 bracket (sliding bolts) as shown.
- 2.2. Adjust the seat depth correctly and tighten the bolts. Adjust each side to the same depth.
- 2.3. Tighten the bolts (loosened in 2.1).

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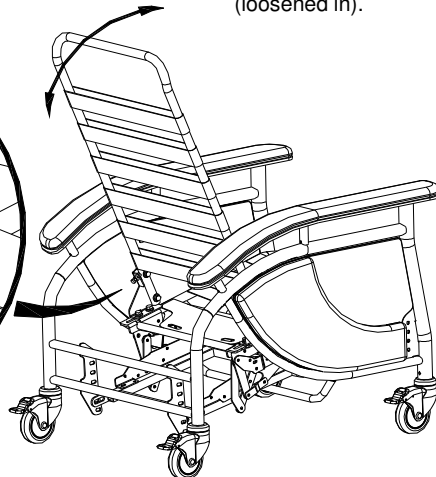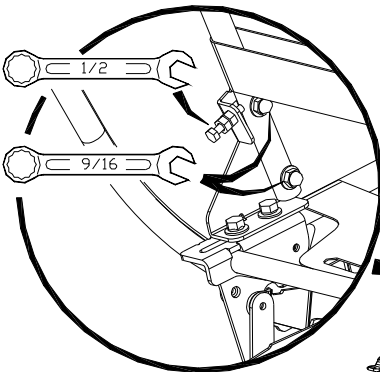
- 3.1. To adjust the backrest angle, loosen or tighten the bolts at the rear of the bracket until the desired angle is achieved.
- 3.2. Tighten the bolts (loosened in).

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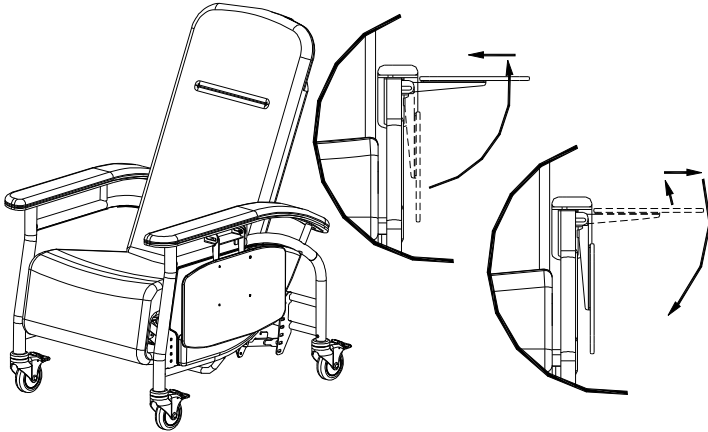Accessories

Lateral Tray (right or left)

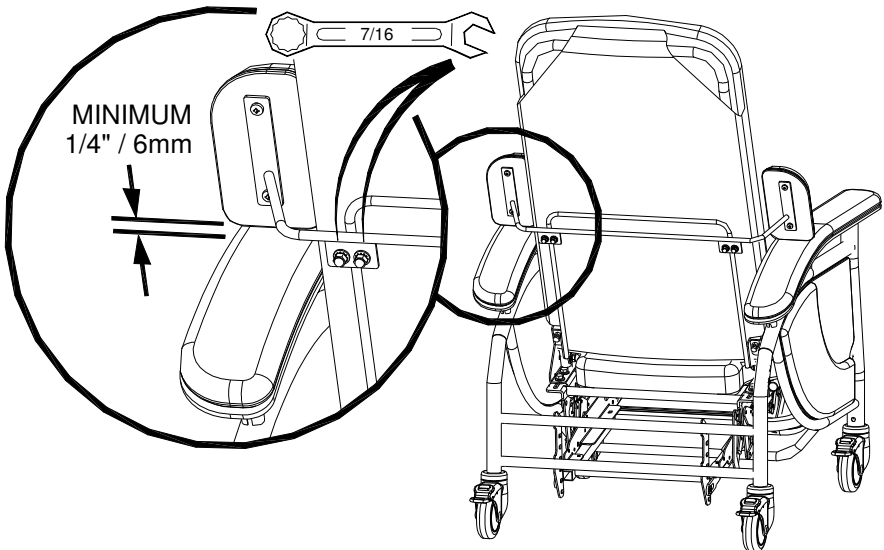
- Fold up the tray to the horizontal position and push back to secure it.
- To fold down, raise it slightly and pull out the tray.

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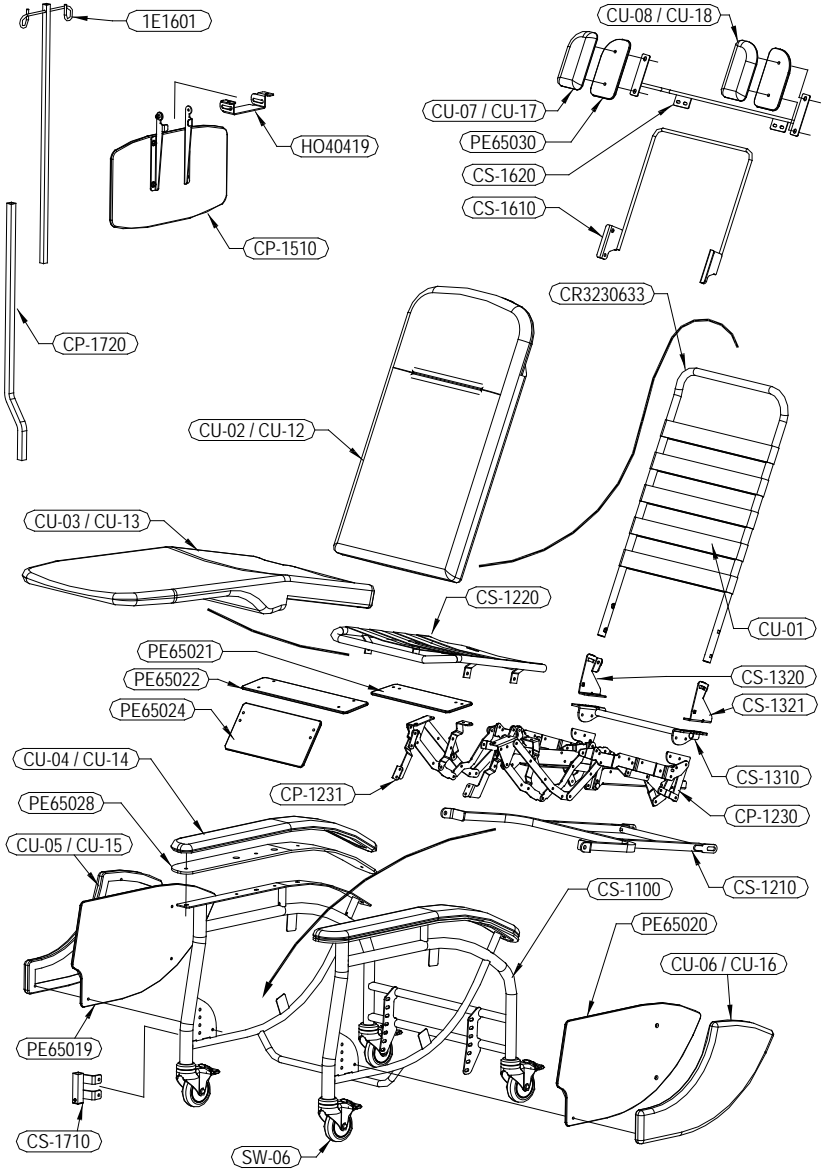
Elbows Supports Adjustment

- Loosen (NOT REMOVE) the 2 U-bolts on the elbows support.
- Adjust it to minimum 1/4" over the armrest on each side when the chair is closed.
- Tighten the U-bolts on each side.

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Always specify fabric name, pattern, color, and the HARMONY's serial number for all replacement part orders. Upholstered replacement parts may vary slightly in color from the original part due to dye lot differences. LPA Medical will not be responsible for dye lot differences.

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TERMS & CONDITIONS

GENERAL:

• All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

• LPA MEDICAL can not accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsided damage.

RETURN GOODS

• Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request most include the following information:

Reason for requested return

LPA MEDICAL invoice or acknowledgement number

LPA MEDICAL model or part number and number of cartons and items being returned.

Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY

• LPA MEDICAL warranty covers all defects in workmanship, except the exclusions listed in the not covered section below, for the following periods of time from date of purchase:

Lifetime for metal parts (frame): R-505, R-605, R-705, T-40, U-110, U-120, W-2, Z-15, Z-25 chairs

5 years for metal parts (frame): C-207, Optimum, O-07, R-505/605/705-SHD, S-150, S-250, Z-45 chairs

3 years only for auto lock glider mechanism

1 year for all other mechanism, gas spring cylinder or mechanical lock, control cable, wheels & casters, tray, upholstery.

If service is required under this warranty, contact our Customer Service Department in writing, giving the following information

- 1- Model or part number involved
- 2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number
- 3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

• LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric fading and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, you may also have other rights which vary from Province to Province or State to State.

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SPECIAL NOTES ABOUT WHEELCHAIR TIE-DOWN WHILE TRANSPORTED

LPA Medical recommends that wheelchair users NOT be transported in vehicles of any kind while seated in wheelchairs.

As of this date, the Department of Transportation has not approved any Tie down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type.

It is **LPA Medical's** position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the automobile industry.

LPA Medical cannot and does not recommend any wheelchair transportation systems.

REGARDING RESTRAINTS - SEAT POSITIONING STRAPS - IT IS THE OBLIGATION OF THE DEALER, THERAPIST, TRANSPORTER AND OTHER HEALTH CARE PROFESSIONAL IN CHARGE TO DETERMINE IF A SEAT POSITIONING STRAP IS REQUIRED TO ENSURE THE SAFE OPERATION OF THIS EQUIPMENT BY THE USER. SERIOUS INJURY CAN OCCUR IN THE EVENT OF A FALL FROM A WHEELCHAIR.

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Adresse

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FOR YOUR RECORDS

Purchaser's record for LPA MEDICAL WARRANTY

Date of purchase

Purchased from

Address

Model No, serial No. and quantity