

## Jazz Dual Crutch & Cane Holder - MS06154

### Intoduction

Your Jazz Dual Crutch & Cane Holder provides an easy and convenient way to carry your crutch or cane on the go with your scooter.



### Parts

1. Upper post (height adjustment)
2. Lower post (scooter connection)
3. Fixing tools



### Installation

1. Adjust the position between the upper and lower posts to a suitable height to fit the length of your crutch or cane, then assemble the upper and lower posts by using fixing tools (bolt, washer, nut, wrench). (Fig 1.)
2. Connect the Jazz Dual Crutch & Cane Holder to the seat with fixing nut. (Fig 2.)
3. Secure your crutch or cane to the holder using the Velcro strap. (Fig 3.)

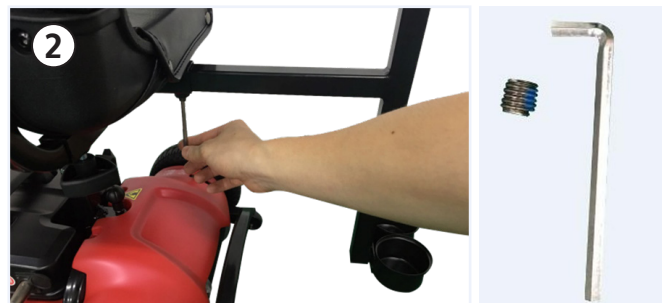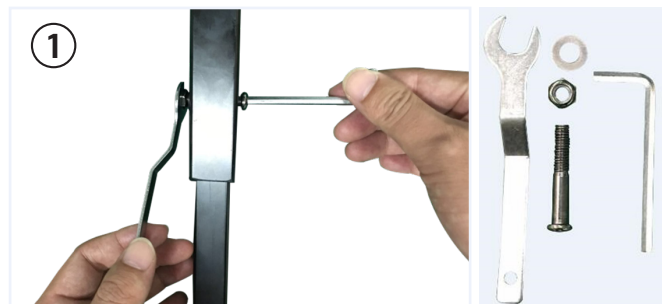

### Warnings

1. When the crutch & cane holder is attached to your scooter, please ensure there is enough space to swivel the seat.
2. When driving your scooter in reverse, or turning when the crutch & cane holder is attached, please pay careful attention to whether any people or obstacles are nearby.