




237- 357 mm (9 1/4" - 14")

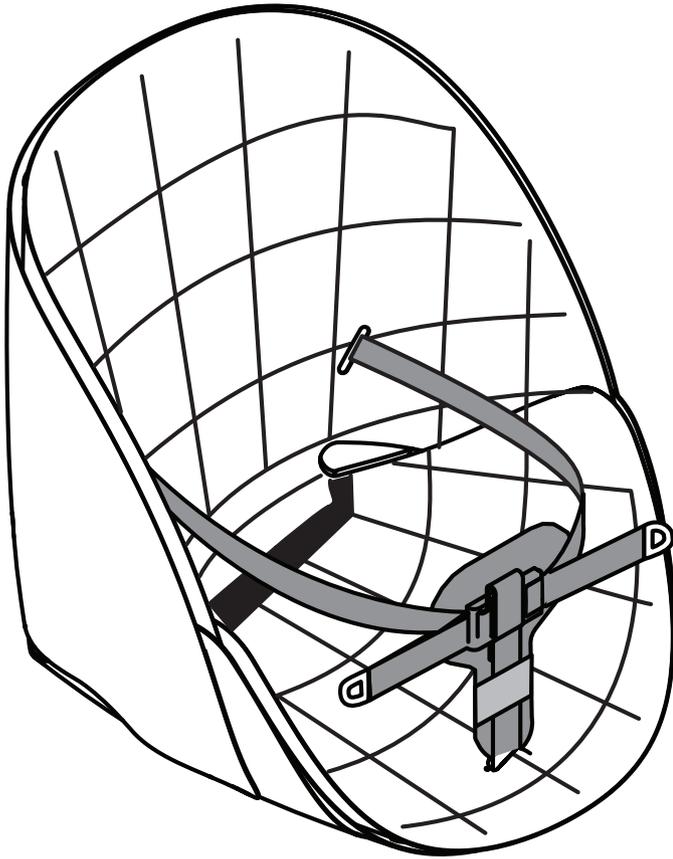

Scallop User Guide



3



4



8



8



8




8



8




8

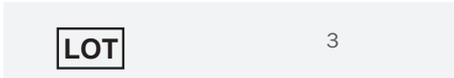

3

mm (inch)







8



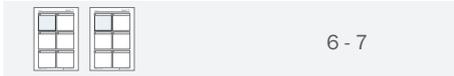


6 - 7

ENGLISH

9 - 12

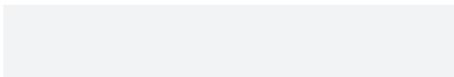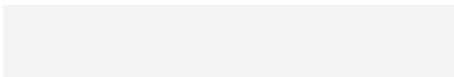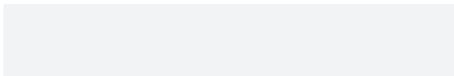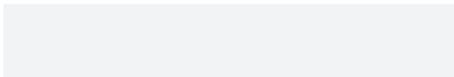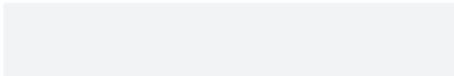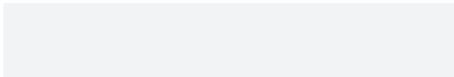


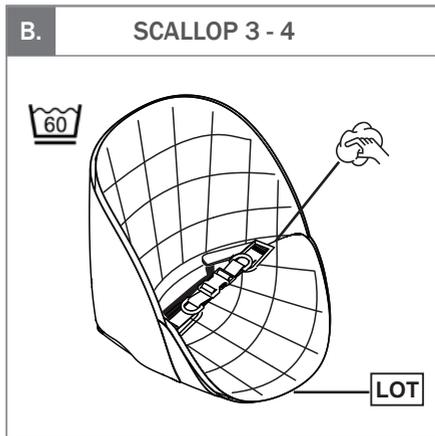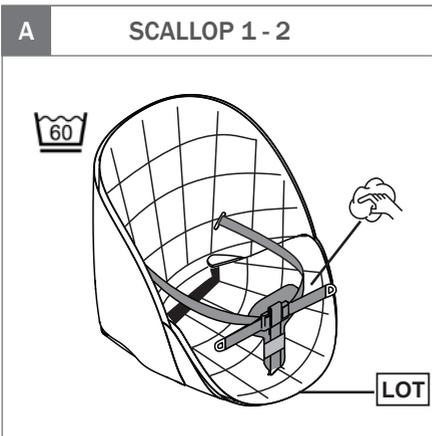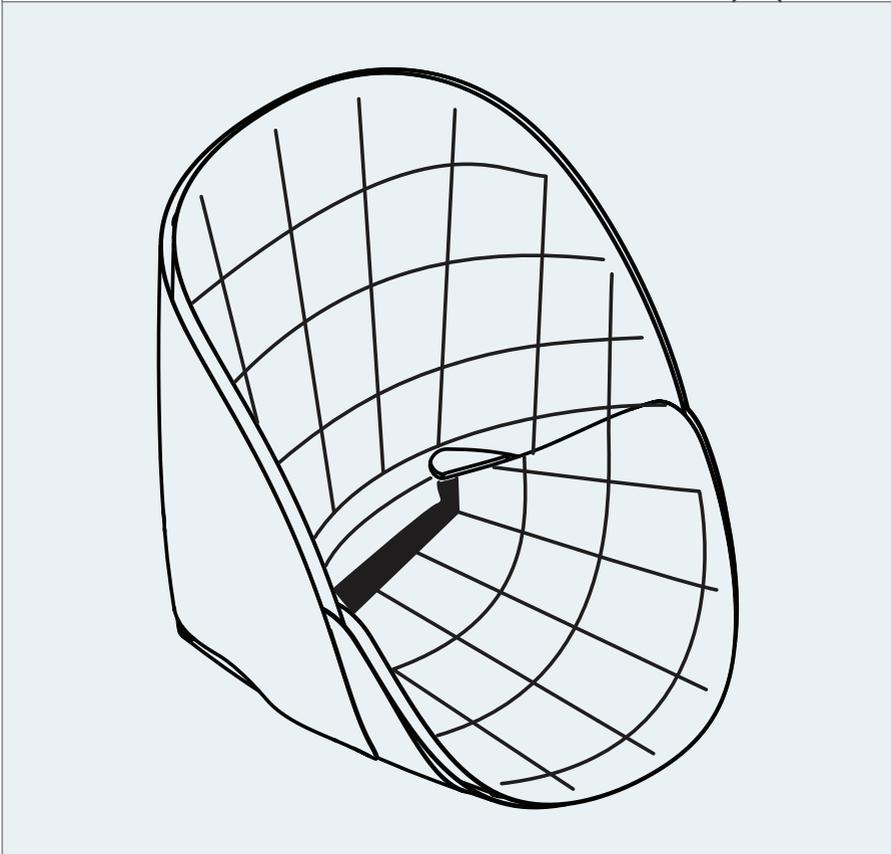




A.


SCALLOP 1 - 2

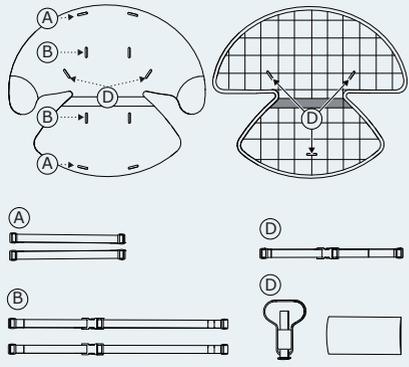


1.

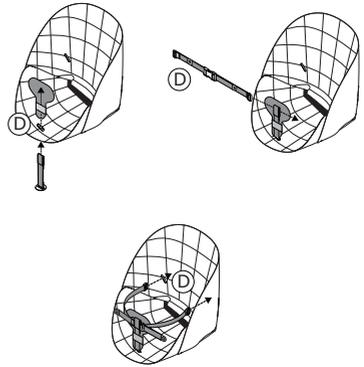


2.

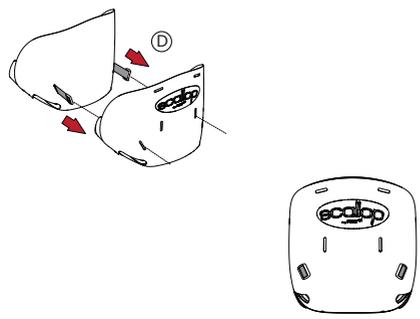


3.

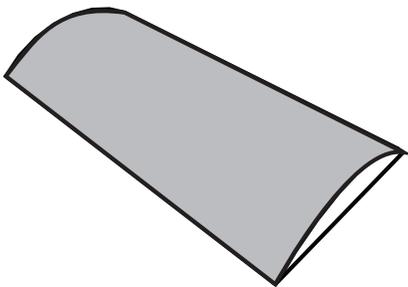

4.

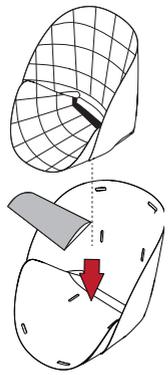

5.

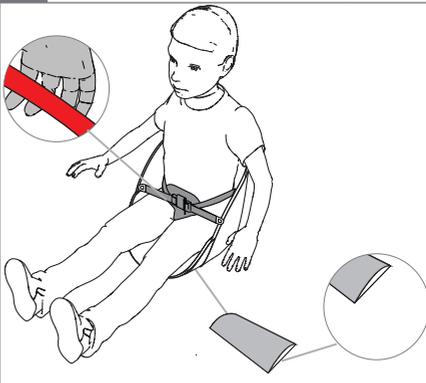



B. SCALLOP 3 - 4

A

B

C

A

B

C

1.

2.







C





3.

4.

5.

♿

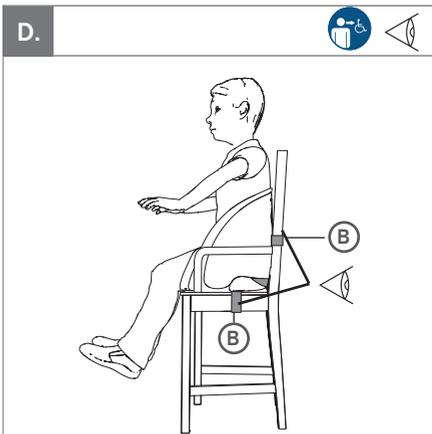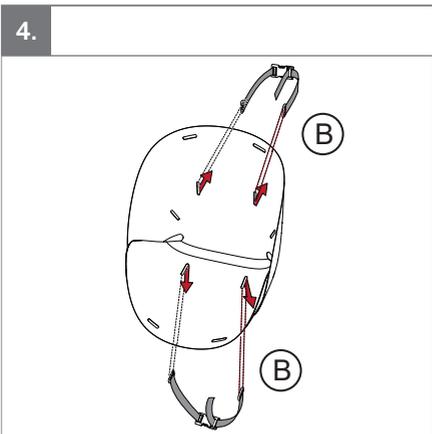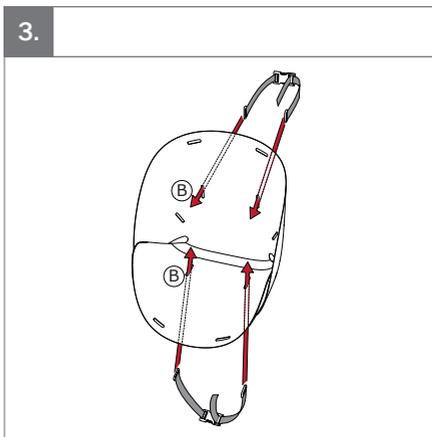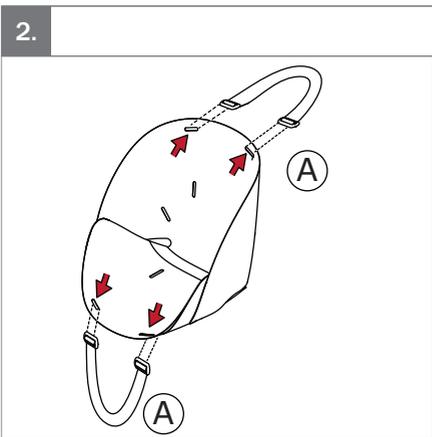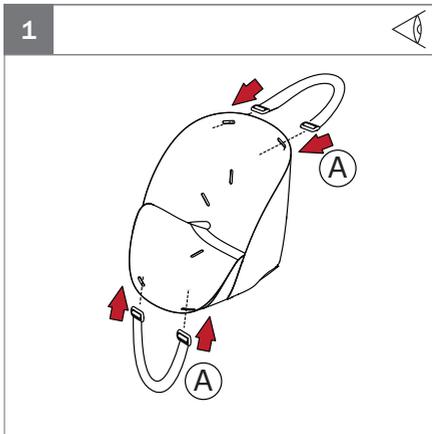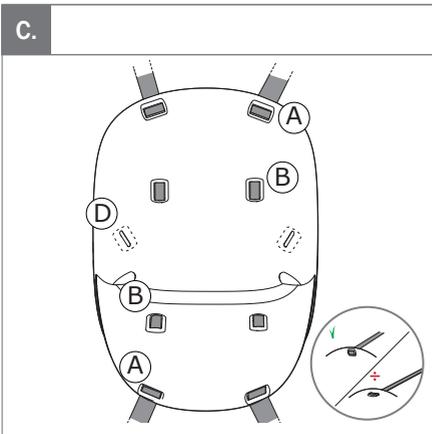

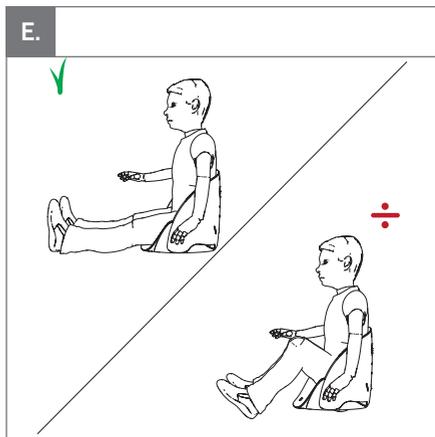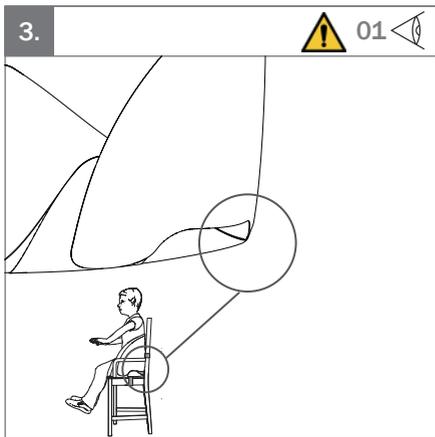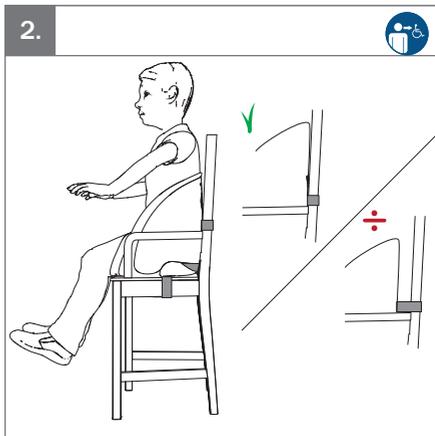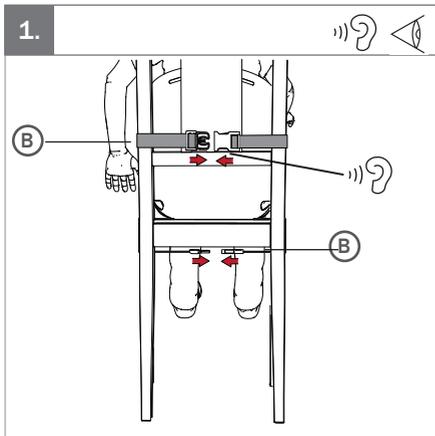



mm (inch)				
	 1	 2	 3	 4
 Mass device	0.60 kg (1.3 lb)	0.78 kg (1.7 lb)	1kg (2.2 lb)	1.44 kg (3.2 lb)
 Seat width	19.2 cm (7½")	23 cm (9")	27 cm (10½")	29.6 cm (11½")
 Seat depth	23.7 cm (9¼")	27.5 cm (10¾")	31.5 cm (12¼")	35.7 cm (14")
 Seat height	23.7 cm (9¼")	27.5 cm (10¾")	31.5 cm (12¼")	35.7 cm (14")
				
 Hip width	15 - 21 cm (5¾" - 8¾")	21 - 24 cm (8¼" - 9¼")	24 - 27 cm (9¼" - 10½")	27 - 38 cm (10½" - 14¾")
 Back height	33 - 38 cm (12¾" - 14¾")	38 - 42 cm (14¾" - 16½")	42 - 47 cm (16½" - 18½")	47 cm (18½")
 Max.user mass	15 kg (33 lb)	22 kg (48.5 lb)	29 kg (64 lb)	70 kg (154.3 lb)

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended use

The Scallop is a simple, portable and versatile seating solution for children who require extra support whilst floor sitting or sitting on a chair. The Scallop is extremely lightweight and easy to transport using 2 carry handles. It can be used for indoor and outdoor purposes. The Scallop comes in 4 sizes, 2 styles and 2 reversible cushions. This product is suitable for children with GMFCS level 1-3, children who require a little extra help while sitting. This product is also great for children with behavioral disorders eg. autism as it provides tactile stimulation which helps to reassure the child and enable them to relax and concentrate in the product. The Scallop is made up of a flexible shell which provides stability without limiting movement, a stylish cushion, 2 carry handles, a belt and 2 attachment straps for a chair.

Declaration of Conformity -



This product conforms to the requirements of the Medical Devices Regulation (2017/745). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings.

R82 Warranty

R82 offers a 2-year lifetime warranty against defects in workmanship and materials. The warranty would be adversely affected if the customer's responsibility of servicing and/or daily maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the r82.org page.

The warranty can only be sustained if the R82 product is in use in the same country where it was purchased and if the product can be identified by the LOT number on the cushion. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original R82 parts are used, or if the product has not been maintained according to the Maintenance responsibilities table. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for.

Disposal -



When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

ENGLISH

Symbols



For indoor and outdoor use



Consult instructions

The latest version of the instructions is always available at the www.Also4Me.com webshop and can be printed in larger sizes



Warning

This symbol appears in the User Guide along with a number referring to the instructions below. It draws attention to situations where the product or the user or carer's safety may be in danger

01 = Entrapment Hazards



Never leave the user unattended. Ensure permanent supervision by an adult



Manual cleaning

Keep clean to maintain optimum function



Inspection



Listen to click sound



Washing, mild process, maximum 60 Celsius



Do not tumble dry



Do not iron



Do not bleach



Correct use



Incorrect use

ENGLISH

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

Read all of the instructions carefully before use and keep them for future reference. Incorrect use of this product may cause serious injury to the user

This product contains small parts which could present a choking hazard if removed from their intended placing

Always use the correct lifting and transfer techniques for this purpose

Never leave the user unattended in this product. Ensure permanent supervision by an adult

Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier

If there is any doubt as to the continued safe use of your R82 parts or if any parts should fail, stop using the product immediately

Environment

For the purpose of safety and comfort, always move the product away from the sun and allow to cool before use

Use this product on a level surface without any obstacles during floor sitting or when attached to a chair

User

If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

Carry out all positional adjustments on the product and parts and ensure that all buckles are securely fastened before use.

Inspect the product and all of its parts before use

Be careful not to get your fingers trapped/squeezed.

Never use the product without the cushion

This product is tested for resistance to ignition at level non-ignition in accordance ISO 7176-16, EN1021-1 and EN1021-2

The product might lose its flame resistant characteristics if using other than the original cushion

Always remove "A" and "B" straps before using for floor sitting

"A" straps are only to be mounted on the product for carrying. Never place a child in the seat while the "A" straps are mounted

Adjust/change the seat as the child grows

ENGLISH

Maintenance responsibilities table

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers.

Frequency	Description
1st Day	Read the instructions for use (IFU) carefully before use and store them in a safe place for future reference
Daily	Use a damp cloth to clean the product
	Ensure that all buckles are in place and fitted correctly
	Check all buckles for signs of worn out parts
Weekly	Wipe off any residue and dirt from the product using a cloth with warm water and a mild detergent and dry before use
Monthly	To avoid failures, conduct a monthly general check of the shell and velcro

Washing the cushion in a machine

This cushion can be washed at 60°C using a mild detergent in a washing machine. Remove all parts before washing the cushion. Clean the parts separately. Please allow the product to dry before reuse.