GUARDIAN[®]

Family-owned. Trusted by
hospitals for over 50 years.

FOOTPIECE ACCESSORIES FOR WALKERS

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The following photos are an aid in the identification of product types.
Specific accessories may vary in size and appearance.

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WARNING:

For the user's safety, use only those accessories as instructed by a healthcare provider. Prior to changing an accessory on a Guardian walker, read all instructions and cautions. Wheels should face the outside of the Walker. Only use wheels on the front legs. Only wheels that contain a brake can be used on the rear legs.

BEFORE EACH USE

- ✓ The walker should be adjusted so that when the user's arms are extended with a natural bend, the handle is at wrist height.
- ✓ Ensure the walker sits level and all legs are adjusted to an equal height.
- ✓ Check the locking mechanisms and spring buttons to ensure they are fully engaged and in good working order.
- ✓ Check the walker's tips are not ripped, worn or missing. If the tips appear to be damaged in any way, replace them before use.

FOOTPIECE – Wheels and Brakes

1. Depress the spring button on the walker and slide the specific footpiece up or down.
2. Engage the spring button into hole that provides correct height.
3. Repeat steps 1-2 for each leg, ensuring the walker is level.

WHEEL ACCESSORIES

For Front leg placement ONLY

- Insert the 3" (7.6cm) wheeled footpieces into the lower leg extensions.
- Position the 5" (12.7cm) wheeled footpieces on the outside on the front legs.
- Installing swivel wheels (caster wheels) - To assist in proper placement of the wheel, the letter "L" or "R" is stamped on either wheeled footpiece. The left or right position is determined by using the product in its operating position. The left and right wheels need to be placed on the front legs with the wheels angled toward the rear of the walker.
- Mount with wheel on outside of leg.

GLIDE BRAKE & WHEELED GLIDE BRAKE ACCESSORIES

For rear leg placement ONLY

- Glide brakes and Glide wheel brakes should be placed on the rear legs of the walker **ONLY**. Rear wheel brakes should be positioned in the back so that wheels are on the outside of walker.

GLIDE CAP INSTALLATION

For rear leg placement ONLY

- ✓ Place one glide cap over each of the walker's two (2) rear footpiece non-skid tips.
- ✓ To ensure caps are in position, set the walker on the floor and place weight firmly down on the walker's handle.

Note: Glide caps should fit snugly on tips.

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CAUTION

- Glide caps do not grip the floor; extra caution must be taken when using the walker. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in the user's path.
- Glide caps are designed for use on indoor floor surfaces and on some outdoor surfaces.
- The glide caps will wear rapidly on rough concrete and asphalt surfaces.
- Glide caps may interact with some floor surfaces and sealers. Be sure to check and clean glide caps as needed to prevent scratches or marks on the floor's surface. Guardian accessories are for use on Guardian products only. Use on other walkers may result in an improper fit causing potential injury to the user.

FAILURE TO FOLLOW ALL OF THE ABOVE INSTRUCTIONS MAY RESULT IN SERIOUS INJURY.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Warning: This product contains a chemical known to the State of California to cause cancer.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2017 Medline Industries, Inc. Guardian and Medline are registered trademarks of Medline Industries, Inc.

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