

Assemble the mattress

1. Remove any existing mattress from the bed frame.
2. Check that there are no protruding sharp objects on the frame surface.

3.




WARNING

To avoid death or serious injury by entrapment, always select the correct mattress size for the bed.

Select the correct mattress size for the bed frame. Make sure that there are no gaps to trap the patient's head or body.

4. Position the mattress on the bed frame. Make sure that the mattress is orientated correctly.

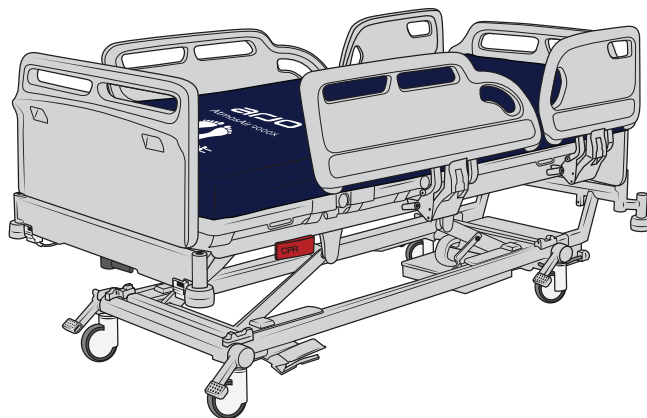

Reactive Therapy

The AtmosAir 9000X reactive therapy is controlled by the Air Redistribution Module (Velaris ARM™) that allows pressure to be distributed between the air cells across the mattress surface. It adjusts itself by slowly, letting air in and out of the mattress, to maintain an optimum pressure level inside the air cells.

Patient position

Place the patient on the mattress. Make sure that the patient's head is placed on the head end of the mattress.

