GUARDIAN[®]

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General Warnings

To ensure the safe assembly and operation of the commode, these instructions must be followed. **FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.** If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- **DO NOT** assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- **DO NOT** use if components are missing or are damaged. Contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.
- **DO NOT** exceed weight capacity of 1,000 lbs (453 kg).
- **DO NOT** make any adjustments while using the product.
- **DO NOT** attempt to use the arms to support full body weight.

Cautions

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product.

Prior to use, ensure the following;

1. Toilet seat must be in the down position before use. Failure to do so may results in possible injury or damage.
2. The height adjustment push buttons fully protrude through the same respective hole of each leg extension.
3. There are no loose, broken or missing components and ensure all parts are secure.
4. When using the backrest, ensure the commode remains balanced. Excessive twisting or turning while seated on the commode may cause a loss of balance and/or damage to the product.
5. All four legs touch the floor simultaneously at all times.
6. User evenly distributes weight between the arm supports when transferring in and out of the commode.
7. Armrests are dry.

Figure 1

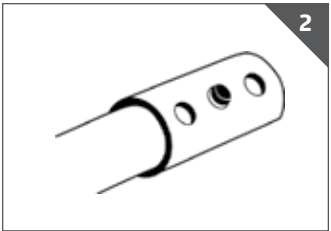
Figure 2

Assembly

The frame is shipped partially assembled. If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, making sure you have all parts.

1. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the pail brackets.) Push down on seat until clamps snap firmly into place.
2. Slide pail in from the front of commode.

Adjusting leg height

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes, per Figure 2.
2. Check that the legs are locked into place by pushing down and pulling up on each leg.

NOTE: If any part or mechanism appears to be unsafe or damaged, **DO NOT USE.**

If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE (633-5463) | 1-800-268-2848.

Cleaning

1. Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
2. DO NOT use abrasive powders, cleaners or scouring pads.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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