

# LUXURY ORTHOPEDIC BACKREST LEATHER

BEST SUPPORT FOR YOUR BACK



**movinglife**<sup>®</sup>

**ATTO Accessories** are created specifically to enhance, support and work with the ATTO mobility scooters in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

SCAN TO LEARN  
HOW TO COMBINE  
OUR ACCESSORIES

## SEATING & SUPPORT



ATTO /ATTO  
SPORT Armrests



Seat cushion



Seat cushion  
leather



Luxury Swivel  
Seat Leather



Belt & Support  
Strap



Luxury Orthopedic  
Backrest

## STORAGE



Essentials  
Pouch



Front & rear  
basket



Carryall  
& Cushion



Carryall  
& Cushion leather



Pet Carrier &  
Cushion Black



Flight Kit



Oxygen Tank  
Holder and Case

## TECHNICAL



Cane / Crutches  
Holders



Mobile Phone  
Holder



Rear Light Lifting  
Handle



ATTO Left Hand  
Throttle Kit



Standard Flight  
Battery



Launch - H2 2023  
Long distance  
battery - not flight  
approved




Cable kit


# How to Use

## LUXURY ORTHOPEDIC BACKREST LEATHER


The kit includes:

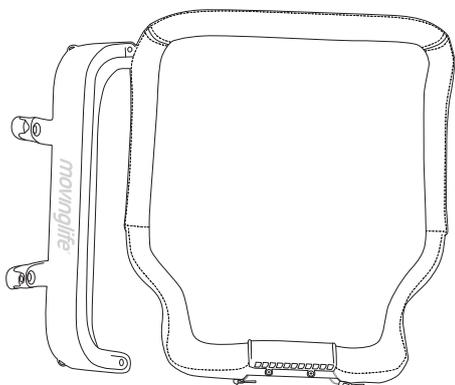

1. Orthopedic backrest and adaptor

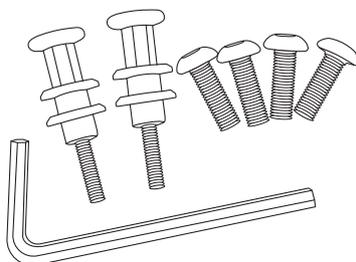


2. Screw kit:  
1 X Hex key  
4 X M5 screws  
2 X long knobs

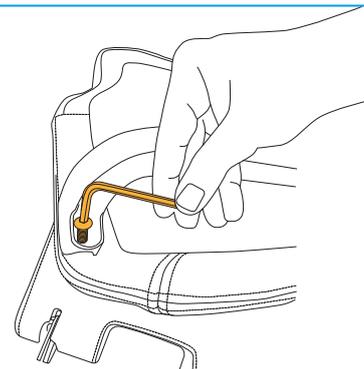


01

First, you need to connect the two parts of the orthopedic backrest to each other, with the help of the Hex Key you received with the kit.

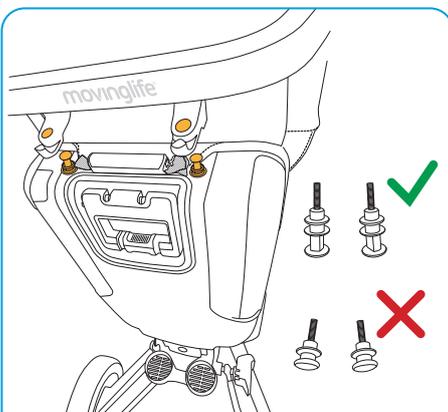


02

Unscrew the SHORT plastic knobs attached to the back. Replace them with the LONG knobs provided in the box. Then, place the holes of the orthopedic backrest over the knobs and make sure it sits firmly in place.

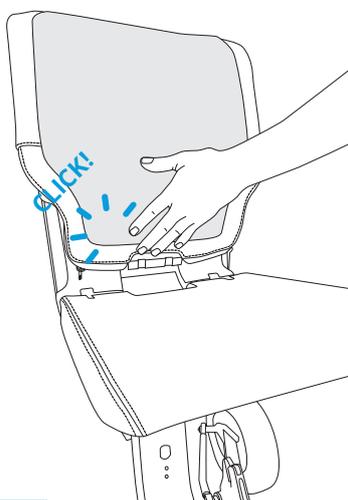

03

Fasten the lower part of the orthopedic back to the lower part of the chair, by pressing.

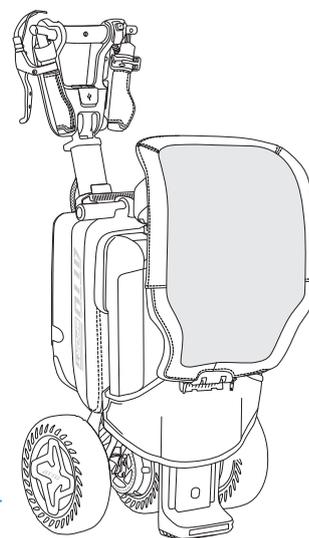

04

Folded mode: attach the orthopedic back to the back of the scooter back, with the long knobs.