


Washing Instruction Symbols and Definition

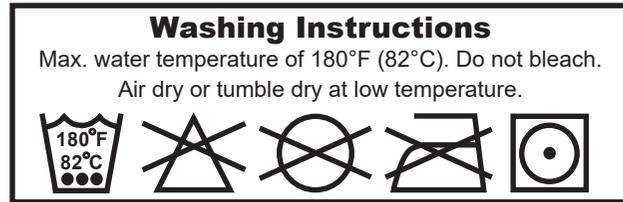

1. Machine Wash, Hot. Initial water temperature should not exceed 82°C or 180°F.
2. Do Not Bleach. No bleach product may be used. The garment is not colorfast or structurally able to withstand any bleach.
3. Do Not Dry Clean. Garment may not be commercially dry cleaned.
4. Do Not Iron. Item may not be smoothed or finished with an iron.
5. Tumble Dry, Normal, Low Heat. A machine dryer may be regularly used at a maximum of Low Heat setting.

Disposable Slings



Warning! DO NOT WASH disposable slings!!

**If label is red
do not use**

If this label is red, the disposable sling is NOT SAFE to use. Replace immediately!

Warranty


Each Bestcare sling is manufactured and tested using approved industry standards. Bestcare LLC warrants that all Bestcare slings are free from defects in materials and workmanship for a period of six months from the date of purchase. It is the responsibility of the purchaser to verify the integrity of each sling prior to each and every use. Slings that have been altered or repaired by the purchaser in any way or washed in an unsafe manner are not covered under warranty.

Cradle Clip Sling Owner's Manual

Bestcare
L.L.C.

2920 Pacific Dr
Norcross, GA 30071
678-679-6690

Sling Hook-Up Guide

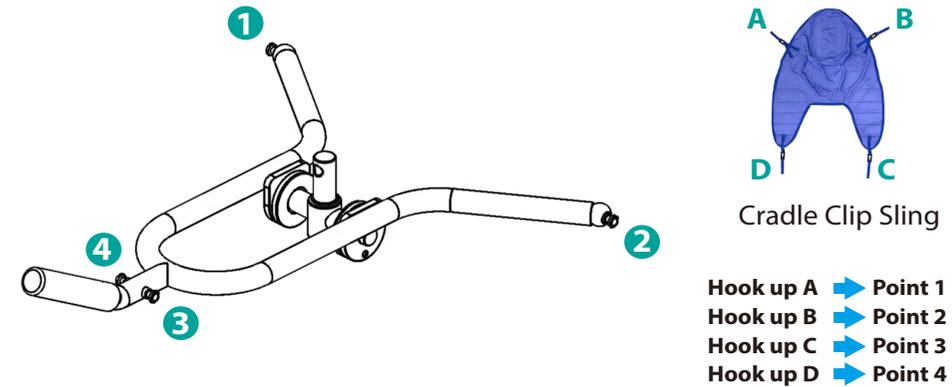

Cradle Clip Sling		
Sling Model	Sling Size	Weight Capacity
MAA4000M-M	M	600LB
MAA4000M-L	L	600LB
MAA4000M-XL	XL	600LB

Cradle Clip Disposable Sling		
Sling Model	Sling Size	Weight Capacity
MFA1000M-M	M	600LB
MFA1000M-L	L	600LB
MFA1000M-XL	XL	600LB



Refer to Owner's Manual for more operating instruction.



Warning!

This sling is only compatible with Bestcare Cradle spreader bar

DO NOT exceed the sling weight capacity.

IF any slings with higher load capacity than the equipment itself is used, **DO NOT** load the equipment more than the lowest maximum safe working load.

Do not alter the slings.

If the sling is frayed, torn, ripped, loops broken or any stitching is not secured, it is not safe to use the sling and must be replaced immediately!

Prior to lifting also ensure the size of the sling is appropriate for the person being lifted. Using a sling too small or too large is very dangerous and may result in serious injury.

Do not use the slings if you or the staff is not familiar with these procedures. Bestcare LLC accepts no liability for slings used incorrectly or after they have been damaged or are in need of replacement.

Please contact Bestcare LLC or your Bestcare dealer if you have any questions regarding the safe usage of Bestcare slings.



Sling Clip



Sling Hook



- Slide the clip over the sling hook on the spreader bar through the hole
- Pull down the clip to lock the sling on to the spreader bar
- Before lifting the patient, pull down each sling loops to ensure the loop is hooked onto the spreader bar properly



- Slide the sling down between the seat and the user's back. Use the pouch on the back of the sling to help pushing the sling down to the lower back.
- Feed the leg sections (C and D) under the thighs.
- Gently pull the leg section up the inner thigh.
- Hook A to 1 and B to 2
- Hook C to 3 and D to 4



Scan code for video demonstration