

Suspended Scale (2016 -)



Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

The content of this publication may not be copied either whole or in part without the consent of Arjo.

Foreword	4
Safety Instructions	5
Scale Overview and Models	6
Scale Models: 700-00511 and 700-00531	7
Attachment to track lift with conventional spreader bar.	7
Scale Models: 700-00511 for Ergolift 600.....	8
Attachment to compatible floor lift with conventional spreader bar.	8
Scale Models: 700-00526 and 700-00536	9
Attachment to track lift (LFXXXXX and 912XXXX models only).	9

Foreword

Thank you for purchasing Arjo equipment.

Please read and fully understand these Assembly and Installation Instructions before using the scale.

These *Instruction* are important for your safety and must be read and understood to help prevent possible injury.

Unauthorized modifications on any Arjo equipment can affect safety. Arjo will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any unauthorized modification to its products.

Support

If you require further information, please contact Arjo for comprehensive support and maintenance to maximize the long-term safety, reliability and value of the product.

Contact your local Arjo representative for spare parts. The telephone numbers appear on the last page of this *Assembly and Installation Instruction*.

Make sure that the assembly is carried out

- by qualified personnel.
- in accordance with local codes.
- in accordance with local regulations.

When the work is completed check that:

- all details are mounted corresponding to the assembly instructions.
- all parts which a user may come into contact, are free from sharp edges.
- the scale functions operate correctly.

Definitions in this Assembly and Installation Instructions

WARNING

Means: Safety warning. Failure to understand and follow this warning may result in injury to you or to others.

CAUTION

Means: Failure to follow these instructions may cause damage to all or parts of the system or equipment.

NOTE

Means: This is important information for the correct use of this system or equipment.



Means: The name and address of the manufacturer.

Safety Instructions

WARNING

To avoid injury, always follow the allowed combination listed in the *Instructions for Use (IFU)*.

NOTE

Refer to the exploded view corresponding to your equipment for proper assembly instructions.

WARNING

The Scale is intended to be used for patients whose weight is within a specified safe working load (SWL). To avoid injury, do not attempt to lift more than the lowest weight limit indicated on the following:

- the track system;
- the lift;
- the “maximum load” on the lift;
- the “maximum load” label on the scale;
- on the spreader bar;
- on the sling.

CAUTION

CLASS III MODEL 700-00531, 700-19490 and 700-00536 scales must be verified, calibrated and sealed by certified personnel only. Contact your local dealer to get more details.

Scale Overview and Models


Scale Models:	Compatible with products:
• 700-00511(Rest of the world) • 700-00531 (Class III - European market) 	• Maxi Sky 600 (V4, V4i) • Maxi Sky 440* • Voyager* • Voyager Duo • V5 Duo • Ergolift 600 (only with scale model 700-00511) * Use portable adaptor, part number 700.05725. See the <i>Portable Adaptor Installation Procedure</i> for details.

Scale Models:	Compatible with products:
• 700-00526 (Rest of the world) • 700-00536 (Class III - European market) 	• Maxi Sky 1000 (V10)

Scale Models: 700-00511 and 700-00531


This applies to product Maxi Sky 600 (LD1XXXX), Voyager Duo (913XXXX) and V5 Duo (923XXXX).

Attachment to track lift with conventional spreader bar.

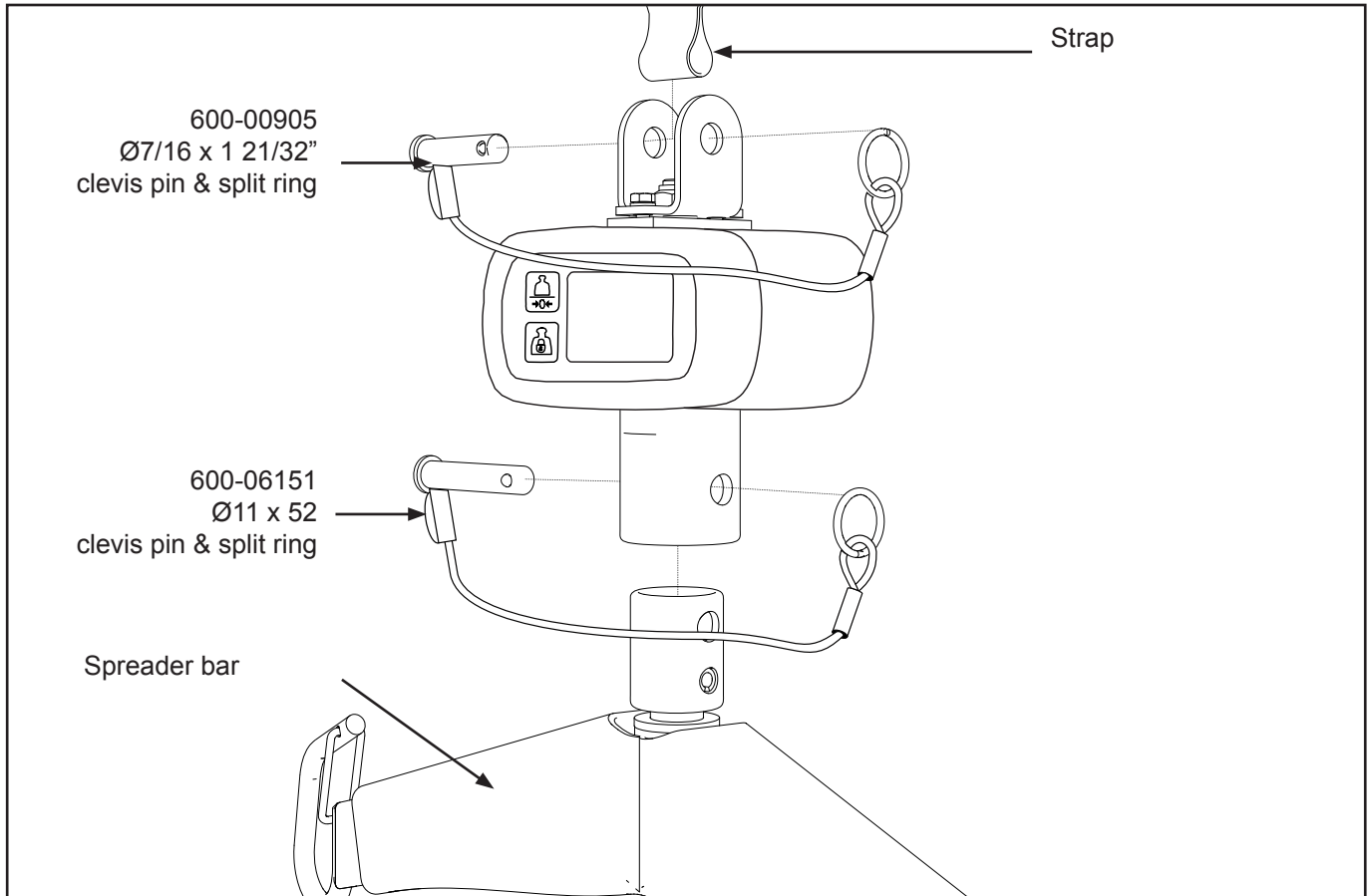


WARNING

To avoid injury, make sure that all clevis pins are correctly secured by means of the split ring.

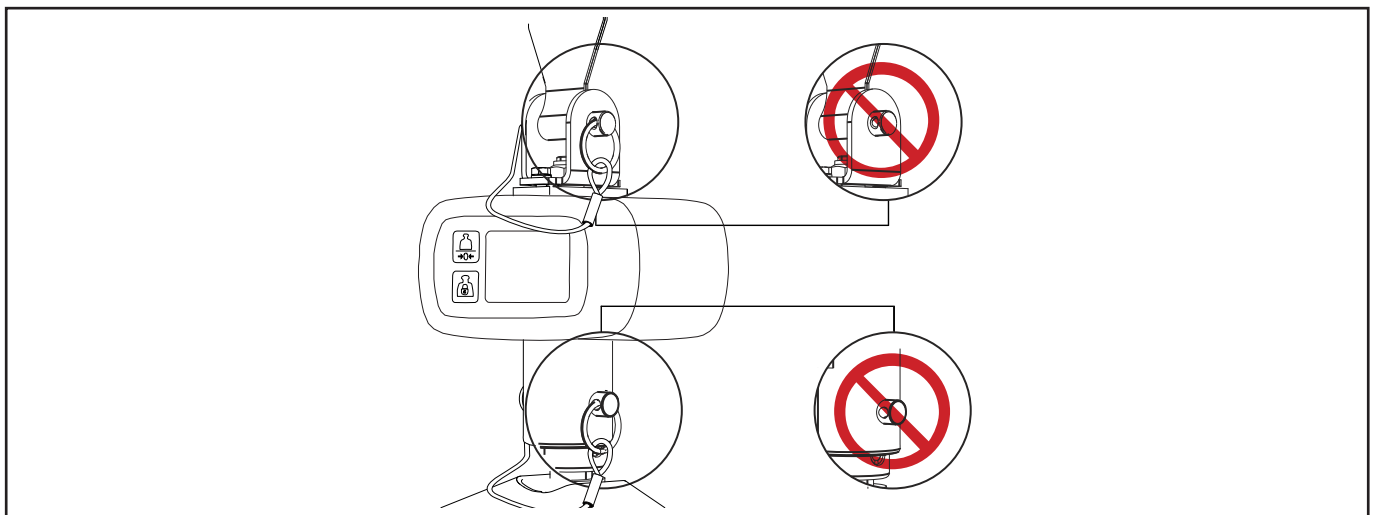


Scale Models: 700-00511 for Ergolift 600

Attachment to compatible floor lift with conventional spreader bar.

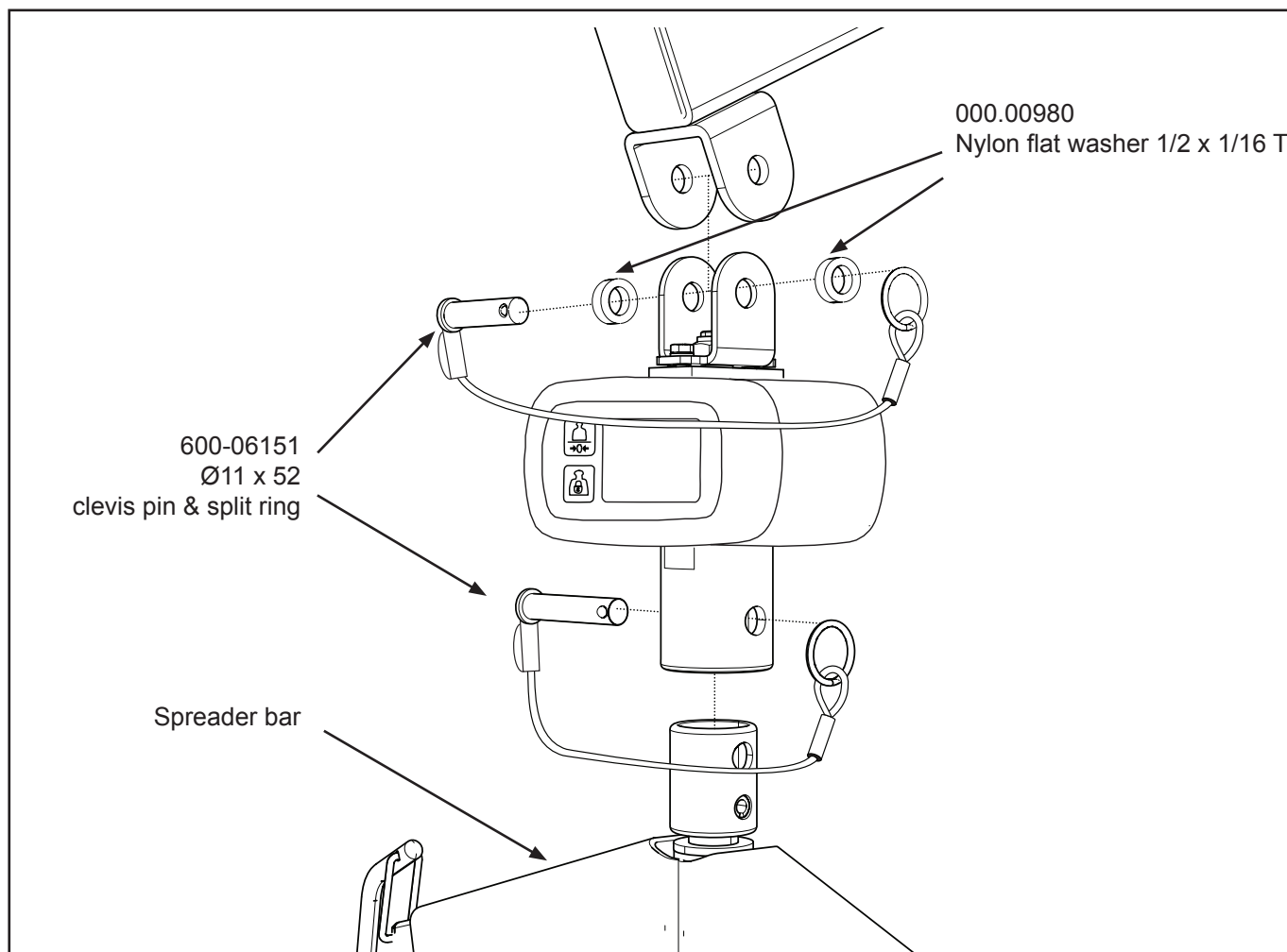

WARNING



To avoid injury, make sure that all clevis pins are correctly secured by means of the split ring.

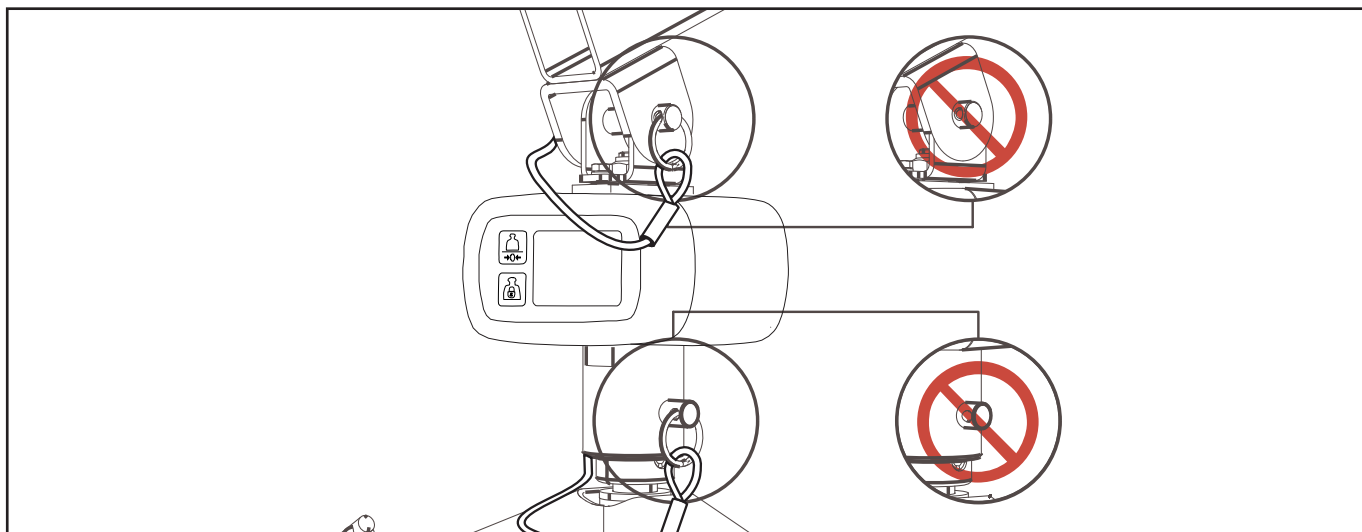

Scale Models: 700-00526 and 700-00536


This model applies to Maxi Sky 1000.

Attachment to track lift (LFXXXXX and 912XXXX models only).

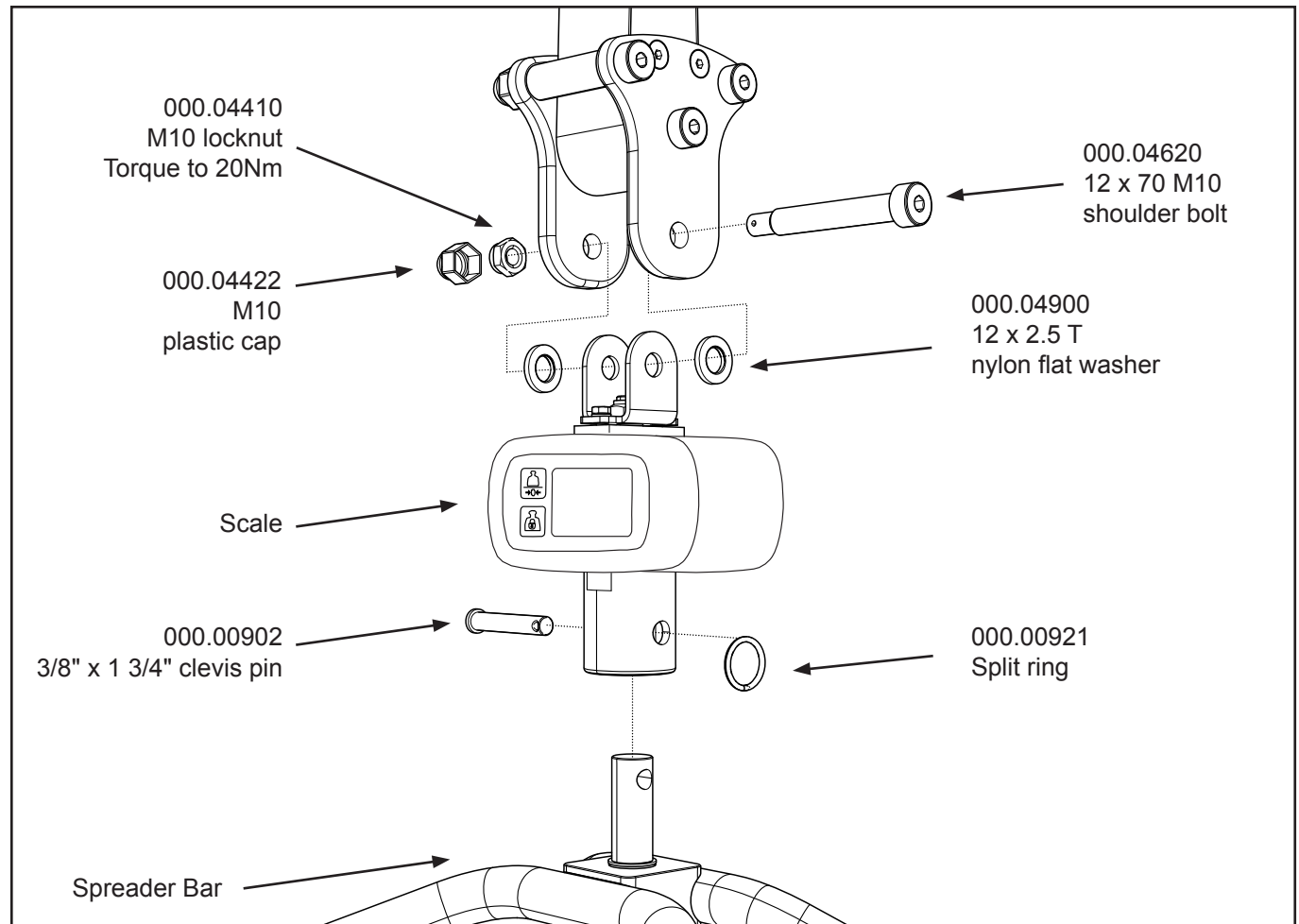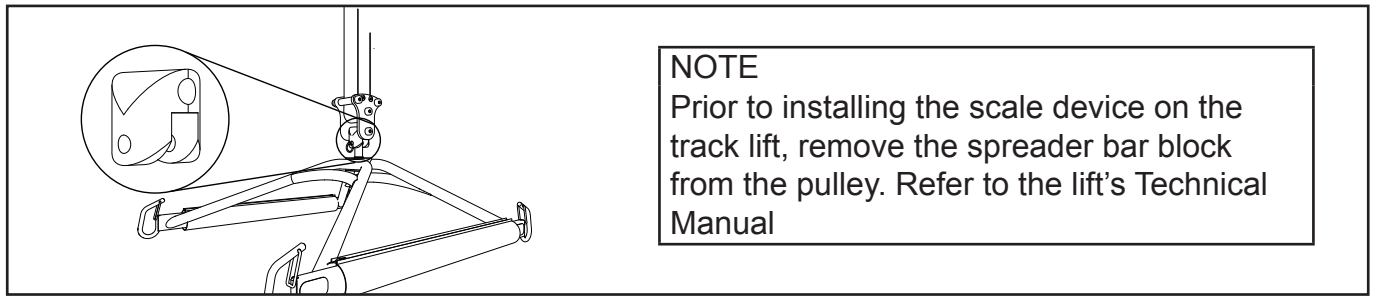

WARNING

To avoid injury, make sure that all clevis pins are correctly secured by means of the split ring and that the lock nut is fastened.

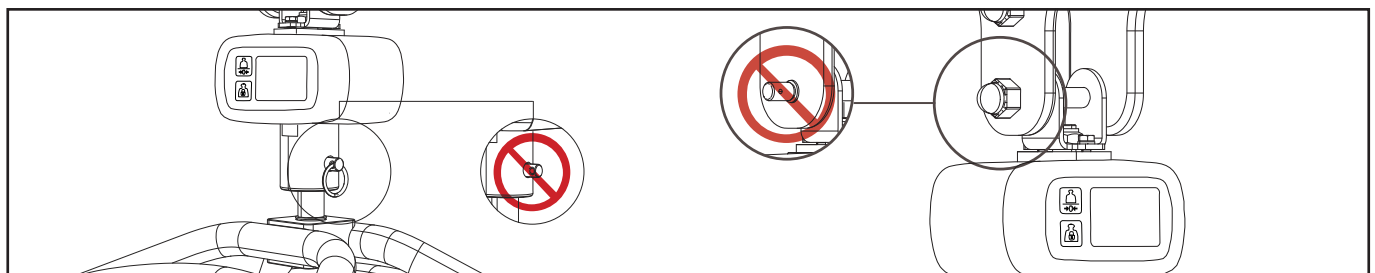

Intentionally left blank

AUSTRALIA

Arjo Australia Pty Ltd
78, Forsyth Street
O'Connor
AU-6163 Western Australia
Tel: +61 89337 4111
Free: +1 800 072 040
Fax: +61 89337 9077

BELGIQUE / BELGIË

Arjo NV/SA
Evenbroekveld 16
BE-9420 ERPE-MERE
Tél/Tel: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info.belgium@arjo.com

BRASIL

Arjo Brasil Equipamentos Médicos Ltda
Rua Marina Ciufuli Zanfelize, 329 PB02 Galpão
- Lapa
São Paulo – SP – Brasil
CEP: 05040-000
Phone: 55-11-3588-5088
E-mail: vendas.latam@arjo.com
E-mail: servicios.latam@arjo.com

CANADA

Arjo Canada Inc.
90 Matheson Boulevard West
Suite 300
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 905 238 7880
Free: +1 800 665 4831 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 905 238 7881
E-mail: info.canada@arjo.com

ČESKÁ REPUBLIKA

Arjo Czech Republic s.r.o.
Na Strži 1702/65
140 00 Praha
Czech Republic
Phone No: +420225092307
e-mail: info.cz@arjo.com

DANMARK

Arjo A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail:
dk_kundeservice@arjo.com

DEUTSCHLAND

Arjo GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjo.com

ESPAÑA

Arjo Ibérica S.L.
Parque Empresarial Rivas Futura, C/Marie
Curie 5
Edificio Alfa Planta 6 oficina 6.1-.62
ES-28521 Rivas Vacia, MADRID
Tel: +34 93 583 11 20
Fax: +34 93 583 11 22
E-mail: info.es@arjo.com

FRANCE

Arjo SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjo.com

HONG KONG

Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND

Arjo BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND

Arjo Ltd
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE

Arjo Norway AS
Olaf Helsets vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

ÖSTERREICH

Arjo GmbH
Lemböckgasse 49 / Stiege A / 4.OG
A-1230 Wien
Tel: +43 1 8 66 56
Fax: +43 1 866 56 7000

POLSKA

Arjo Polska Sp. z o.o.
ul. Ks Piotra Wawrzyniaka 2
PL-62-052 KOMORNIKI (Poznań)
Tel: +48 61 662 15 50
Fax: +48 61 662 15 90
E-mail: arjo@arjo.com

PORTUGAL

Arjo em Portugal
MAQUET Portugal, Lda.
(Distribuidor Exclusivo)
Rua Poeta Bocage n.º 2 - 2G
PT-1600-233 Lisboa
Tel: +351 214 189 815
Fax: +351 214 177 413
E-mail: Portugal@arjo.com

SUISSE / SCHWEIZ

Arjo AG
Fabrikstrasse 8
Postfach
CH-4614 HÄGENDORF
Tél/Tel: +41 (0) 61 337 97 77
Fax: +41 (0) 61 311 97 42

SUOMI

Arjo Scandinavia AB
Riihitontuntie 7 C
02200 Espoo
Finland
Puh: +358 9 6824 1260
E-mail: Asiakaspalvelu.finland@arjo.com

SVERIGE

Arjo International HQ
Hans Michelsensgatan 10
SE-211 20 MALMÖ
Tel: +46 (0) 10 494 7760
Fax: +46 (0) 10 494 7761
E-mail: kundservice@arjo.com

UNITED KINGDOM

Arjo UK and Ireland
Houghton Hall Park
Houghton Regis
UK-DUNSTABLE LU5 5XF
Tel: +44 (0) 1582 745 700
Fax: +44 (0) 1582 745 745
E-mail: sales.admin@arjo.com

USA

Arjo Inc.
2349 W Lake Street Suite 250
US-Addison, IL 60101
Tel: +1 630 307 2756
Free: +1 800 323 1245 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 630 307 6195
E-mail: us.info@arjo.com

JAPAN

Arjo Japan K.K.
東京都港区虎ノ門三丁目7番8号 ランディッ
ク第2虎ノ門ビル9階
電話 : +81 (0)3-6435-6401

Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.



ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

CE
2797