


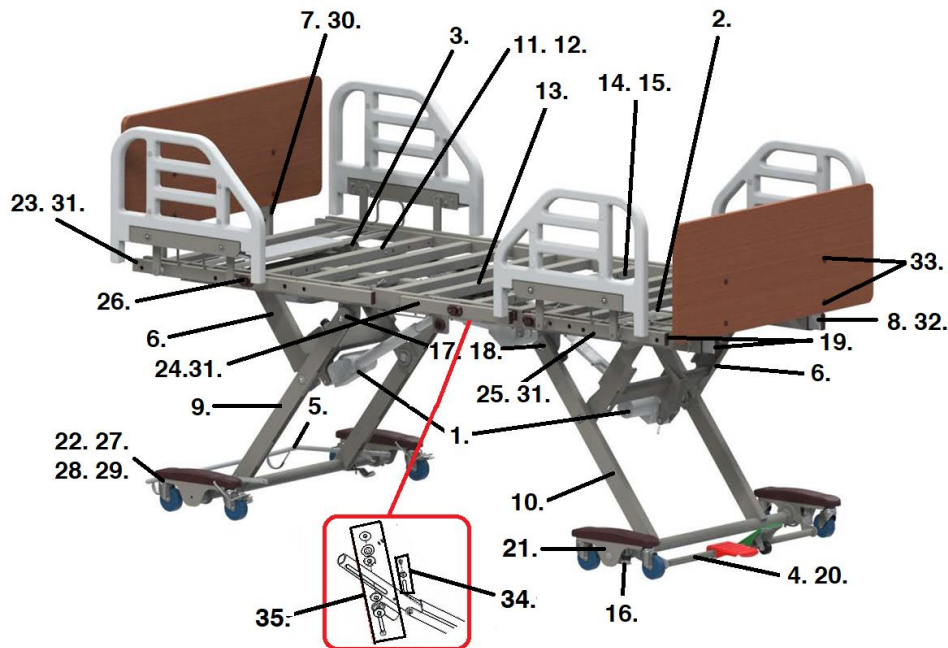

- | | |
|---|---|
| 1. Hi/Lo Actuator- SP01- TA23-1060-004 | 22. Head End Truck Assembly Left- SP01-P907-12E |
| 2. Leg Actuator- SP01-TA31-1060-007 | 23. Head Width Ext. Assembly Right- SP01- P750-10019E |
| 3. Head Actuator- SP01-TA31-1060-008 | 23. Head Width Ext. Assembly Left- SP01-P750-10020E |
| 4. Brake Pedal Assembly- SP01-P750-10016E | 24. Middle Width Extension Left- SP01-P750-10023E |
| 5. Wall Bumper Assembly- SP01-P907-10E | 24. Middle Width Extension Right- SP01-P750-10024E |
| 6. Leg Assembly (Upper)- SP01-P750-10001E | 25. Foot Width Extension Right- SP01-P750-10022E |
| 7. Head Board Brackets- SP01-P907-05E | 25. Foot Width Extension Left- SP01-P750-10021E |
| 7. Head Board Bracket Knobs (pr)- SP01-P903-10006E | 26. Width Extension Button- SP01-P903WE-10007E |
| 8. Foot Board Brackets- SP01-P907-06E | 27. Non-Locking Caster- SP01-P907-11-03E |
| 9. Leg Assembly (Head)- SP01-P750-10002E | 28. Locking Caster -SP01-P907-11-02E |
| 10. Leg Assembly (Foot)- SP01-P750-10003E | 29. Caster Bail- SP01-P907-11-04E |
| 11. Head Deck- SP01-P750-10004E | 30. Headboard Lock Pin- SP01-UA30120731E |
| 12. Head/Thigh Deck Bolt/Washers(set)- SP01-P750-10007E | 31. Width Ext Locking Pin- SP01-UA30120451E |
| 13. Thigh Deck- SP01-P750-10005E | 32. Foot Board Bracket Locking Pin- SP01UA30170361 |
| 14. Foot Deck- SP01-P750-10006E | 33. Head/Foot Board Mtg. Screws- SP01-P903-10008E |
| 15. Foot Deck Bolt/Washers (set)- SP01-P750-10008E | 34. Foot Lift Actuator Hardware- SP01-P902-10017E |
| 16. Adjustable Foot- SP01-PA18108302E | 35. Pin Grommet Hardware- SP01-P902-10040E |
| 17. Bearing Kit (Pr)- SP01-P902-10018E | Not Shown |
| 18. Bearing Kit- SP01-P902-10004E | Control Box- SP01-TC21-1060-004 |
| 19. Bed End Plastic Plug Set- SP01-P750-10014E | Power Cord- SP01-P703-60-02E |
| 20. Brake Assembly- SP01-P907-13E | Hand Pendant W/Backlit- SP01-TH12-1060-001 |
| 20. Brake Pedal Green (ea)- SP01-PO10040209E-G | "Y" Cable- SP01- P703-70E |
| 20. Brake Pedal, Red (ea)- SP01-PO10040209E-R | "Y" Cable Mounting Bracket- PAN6300-36E-DS |
| 21. Foot End Truck Assembly Right- SP01-P750-10017E | Foot Deck Mounting Brackets- SP01-P750-10009E |
| 21. Foot End Truck Assembly Left- SP01-P750-10018E | Under Bed Light- SP01-TBL-1060-001 |
| 22. Head End Truck Assembly Right- SP01-P907-11E | Pull Pin Clip Hardware Kit- SP01-P903-10017E |

[1] When stock is gone the part is no longer available.

[2] Stock is gone the part is no longer available.