



# Cushion Instructions



## Package Contents:

- one (1) LTV SEAT cushion (adjustable air insert)
- easy-to-use hand pump
- zippered, padded outer cover
- instruction sheet

## Adjusting Your LTV SEAT Cushion:

1. Turn the valve counterclockwise to open.
2. Add air using the pump until all of the cells are visibly filled. Turn the valve clockwise to close, then remove the pump.
3. Place the cushion on your chair with the non-skid side down and the zippered portion of the cushion at the back. The valve should be on the front left corner.
4. Sit on the cushion in your normal sitting position.
5. Open the valve to release air. As air is released, you will slowly sink into the cushion. Release air until you are close to a comfortable level and then turn the valve clockwise to close. Fine tune by releasing air or pumping more air in until you reach your desired comfort level.

## Washing Instructions:

Hand wash, mild detergent. Wipe or air dry.



## CAUTION

Protect the LTV SEAT cushion from pins or any other sharp objects as well as exposure to hot ashes.

## WARNING

Do not use as a water flotation device as it will not support you in water.

MADE IN THE U.S.A.



100 north florida avenue  
belleville il 62221-5429 USA  
p 800.851.3449 618.277.9173  
f 618.277.9561  
www.shapefitting.com

© 2005 The ROHO Group, Inc. LTV SEAT™, ROHO®, and Shape Fitting Technology® are registered trademarks of The ROHO Group.

## Limited Warranty

### WHAT IS COVERED?

The ROHO Group covers any defects in materials or workmanship of the entire line of ROHO® Shape Fitting Technology® products.

### FOR HOW LONG?

12 Months from the date the ROHO Shape Fitting Technology Cushioning Product was originally purchased as a new cushion, no matter who is the present owner.

### WHAT WE WILL DO.

Within a reasonable amount of time after you return the ROHO Shape Fitting Technology product to us, we will repair any defect in material and workmanship and ship it back to you, all free of charge.

### WHAT WE WILL NOT DO.

We will not repair a cushion free of charge if it has been misused, damaged by an accident or if the product is not under warranty. This also includes damages stemming from acts of God, e.g., flood, tornado, earthquake or other catastrophic event, e.g. fire. Please note that any alteration to the product, unauthorized by The ROHO Group, Inc., will void the warranty.

### WHAT THE CUSTOMER MUST DO.

The customer must send the ROHO Shape Fitting Technology Cushioning Product to us with the original sales ticket or other evidence of date of purchase, and must pay all delivery costs. It must be sent to The ROHO Group, 100 N. Florida Avenue, Belleville, IL 62221-5429 U.S.A. We will not require the original sales ticket if you or the previous owner, if applicable, have completed and returned the warranty registration card, attached to the operations manual, and we have it on file. Failure to submit the original sales ticket or failure to have the warranty registration card on file voids the warranty.

### ANY OTHER QUESTIONS?

Call The ROHO Group toll free in the U.S. or Canada at (800) 851-3449, or outside the U.S. and Canada at 1-618-277-9150 if you have any additional questions regarding warranty coverage.

### DISCLAIMER

Any implied warranties, including those of MERCHANTABILITY and FITNESS FOR A PARTICULAR PURPOSE are also limited to 12 Months from the date of original purchase and any action for a breach of such warranties or any express warranties herein must be commenced within this time. The ROHO Group, Inc. shall not be responsible for any incidental or consequential damages with respect to economic loss or injuries to property, whether as a result of a breach of express or implied warranties or otherwise. The ROHO Group, Inc. will not be responsible for the results of careless handling, unreasonable or improper use of this product.

### WHAT ARE YOUR STATE LAW RIGHTS?

This warranty restricts the duration of all implied warranties to 12 Months and limits or excludes recovery for incidental or consequential damages. Some states do not allow limitations on how long an implied warranty lasts and some states do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights, which vary from state to state.