

## HIGHLANDER

# **Enclosure with Commercial Platform Lift**



EPL600 EPL800

**EPL400** 

**EPL1000** 

**EPL1200** 

**EPL1400** 



Read manual thoroughly before attempting to install or operate lift.

This manual has been provided to assist you with lift installation and operation. For further assistance please contact your authorized Harmar dealer or Harmar's Technical Service department.

Tel: 866-378-6648 Fax: 816-537-0641

Dea	ler
Dea	וכו

Serial Number:

## **Table of Contents**

### Installing the lift

Using the Manual	2
When You Receive the Lift	3
Specifications	3
Safety	3
Code Requirements	4
Site Requirements	4
Required Tools	4
Required Materials	4
Preparing to Install the Lift	5
Controller Harness Connections	5
Placing the VPL	6
Installing the platform	8
Installing the outer guard panel	8
Installing the Enclosure walls	9
Installing the fixed ramp	11
Anchoring the Lift	12
Setting the Limit Switches	
Verifying Operation of the Lift	14
Manual Override	15
Call-Sends (optional)	1ĺ
Top Landing Gate (optional)	1Î
EMI and Flush Strike (optional)	1Ï
Alarm (optional)	1Ì
Owner's Section	1J
Safety	Œ
Controls	Œ
Operating the Lift	20
Warranty	3F

## **Using the Manual**

This manual will provide step by step instructions on how to install and operate your lift. Read and understand the entire manual before beginning to install the lift.

If you have any questions, contact Harmar technical service at 1-866-378-6648

## When You Receive the Lift and Enclosure

- Check the lift and enclosure for shipping damage. If you see any damage, contact the freight carrier to file a damage claim.
- Verify that the packing list attached to the exterior packaging reflects your intended order.
- Verify that you have received the items on the packing list.

## **Specifications**

Payload Capacity\_\_\_\_ 750 lbs

Vertical Travel\_\_\_\_ 54" – 171"

Height\_\_\_\_ 77" – 197"

Foot Print\_\_\_\_ VARIES

Platform Sizes\_\_\_ 36" x 60" or 42" x 60"

Input Voltage\_\_\_ 115vac - 20a (or 24 VDC for DC lift, optional)

Control Voltage\_\_\_ 24vac or 24vdc

Platform Speed\_\_\_ 10 ft/min

Motor\_\_\_ 1/2hp-90vdc or 1/2hp-24vdc

## <u>Safety</u>

- Read all instructions in this manual before installing or operating the lift.
- Do not exceed the maximum payload capacity of 750 lbs.
- Do not ride on the lift until it is anchored in place.
- This product is designed only for lifting people and wheel chairs. Do not use it for any other purpose.
- Always wear eye protection while installing or servicing this product.
- Always disconnect this product from the electrical source before servicing it.
- Only use the fasteners supplied with this lift.
- Do not wear loose clothing or jewelry when working on this product.
- Do not disable any safety equipment or switches supplied with this lift.
- Stay away from all drive train components while the lift is operating.

## **Code Requirements**

Your lift and enclosure have been designed to meet ASME A18.1 section 2 and 5 and CSA B44/ASME A17.5, with the addition of certain options. Code requirements for Vertical Platform lifts vary depending on location.

It is the installers' responsibility to contact their local code enforcement office and determine all of the regulations they are subject to. You must do this before installing the Vertical Platform Lift. If you have questions about this, feel free to call Harmar to be directed to the right officials.

## Site Requirements

- 115vac 20amp grounded circuit, if outdoor, GFI protected...
- Minimum 5" thick, level, 3,500 psi reinforced concrete slab sized to your specific install drawing.
- If you choose to install the lift in a pit so the platform is at floor-level, Harmar recommends that the pit be sized to install the enclosure in it also.

## Required tools

- Step ladder
- Extension ladder or scaffold (helpful for tall units)
- Hammer Drill
- 3/8" Masonry Drill Bit, min 8" long
- Appliance Dolly
- Hammer
- Level
- Measuring Tape

- Socket Wrench Set
- Socket wrench or drill extensions up to two-feet long (if installing walls from platform)
- Phillips screwdriver or bits, size #2 & #3
- Wire stripper and cutter
- Conduit cutters, punches and tools needed with electrical conduit

## Required materials

- Up to 14 concrete anchors, wedge-type. Harmar recommends securing the lift with our Anchor Kit p/n
  If you use your own, we recommend 3/8-16 anchor bolts with embedment of 4 inches. These shall
  have a min pull out strength of 3000 lbs-force. If used in an extreme environment (wet, salty), use
  galvanized or stainless steel
- Various shims, slotted preferred, but washers can work. Min reqd. about 1/16<sup>th</sup> thick, Qty, up to about 25 max. These are to align enclosure and lift with the building. Often these aren't needed.
- Lag screws or similar and washers to attach upper portion of the enclosure to the building. You may need lumber for this step too.
- Up to ten wire nuts, med size.
- Caulking, outdoor 25 year-silicone preferred
- Electrical conduit and fittings. For outdoor installation, use liquid-tite and appropriate fittings

## **Preparing to Install the Lift**

#### Site Requirements:

Verify the surface the lift will mount to is smooth and level. This surface must be made from 3,500 psi reinforced concrete with a minimum thickness of 4".

Verify that there is enough space for the lifts foot print. Be sure to include space necessary for platform.

## Caution: Verify that you have overhead clearance and the lift complies with codes for your area.

- Determine if the overhead height of the enclosure will encounter any difficulties with the building (soffits, building overhang, gutters) and determine a plan to deal with these.
- For enclosures without a dome, code requires 6' 8" of clearance above the platform floor when the lift is at the upper landing. Overhead clearance is your responsibility
- Since you've purchased a Harmar enclosure for your lift, side-to-side running clearances required by codes are satisfied by this design.

Warning: If you've purchased a 3-sided enclosure, the area between the lower floor and the top landing must be smooth and vertical so that the distance from the wall to the platform will stay between 3/8" and 3/4" to meet code. This is to eliminate any pinch points. If you're using your own upper flush-door, you must also assure the threshold meets this dimension requirement too, so there won't be a pinch point.

#### **Electricity Requirements**


The lift will require a 115 VAC 20 amp grounded electrical circuit.

Depending on local codes, this connection may need to be routed in electrical conduit and hard-wired.

If you have a dome and ventilation/light unit, you'll need to get 115 VAC 4 amps to it also. This can be wired in conduit to the upper terminal strip of the Pinnacle or separately,

Warning: Do not ride on the lift until it has been anchored in place.

## **Controller Harness Connections**

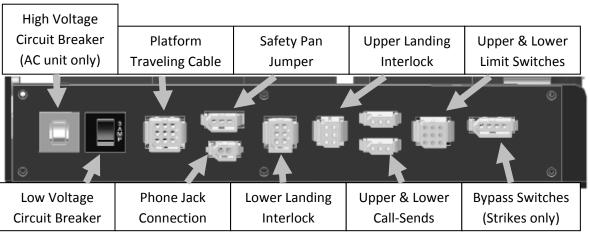

## Placing the VPL:

#### Step 1


Unwrap and un-package the Vertical Platform Lift as near to the installation location as practical. Be careful to not scratch or damage any of the panels. You'll unwrap the platform and its panels and a small-parts box. Just set these out of the way for now.

#### Step 2

Since it's shipped horizontal, you'll need to stand it up and move it to approximately where it will be installed. These lifts weigh between 700 and 1100 lbs without their platforms, depending on their height, but when lifting, you'll only be lifting one end, so half that weight. Though you may use equipment such as a loader or a come-along or winch to stand them up, three strong people are usually sufficient to do this. If you use equipment to stand it up, be sure to lift from the heavy steel frame on the inside and use straps—don't lift from the outside body panels. Note that a VPL for an enclosure differs from a 'normal' VPL because it does not have a top-cap and its platform doesn't have a safety pan. Part of the enclosure will serve as the top cap, and a pit-switch does the duty of the safety pan.


#### Step 3

Position the lift to the building according to the dimensions below. Note that both platforms,36" and 42" center on the door of the building, the dimensions are shown below

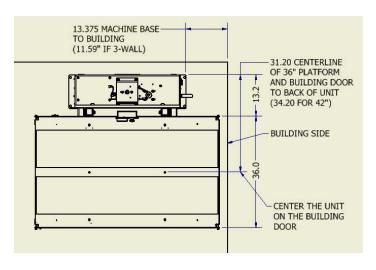

#### Step 4

Install the tower-side walls that will be against the building side onto the VPL lift. If it's a very tall lift, just put in a few lower ones, you can do the rest when you have more of the enclosure assembled. You'll need to install the lowest one first, that has vents. Note the one that has the large hole in it for the pit switch will go on the other side of the tower, don't use it yet. To install these to the tower, you'll remove the screw that's in the tower and put the side panel there.

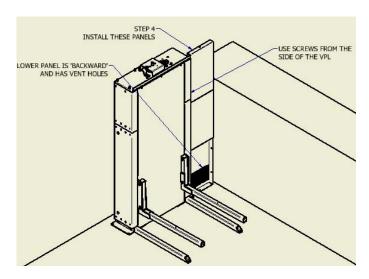


#### Sten 5

Un-package the enclosure parts. If you've ordered an enclosure with acrylic windows, they'll arrive with a paper or plastic protectors on all but the outside edges of the windows. You should leave these protectors on the windows until you're ready to finally place them; at that time, you may peel and dispose of them. Also, be careful not to scratch the powder coat on any of these parts.


#### Step 6

Put the enclosure alignment panel across the ends of the two tower legs. The panel is to assure that you get the correct spacing of the vertical panels to the lift.

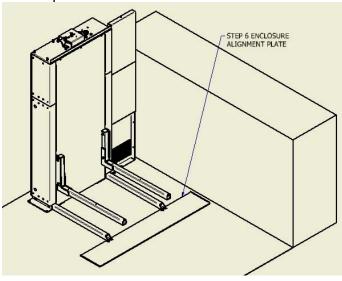


#### Step 7

Next you'll need to bolt together some of the panels on the wall-side first because you won't have access to these bolts and holes when it's together. If you have a really tall tower, you'll be able to bend it out to put in the higher bolts, but go ahead and build most of this. Use the 3/8" x 1.0 washer-face SS bolts that you use for most of this assembly. Also, 'pop' a hole plug into the outside hole of these after you've installed the bolt. Next you'll need to bolt together some of the panels on the wall-side first because you won't have access to these bolts and holes when it's together. If you have a really tall tower, you'll be able to bend it out to put in the higher bolts, but go ahead and build most of this. Use the 3/8" x 1.0 washer-face SS bolts that you use for most of this assembly. Also, 'pop' a hole plug into the

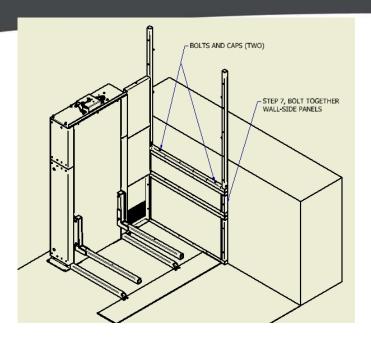

#### Step 8

Put bolts through the tower side panels (step 5) into the wall panels that you just installed (step 7). Same bolts as before.

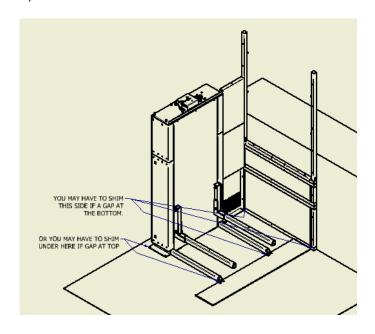

#### Step 9

At this time, look at the spacing for the tower-side panels and decide if you're going to need to shim the VPL unit to make it fit the wall. The tower side panels' edge needs to be parallel with the wall. Depending on the angle, if it's open at the bottom, shim the wall, if open at the top, shim the tower. The shims will have to go under the enclosure walls that you haven't put up yet also.


## **Installing the Platform**

#### Step 1

The mounting bolts, nuts and spacers that secure the platform to the lift carriage are packaged in the small parts box. Begin by locating these pieces and them setting aside.



## **Step 2**Position the platform by aligning the support legs with the carriage flanges that protrude from the front cover.

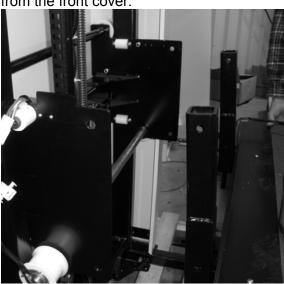

#### Step 3

Align the four mounting holes with those on the carriage. Insert  $\frac{1}{2}$  x 3" hex head bolts into each of the (4) holes. Install the low profile nuts on the upper bolts and the standard ones on the lower holes.



#### Step 4


Plug in the harness for the platform safety pan and unplug the safety pan jumper on the controller (ref pg 5). Secure the harnesses under the clip on top of the carriage flange.



## **Installing Enclosure tower walls:**

#### Step 1

Place and shim (if necessary) the lowest front wall, the door frame and the tower-side walls on the door-side of the enclosure. Again, the lowest tower side-panel has the vent holes and goes in 'backward.' This side has the big hole for the pit-emergency stop switch. Assemble and install the pit switch and its box. Note, the doorframe may be received with the door already installed.

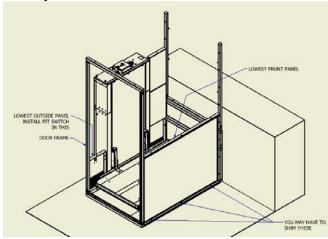



#### Step 2

Install the rest of the front and tower-side panels. Now you're able to put on the Lift Top Cap. For lifts taller than eight-foot lift, you may skip ahead to the anchor-step, page 10.

#### Step 3


Your door will come assembled to the frame and may already be on the unistrut (step 1) If not, put eight twist-type strut-nuts into the channels. The door assembly weighs more than 200 lbs, so lift it with two people minimum and be careful not to scratch it. Put eight bolts into it loosely. You'll need to adjust the position both vertically and horizontally to match the threshold of the ramp (or upper landing) If you have a gate, put the gate-mount brackets on the other side (or door if it doesn't have a gate) and put install the pre-assembled gate.

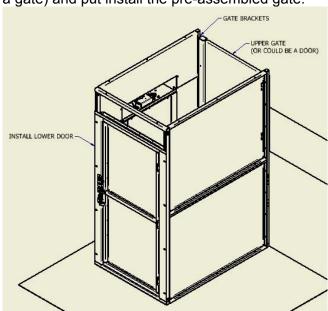

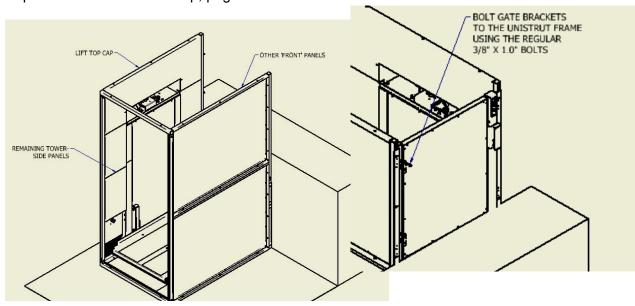

#### Step 4

Whether the enclosure uses a fixed ramp or you have a pit or have built your own ramp, the next step is to adjust the height of the doors or gates to fit their thresholds. Pry them gently to the correct height with a piece of wood to prevent damaging the finish. Check that the doors or gates are properly centered on the platform, then tighten the bolts. When you get the height right, you may need to fine-adjust the lower and upper stops on the lift to get it to stop in exactly the right place,

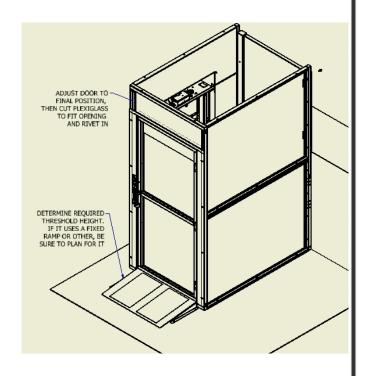

#### Step 6

You can adjust the upper and lower limit switches to make the lift stop exactly where needed. Sometimes it comes from the factory right, so this isn't required.

#### Step 5

You have received one piece of Plexiglas that isn't installed in any frame, even if you have an enclosure with metal panels. You will saw this piece to fit exactly in the space that's left above the door after you've adjusted the height. Measure the vertical height carefully, then mark the desired width on the plex. Place it on saw horses and clamp a long straight-edge to it, adjusted for the specific circular saw you plan to

use. With a sharp blade, moving slowly, you can saw the acrylic. Install the acrylic with the angles and flats provided, which rivet in.

## **Installing the Platform Guard Panels**

#### Step 1

Insert the guard panel posts into the pockets in the platform. The smooth side of the guard panels should face in toward the center of the platform. Bolt each post in place with (2)  $5/16-18 \times 2$ ° and (1)  $\frac{1}{4}-20 \times 2$ ° hex head bolts.



If you have a 90° exit platform, install the end guard panel using the provided bracket and bolt to inner guard panel.



**Step 2**Install the platform control box on the rear guard panel using screws and nuts through the panel.



#### Step 3

Plug in the harness for the platform control box. Secure the harnesses under the clip on top of the carriage flange.



## Installing Fixed Ramp

#### Step 1

- Position the ramp after the lift is in its final location.
- Maintain a gap of 3/8" to 3/4" between the ramp and platform.
- Drill a pilot hole into the concrete and fasten using supplied concrete lags through ramp flanges.

## **Anchoring the Lift**

#### **Anchors**


Harmar recommends securing the lift using the concrete anchors provided. If you purchase you own floor anchors they must be 3/8" minimum of sufficient length. All four floor anchors must be installed correctly in accordance to their instructions.

#### Step 1

- · Position the lift in its final location.
- Verify that it is level and perpendicular to its surroundings and all running clearances are the proper dimension.
- Shim if necessary.

#### Step 2

Use the lift's base as a template. Drill 4 holes into the concrete making sure that the holes are deep enough to accept the anchors.

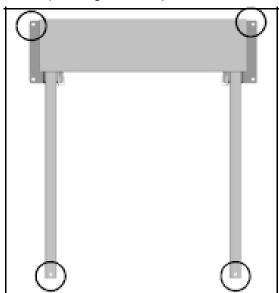

Tip: Concrete dust may have settled into the holes you just drilled. Use a shop vacuum to clean out these holes. This will ensure the floor anchors set correctly.


#### Sten 3

Secure the lift in place by tightening the floor anchor bolts.


Taller lifts must have the top of the lift tower anchored into a solid surface to ensure running clearances remain constant.

There are two options for anchoring the top of the tower:

1) For lifts going up to a landing such as a deck or porch, the optional tower brace is the preferred method. Instructions for the brace are packaged with it.

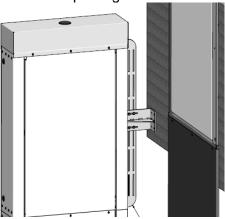

 For lifts that are placed with the back of the tower against a solid wall such as inside of a hoistway, drilling two holes through the top tower cross brace and anchoring is preferred.

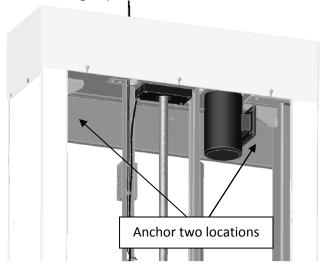

## **Setting the Limit Switches**

Your lift is equipped with upper and lower limit switches. The vertical location of these switches may be adjusted to fit your application. Typically the upper limit switch will need to be adjusted so the platform will stop level with the upper landing. The lower limit will typically not need adjusting.

#### Step 1

Verify the emergency switch is in the ON position. Run the lift in the up direction until the platform floor is level with the upper landing. Disconnect the lift's power (at the building's circuit breaker for AC units, at the battery box for DC units) before going to the next step.

Warning: Moving components can cut and crush. Do not operate the lift if you are in close proximity to any drive components. Be aware that loose clothing or jewelry may catch on moving parts.

#### Step 2

Remove the screws at the top of the lift that attach the front cover.




#### Step 3

Remove the front cover by tilting it forward and lifting upwards. The bottom of the front cover sits on a pin on either side of the lift frame. Set the cover aside in a safe location where it will not get damaged.



#### Step 4

Loosen the bolts that attaches the upper limit switch assembly. Slide the assembly down the track until the lower switch in the assembly comes in contact with the lift's car. You should hear the limit switch click as contact is made. Retighten the set screw fastening the limit switch assembly in place.

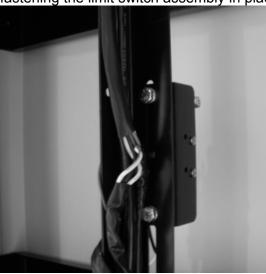

#### Step 5

Replace the front cover and secure it with the screws you removed in step 1.

#### Step 6

Re-connect the lifts electricial power at the circuit breaker. Verify that the emergency switch is in the on position.

#### Step 7

Run the lift in the down direction for several inches. Next, run the lift in the up direction. Continue to press the up button until the upper limit switch has caused the lift to stop. Verify that the platform has stopped level with the upper landing. If it has not, readjust the limit until it is level.

## **Verifying Operation of the Lift**

Caution: Complete the following section before training the customer to use the lift.

#### Step 1

Run the lift up and down for 5 complete cycles. Hold the direction button down and allow the limit switches to stop the lift. At the top, verify that the platform stops level with the upper landing. At the bottom, verify the access ramp (if equipped) unfolds and rests on the ground.

#### Step 2

Verify the operation of the Emergency stop switch. When this switch is pushed in the lift should not run in either direction. When this switch is turned and pulled back out, the lift should operate normally.



#### Step 3

Verify the operation of the sensor pan underneath the platform. Start with the lift at the top landing. Press up on the sensor pan. While holding the pan in this location, press the down switch on the platform. The lift should not run.

Warning: Do not run the lift if anyone is under the platform.

## **Manual Override**

Your lift is equipped with a manual handcrank, to be used in the case of a power failure.

#### Step 1

Before using the manual handcrank verify that it's use is required. Check that the emergency stop switch is pulled out. Check that the electrical cord is connected to the supply. Also check that the buildings circuit breaker has not tripped. Try to run the lift by pushing both the up and down buttons. If the lift still will not run, complete the following steps:

#### Step 2

Disconnect the power from the lift.

Warning: Do not service or operate the manual handcrank while the lift is connected to electricity.

#### Step 3

Remove the screws and remove the top cap at the top of the tower.



#### Step 4


Insert the manual handcrank into the opening on the top of the brake assembly. It may be necessary to slightly rotate the handcrank until it fully seats down on the hex portion of the brake. Rotate the crank to raise or lower the platform.

Warning: Never operate the lift while the manual override crank is inserted into the lift.

## **Call-Sends (optional)**

The optional Call-Send controls are to be used at the upper and/or lower landings to call the platform to you or send it to the other landing.

They should be mounted on the wall at each landing at a convenient height. Consult local codes for placement with consideration to clearances.

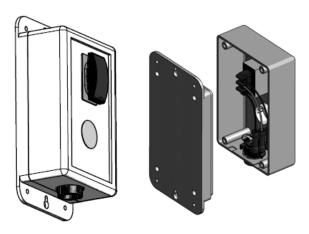

A length of multi-conductor wire will need to be run from the bottom of the lift tower up to the landing Call-Sends. Consult local codes for type and mounting requirements. After wiring is completed, the wiring harness must be plugged into it appropriate receptacle on the controller (ref pg 5).

#### **Wiring Connections**


Call-Send	<u>Lift Harness</u>
Blue	Blue
White	White
Red	Red

## Top Landing Gate (optional)

The optional top landing gate is provided with a combination mechanical lock and electric contact (interlock).

#### The interlock:

- Prevents the lift from running if the gate is not closed.
- Prevents the gate from being opened if the platform is not at the top landing.
- Unlocks when the lift is on the upper limit switch.

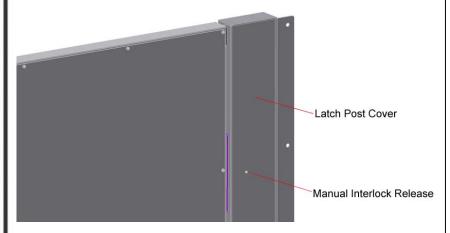

A crescent shaped key is provided to manually unlock the gate during installation. The key is inserted from the back side to lift up on the solenoid that holds the gated locked.

Mount the gate by placing onto the upper landing making sure to align the gate opening with the platform (outer guard rail not shown for clarity).

There are a number of attachment holes provided in the threshold portion of the gate for mounting using wood lag screws or concrete anchors as appropriate.

Remove the latch post cover and connect the call-send and interlock wire harnesses.

The vertical posts of the gate must be attached to a supporting structure, (the gate is not designed to be freestanding).

A length of multi-conductor wire will need to be ran from the bottom of the lift tower up to the landing gate. Consult local codes for type and mounting requirements. After wiring is completed, the wiring harness must be plugged into it appropriate receptacle on the controller (ref pg 5).

#### **Wiring Connections**

Interlock Harness	Lift Harness
Black	Black
Black	Green
Yellow	White
Yellow	Orange
	*Blue
	* Brown

\*Must be tied together

Call-Send Harness	<u>Lift Harness</u>
Blue	Blue
Yellow	White
Red	Red



### **EMI and Flush Strike Interlocks (optional)**

The optional EMI or Flush Strike Interlocks are provided with a combination mechanical lock and electric contact. They are to be used with existing doors. The interlock:

- Prevents the lift from running if the door is not closed.
- Prevents the door from being opened if the platform is not at the landing.

A length of multi-conductor wire will need to be ran from the bottom of the lift tower to each interlock. Consult local codes for type and mounting requirements. After wiring is completed, the wiring harness must be plugged into it appropriate receptacle on the controller (ref pg 5).

#### Flush Strike Interlock

Refer to the instructions inside the strike box for mounting requirements.

#### Flush Strike Interlock Wiring Connections

#### For AC powered lifts

10.710 portoleu m.ts			
Flush Strike Wires	<b>Rectifier Wires</b>	Lift Harness Wires	
Black-White Stripe	Red		
Black-White Stripe	Black		
	Yellow	Black	
	Yellow	Green*	
Yellow		White	
Blue		Orange	
Grey		Blue	
Green		Brown	
Red	Red		
Black	Black		
	Yellow	Red	
	Yellow	Green*	

#### For Optional DC powered lifts

Flush Strike Wires	Lift Harness Wires
Black-White Stripe	Black
Black-White Stripe	Green*
Yellow	White
Blue	Orange
Grey	Blue
Green	Brown
Red	Red
Black	Green*

\*This is a single wire that will have 2 wires connected to it.

#### **EMI Interlock**



- 1) Position interlock to door jamb and mark mounting holes.
- 2) Fasten Interlock to door jamb with #8 wood screws.
- 3) Route 4-conductor Interlock cable thru hole in top of interlock and make wire connections.
- 4) Attach door keeper and emergency key plates to hoistway door.





EIVII IIICETIOCK WITTING	Connections
<u>EMI</u>	Lift Harness
Black (A)	Black
Black (B)	Green
Yellow (C)	White
Yellow (D)	Orange
N/A	Blue+
N/A	Brown+
N/A	Red

+Must be tied together when using an EMI Interlock.

## Platform Control with Optional E-Stop Alarm

If equipped, the e-stop button on the platform lights up and sounds an alarm when pressed.

You will need to obtain a 6 volt lantern battery like the example below.



The battery can sit under the top cap to the right of the motor pulley. There is a wire harness for the battery located here. Wire #1 connects to the positive terminal and wire #2 to the negative terminal.

## **Commercial Platform Lift - Owners Section**



Read the manual thoroughly before operating the lift.

Congratulations on the purchase of your Harmar Vertical Platform Lift. This lift has been engineered to provide trouble free service for many, many years. Please read this manual completely before operating your lift.

## **Safety**

- Do not exceed the maximum payload capacity of 750 lbs.
- Do not ride on the lift until it is anchored in place.
- This product is designed only for lifting people and wheel chairs. Do not use it for any other purpose.
- Make sure any obstructions are cleared from underneath the platform area before use.
- Make sure both the passenger and wheelchair are completely on to the platform before using.
- Do not disable any safety equipment or switches supplied with this lift.
- Do not attempt to service the lift yourself. Contact your Harmar dealer for assistance.
- Do not allow children to operate or play around the lift.
- Read all instructions in this manual before installing or operation the lift.

## **Controls**



#### **Emergency Stop**

In an emergency push this red button to stop the lift. Turn the button clockwise to run.

#### Up

Controls upward movement of lift platform. To move platform up, depress and hold the upper half of the rocker switch. To cease movement, release switch.

#### **Down**

Controls downward movement of lift platform. To move platform down, depress and hold the lower half of the rocker switch. To cease movement, release switch.

#### **Keylock (optional)**

Disables controls from operating when keylock is turned off.

## **Operating the Lift**

#### Step 1 - Up

Drive onto and stop in the middle of the platform. Apply the brakes of your chair or scooter.



**Step 2**Verify that the emergency stop button is not activated by giving it a quick turn clockwise.



#### Step 3

Press and hold the UP rocker. The lift will move in the up direction and stop when it reaches the upper landing.



Warning: Always verify the lift's platform has stopped level with the upper landing. If not contact your Harmar Dealer for assistance.

**Step 4**Release the brakes on your chair or scooter and drive off the platform.



#### Step 1 - Down

Drive onto and stop in the middle of the platform. Apply the brakes of your chair or scooter.



**Step 2**Verify that the emergency stop button is not activated by giving it a quick turn clockwise.



#### Step 3


Press and hold the DOWN rocker. The lift will move in the down direction and stop when it reaches the lower landing.





Warning: Always verify the lift's access ramp unfolds fully and rests on the ground. If not contact your Harmar Dealer for assistance.


**Step 4**Release the brakes on your chair or scooter and drive off the platform.




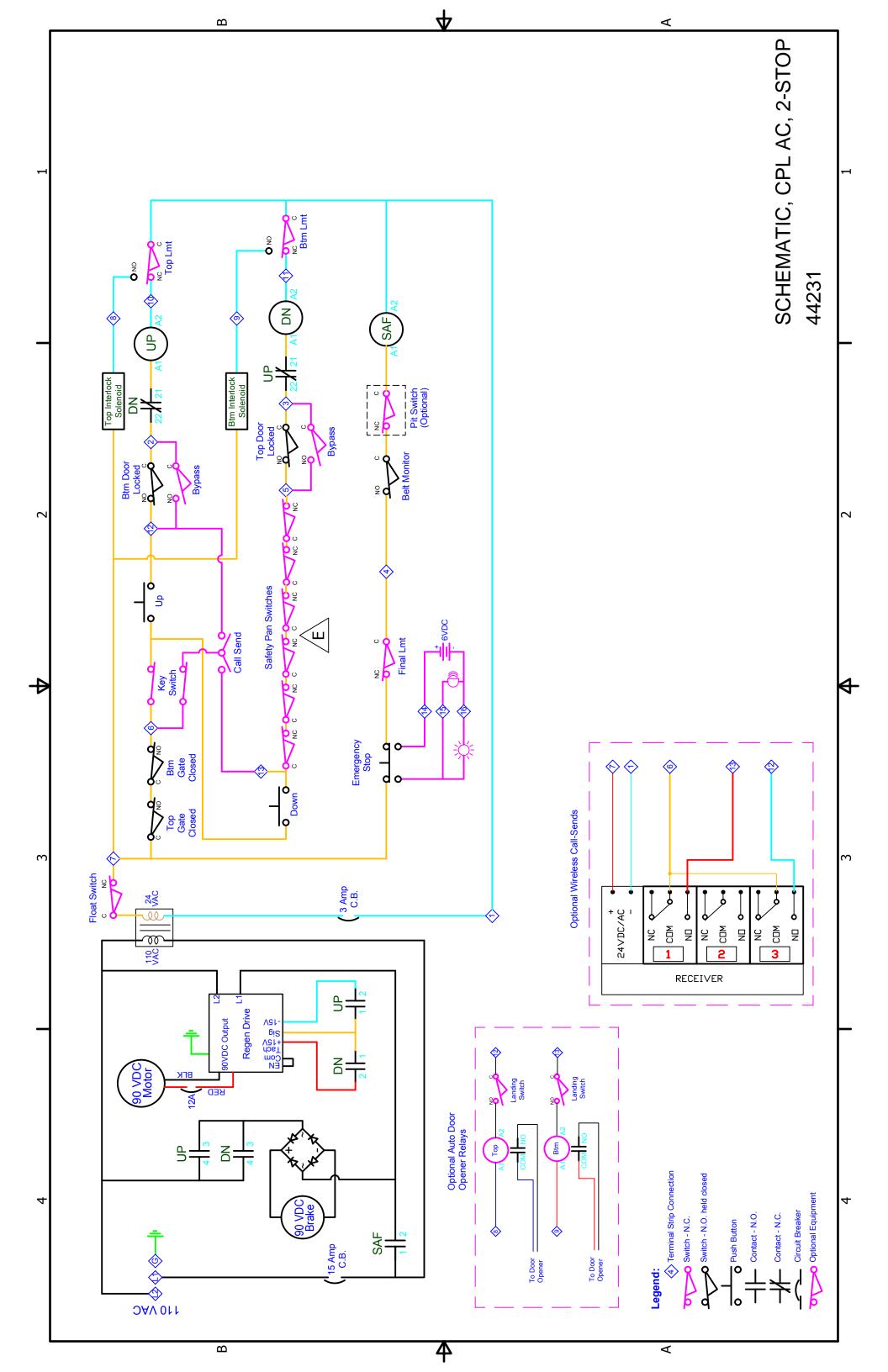


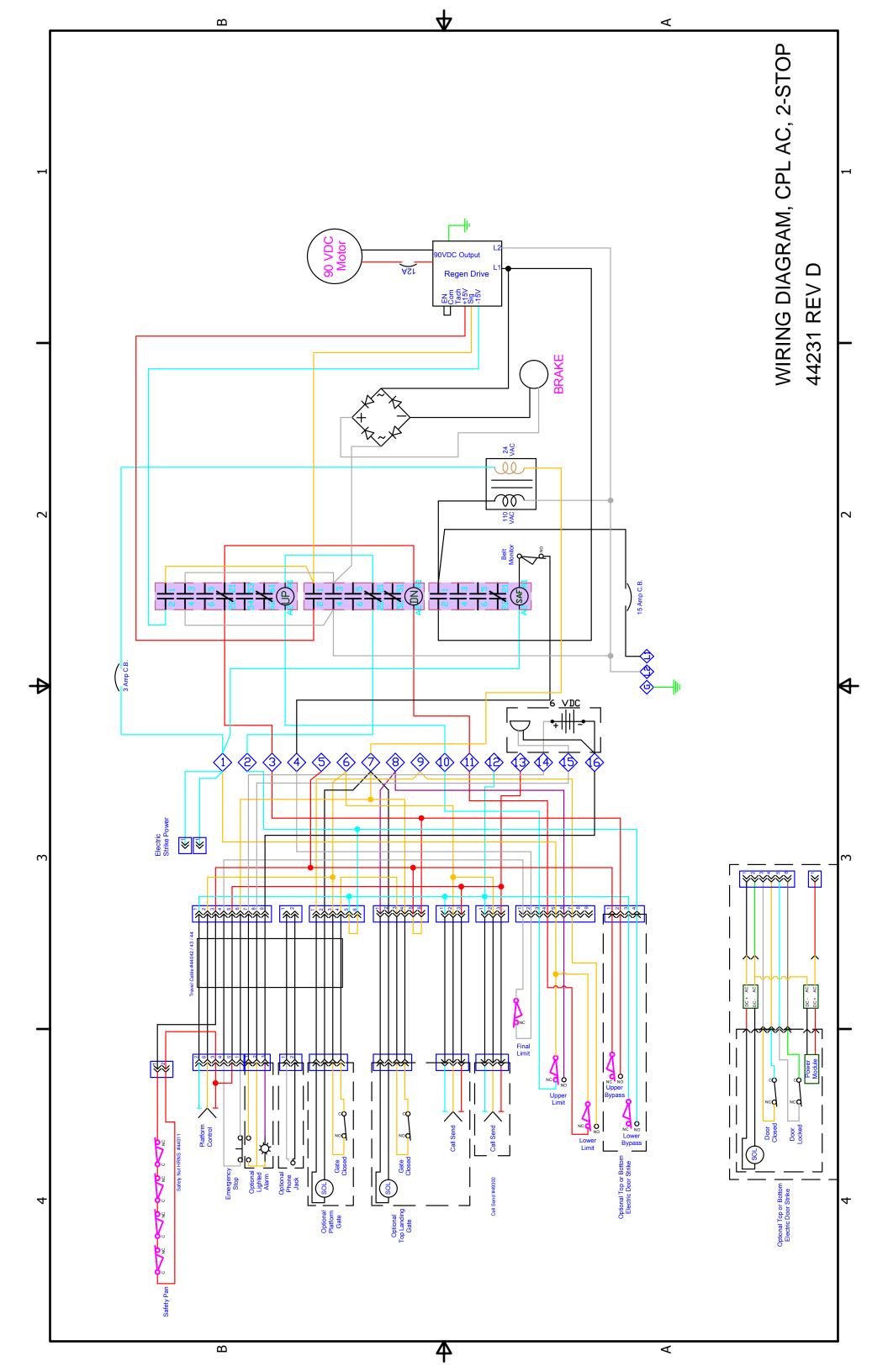


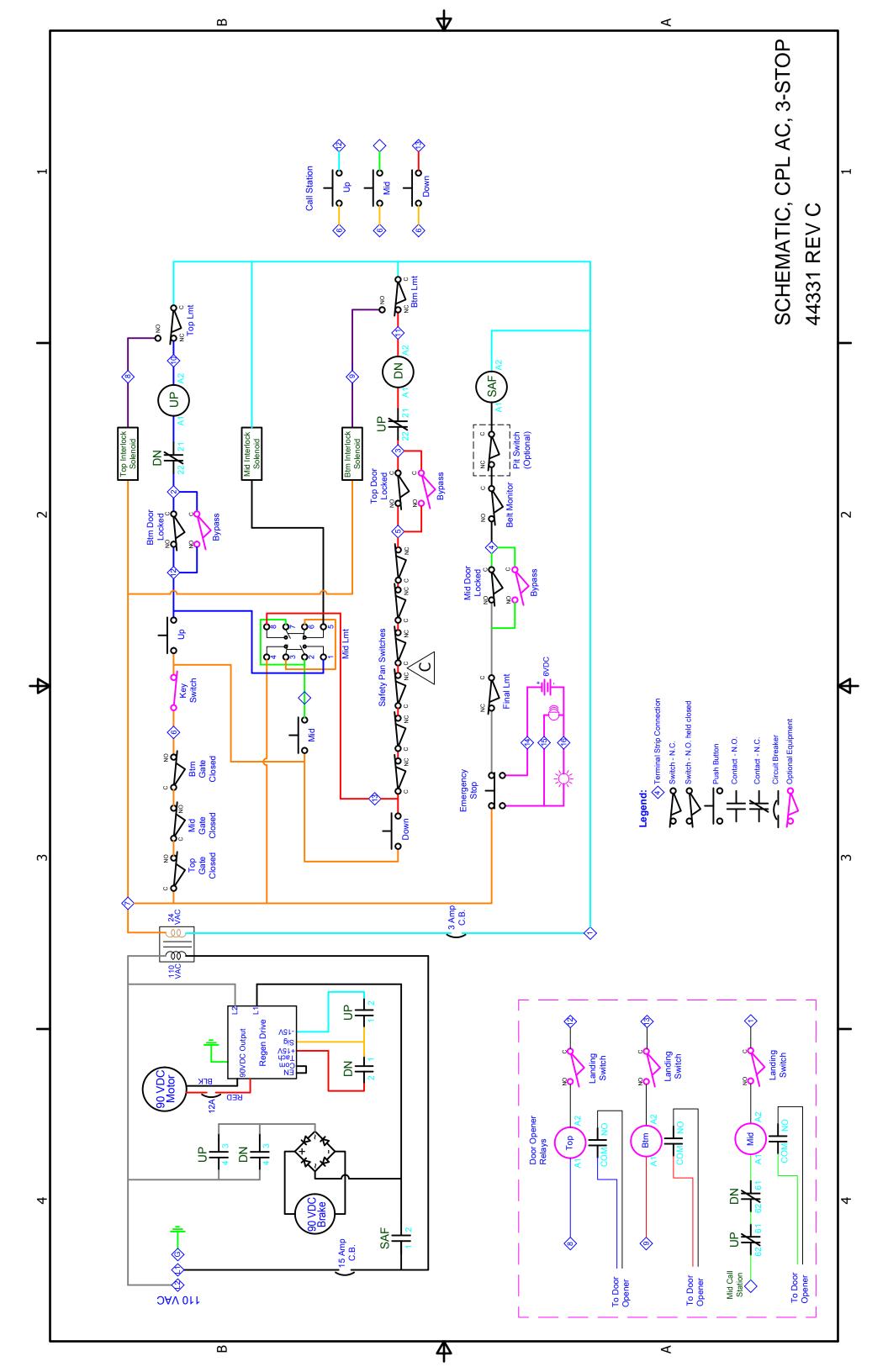

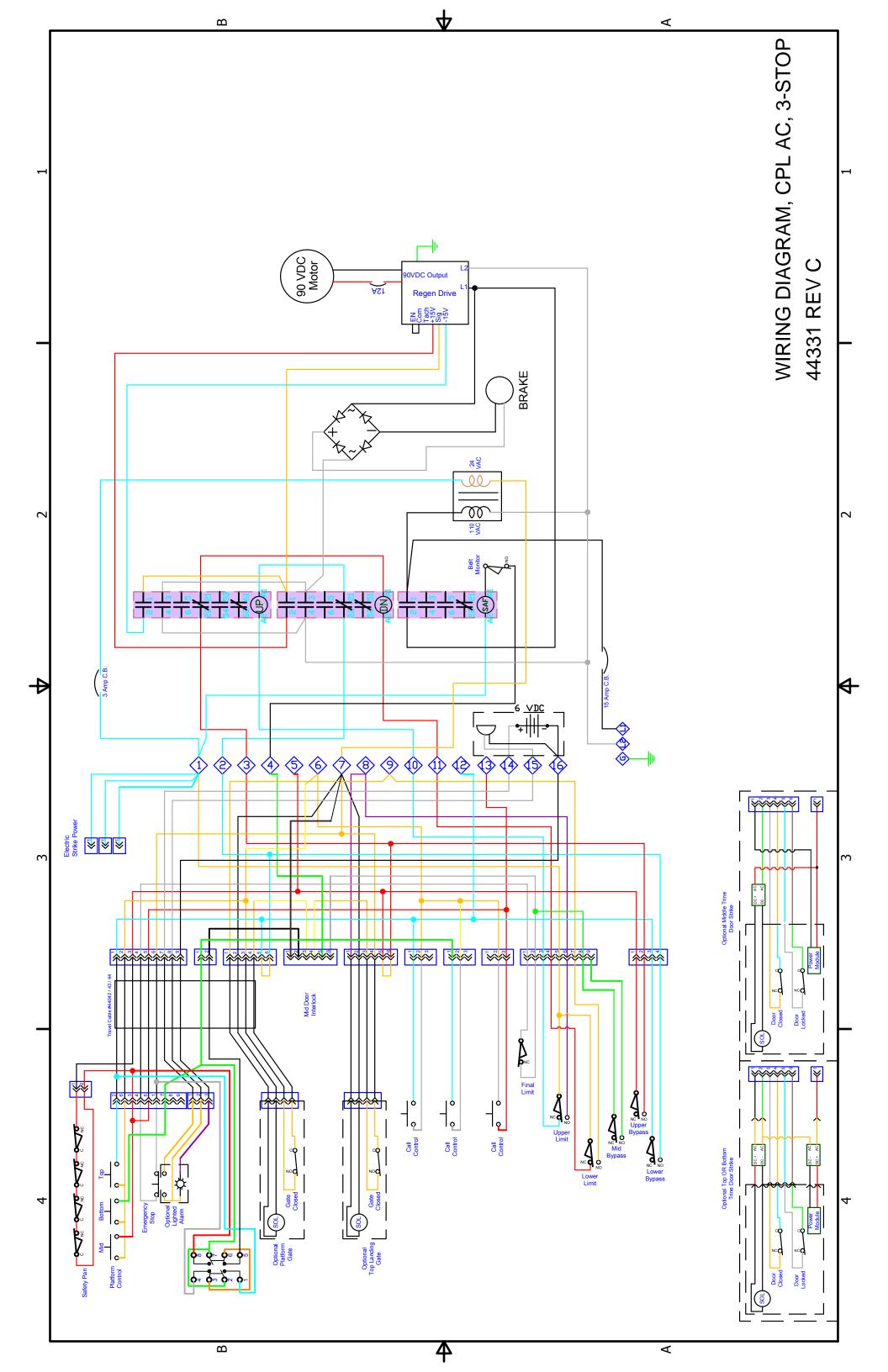

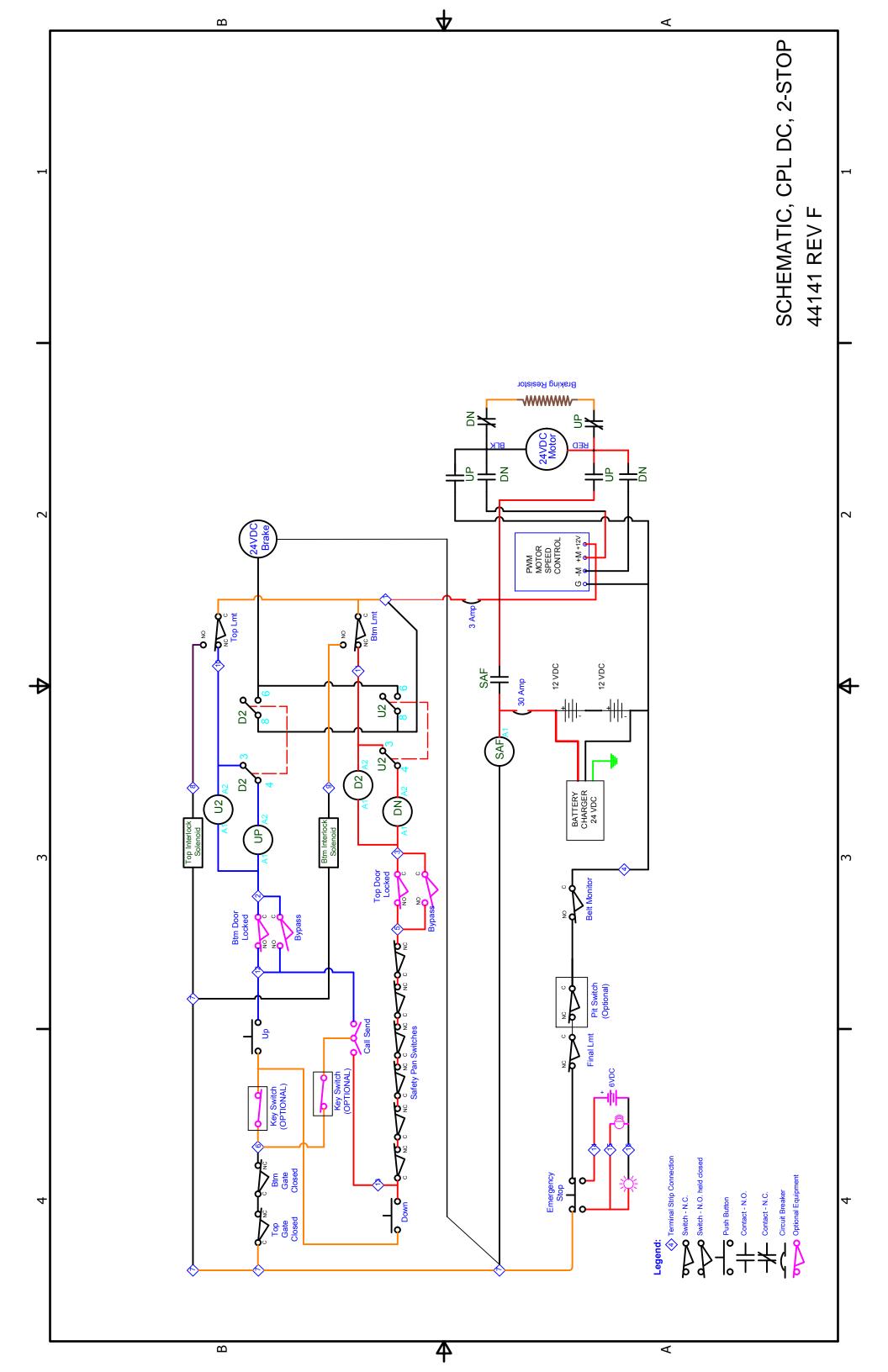

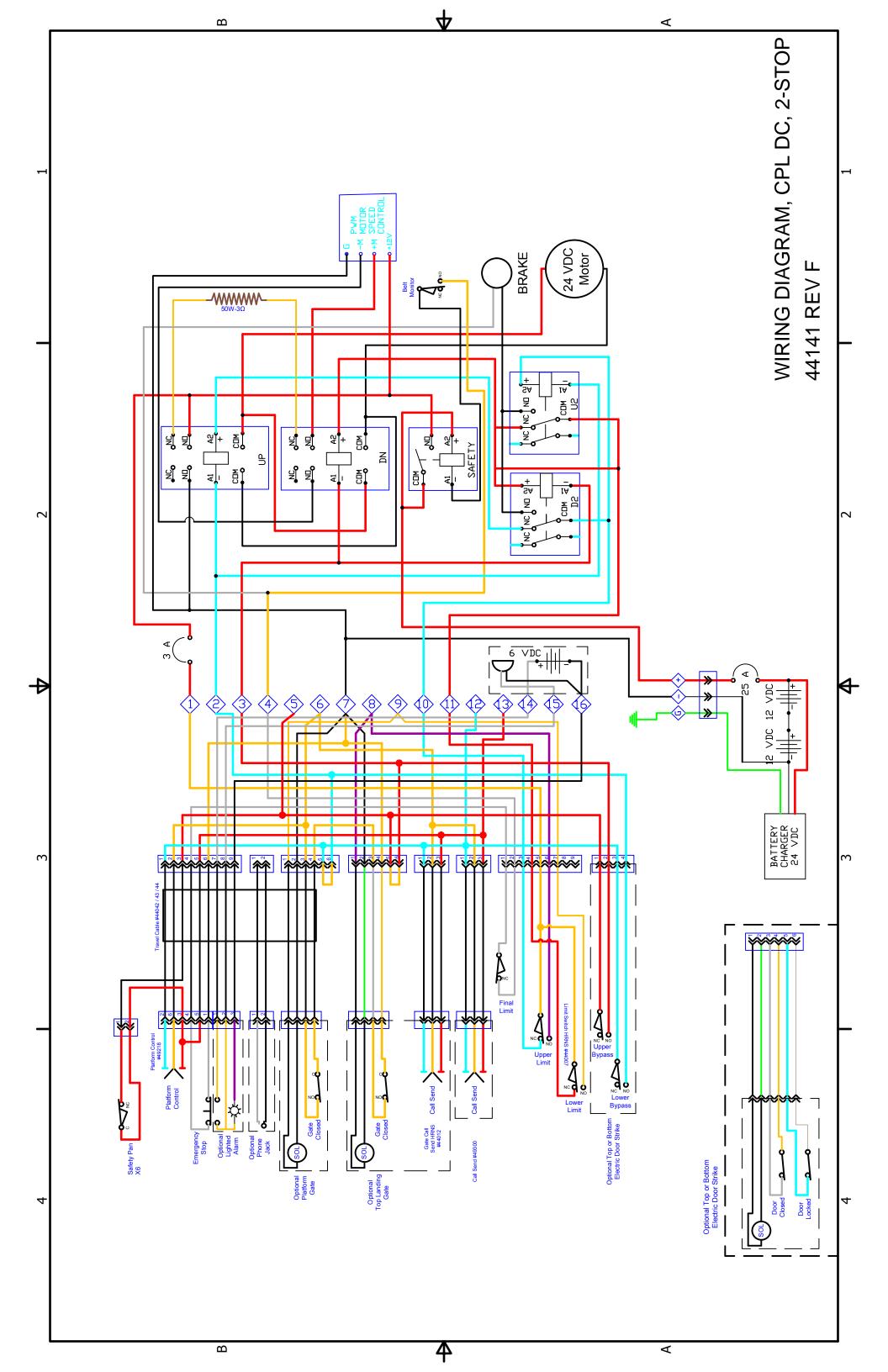

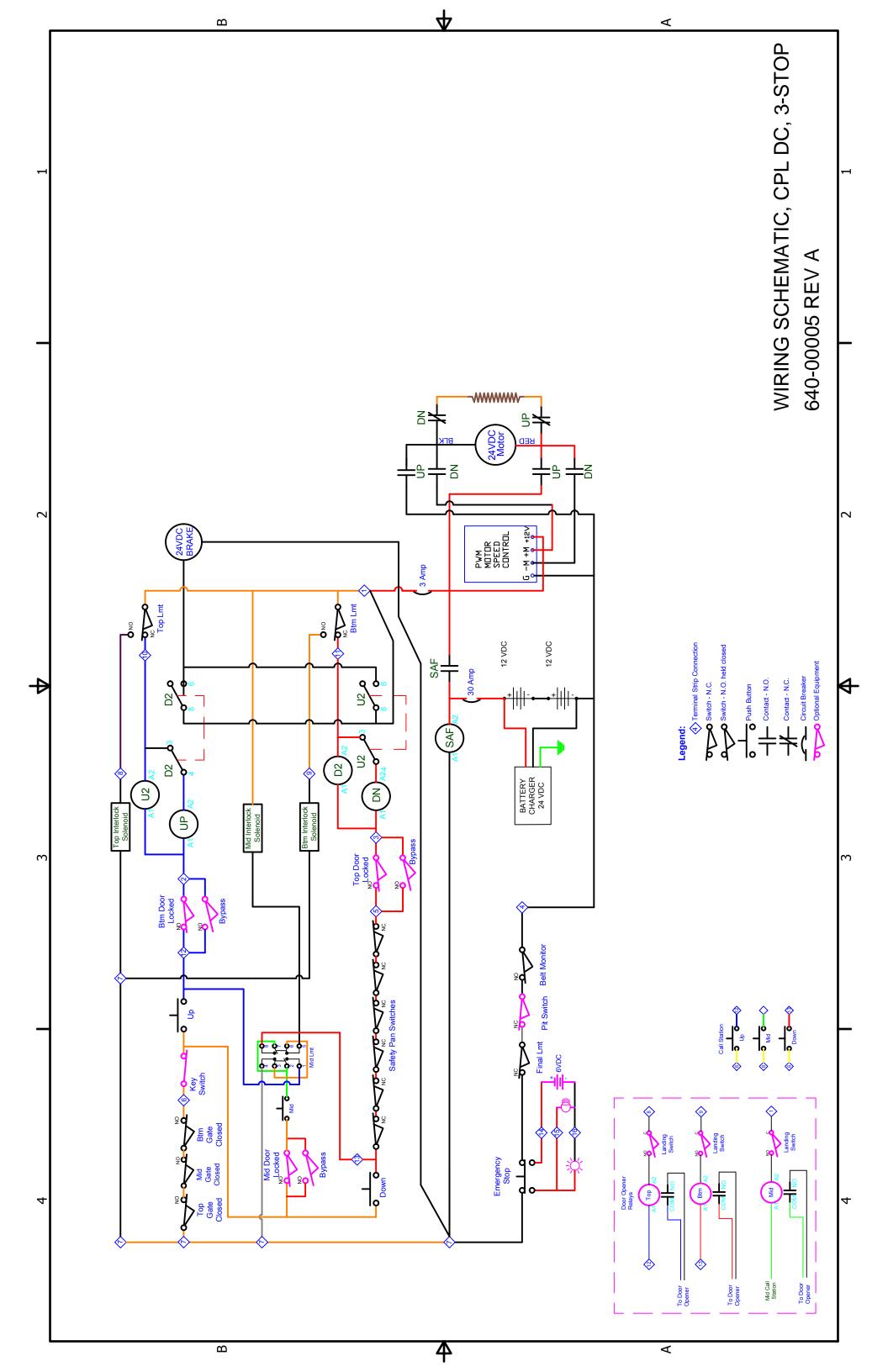







#### LIMITED WARRANTY CERTIFICATE

Products Covered: RPL400, RPL600

Harmar warrants to the original purchaser of a Highlander Residential Vertical Platform Lift (RPL) manufactured by us to be free from defects in material, mechanical and electrical components (parts) for a period of one (1) year, provided that the products have been installed, maintained and operated properly by an Authorized Harmar Distributor or certified Harmar Installer.

#### **EXCEPTIONS TO THIS LIMITED WARRANTY ARE: PLEASE READ CAREFULLY**

- Harmar RPL's installed outdoors must include the Premium Protection Package to maintain 1-year warranty; otherwise warranty is limited to six months.
- Batteries are limited to one (1) year with a Harmar supplied/approved charger.
- Damage resulting from improper installation or operation
- Negligence, alterations, abuse or misuse of the equipment
- · Fire, flood, acts of God
- Shipping damage

- Parts used that are not approved by Harmar Mobility, LLC.
- Labor fees for installation work, repair or service calls are not covered
- Outdoor installation within 1 mile of a coastline reduces warranty to 90 days and expressly excludes rust after 30 days. Warranty is extended to 1 year with Premium Protection Package Option.
- Due to the nature of steel construction being permanently placed in an outdoor environment and its inherent unpredictability, <u>minor paint</u> <u>defects and cosmetic rust are not covered after 30 days</u> by Harmar's <u>Limited Warranty</u>.

This warranty starts on the date of initial product installation (not to exceed 90 days from the date of manufacture), provided the warranty certificate is completely filled out and returned to Harmar within ten (10) days of installation. Harmar and its dealers shall not be liable for any consequential, special or incidental damages arising out of the purchase or use of the unit or resulting from the breach of this Limited Warranty, or any implied warranty. The limit of liability of Harmar and its dealer hereunder shall be the unit's purchase price. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages, or legal remedies, so these above limitations may not apply to you. All warranty claims must be reported to the dealer from whom the lift was purchased as they have responsibility for handling your warranty claim. The dealer is to contact the Technical Services Department of Harmar and provide the serial number of the product along with a description and evidence of the defect(s) supporting a warranty claim. Dealers may charge for labor, service, travel, or other associated costs to make repairs, and such charges are not covered by this Limited Warranty. It is permissible to have any repairs or replacement work done as a result of any defects in material and workmanship by someone other than the Dealer under this Limited Warranty. However, the Limited Warranty does not cover any charges or expenses assessed by any such other person or company performing such repairs or replacement work. All parts used to replace defective materials must be genuine Harmar parts to be covered by this Limited Warranty. This Limited Warranty gives you specific legal rights, and you may have other rights which vary from state to state. Harmar will not be charged for labor, consequential damage or repair expenses. Harmar will not, under any circumstances, be liable for the loss of the use of its products or loss of time. Defective parts must be returned, if requested, prepaid, to Harmar for inspection prior to credit or replacement. At Harmar's discretion, any part found to have been modified, over-stressed, damaged by accident, or misused is not covered by this warranty. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES OR CONDITIONS, INCLUDING ALL IMPLIED WARRANTIES OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND THERE ARE NO WARRANTIES THAT EXTEND BEYOND THE DESCRIPTION OF THE LIMITED WARRANTY DESCRIBED HEREIN.

## Highlander Enclosed Platform Lift Installation and Owners Manual

