


INSTRUCTIONS

HOW TO USE

- 1. Locate the EZ-SHOWER Bedside Shower, dowel, and string.
- 2. Insert dowel into top of Shower.
- 3. Insert string through dowel, and then tie string in knot.
- 4. Fill Shower with warm water. Release clamp to allow water flow; tighten clamp to stop.
- 5. Drain Shower completely after each use.

STORAGE & CLEANING

Product is made of PVC and contains an anti-microbial agent. Rinse with warm water and mild soap, then allow to air dry thoroughly before folding and or storing.

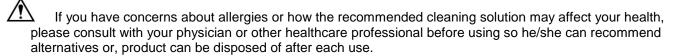

CE Technical Files kept at: Enable Access, Marshmoor Works Great North Road, North Mymms Hatfield, AL9 5SD, UK
© EZ-ACCESS, a division of Homecare Products, Inc. All rights reserved.

All text and images contained in this document are proprietary and may not be shared, modified, distributed, reproduced, or reused without the express written permission of EZ-ACCESS.

12247 REV 01-06-15