

Beach Access Chair

PART #: F-6001

250 LB. [113 kg] MAXIMUM WEIGHT CAPACITY

MANDATORY

LEAVE THIS MANUAL WITH CHAIR OWNER

Read and follow all instructions.

Safety can only be ensured if the beach chair is installed and operated according to these instructions.

- NEVER permit children to play on or around the chair
- DO NOT allow children to use the chair without adult supervision
- DO NOT use beach chair without checking tires for proper inflation pressure. Chair may be unstable or unsafe if the tire pressure is too high or too low
- NEVER operate the beach chair in water. This product is not a floatation device




READ CAREFULLY

Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.

Parts List

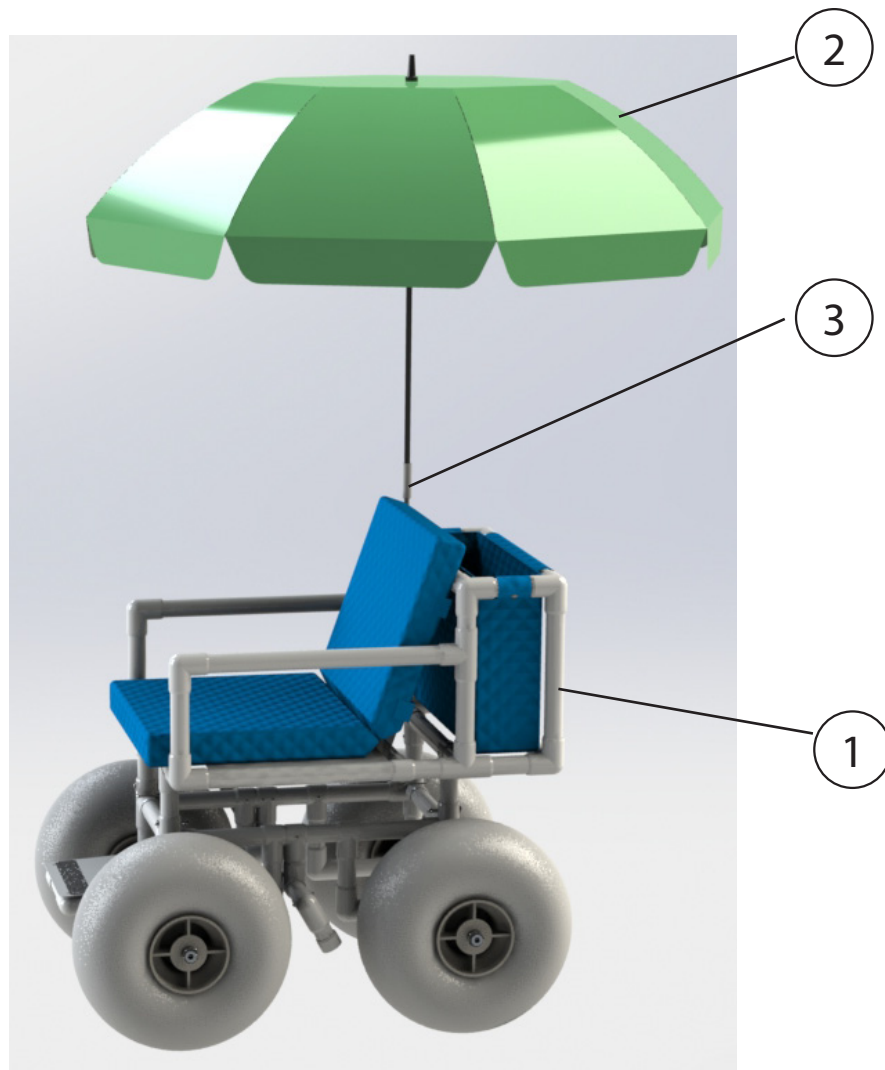


ITEM #	QTY	DESCRIPTION
1	1	BEACH ACCESS CHAIR
2	1	UMBRELLA
3	1	UMBRELLA CLAMP
4	1	SEAT BELT (not shown)

Assembly Instructions

Set the base assembly of the beach access chair on a flat, clean surface. Locate the front axle and slide it through the front of the beach access chair. See pictures below.




Next, locate the rear axle and install it into the back of the beach access chair. Once completed, the chair should look like the picture below.

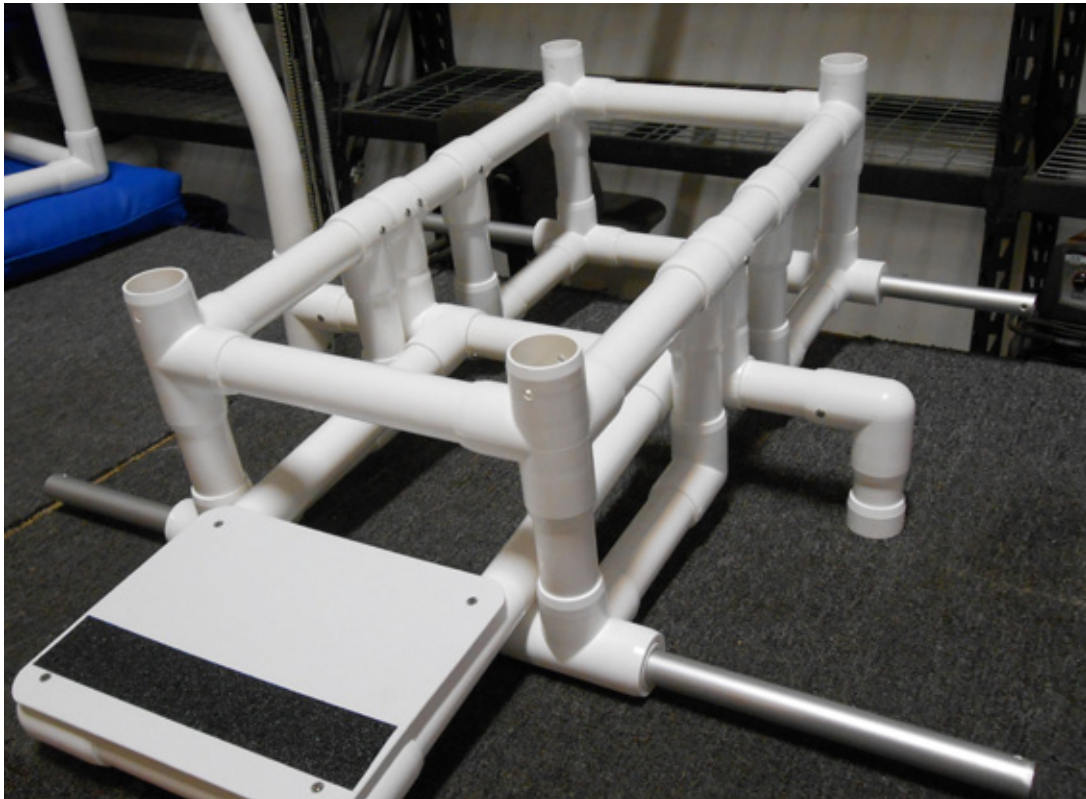


Assembly Instructions

Attach two of the four wheels to the rear axle, one wheel on either side of the frame. Lock the wheels in place by inserting the locking pins into the hole at the end of the axle and flip the ring over the end of the axle. See pictures below.

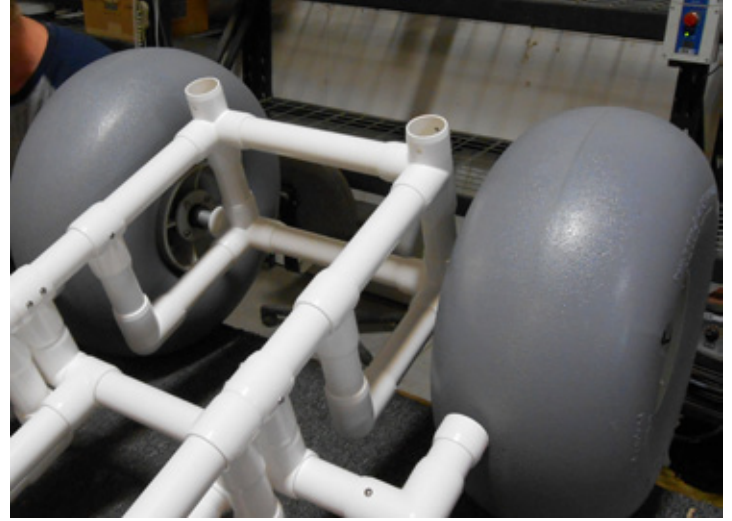




Repeat this process for the front two wheels, attaching them to the front axle. Once completed, the beach access chair should look like the picture shown to the right.

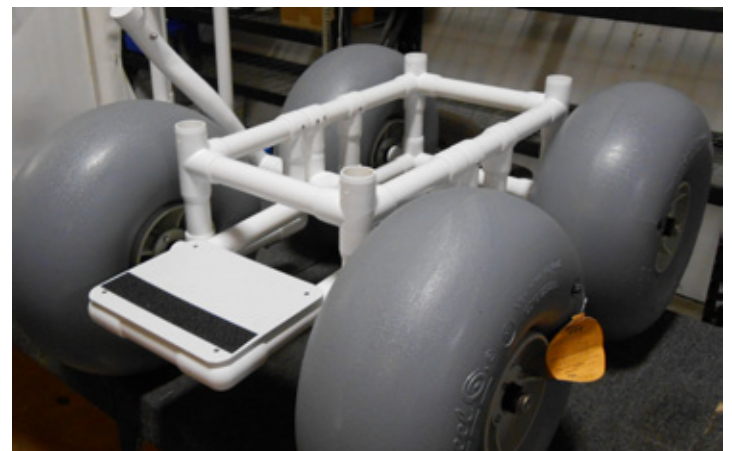

Assembly Instructions

Set the base assembly to the side and place the seat assembly on a flat, clean surface. There are four PVC inserts that came with the unit that are all color coordinated to match the stems on the seat assembly. Install these four pieces, as shown in the pictures below, making sure to match the colors up.

(Note: a phillips screwdriver will be required)

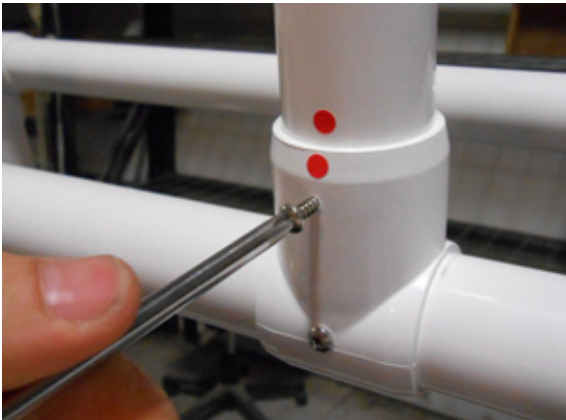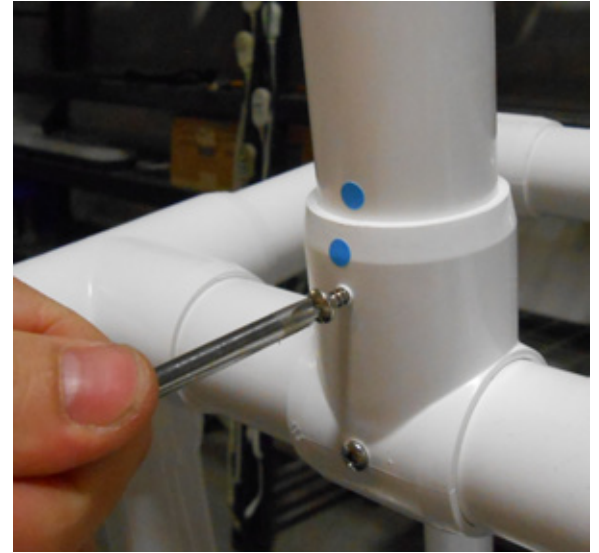
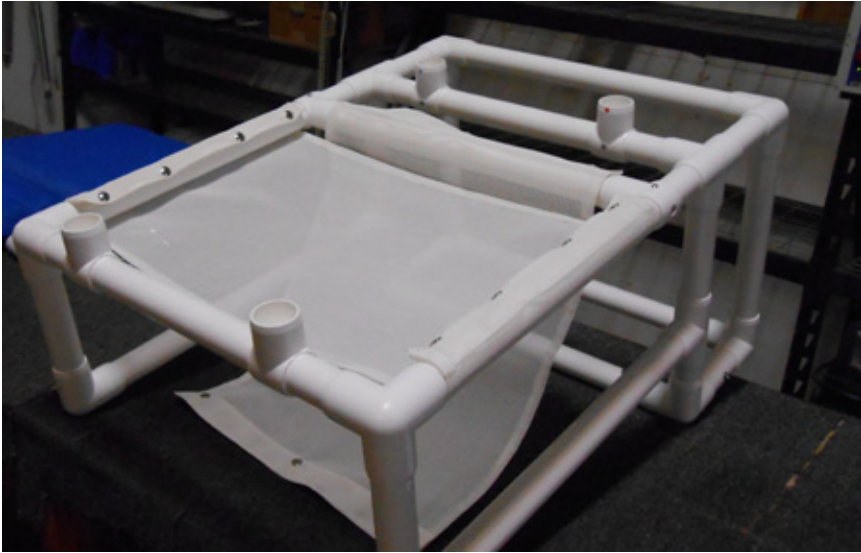


Once all four pieces have been installed the seat assembly is ready to be placed on top of the base assembly as shown in the picture to the right.





Assembly Instructions

Attach locking pins to all four corners to secure the seat assembly to the base assembly. Refer to the pictures below.

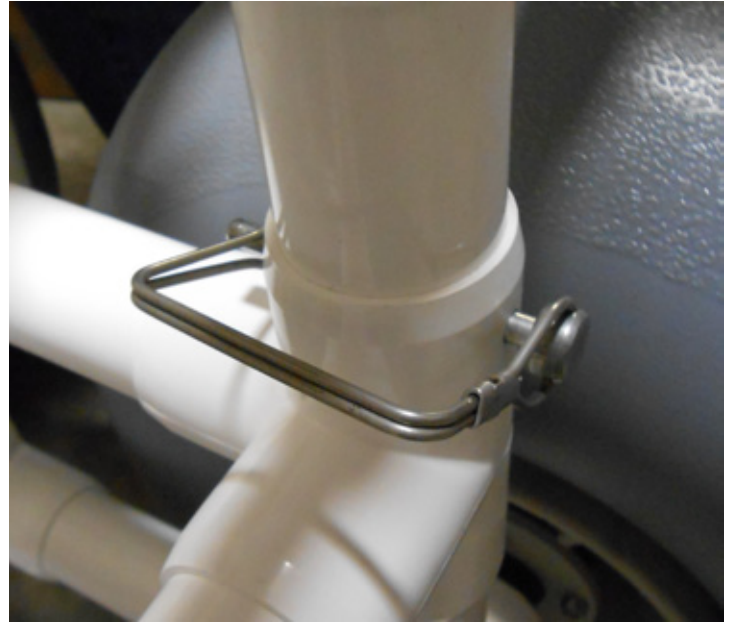
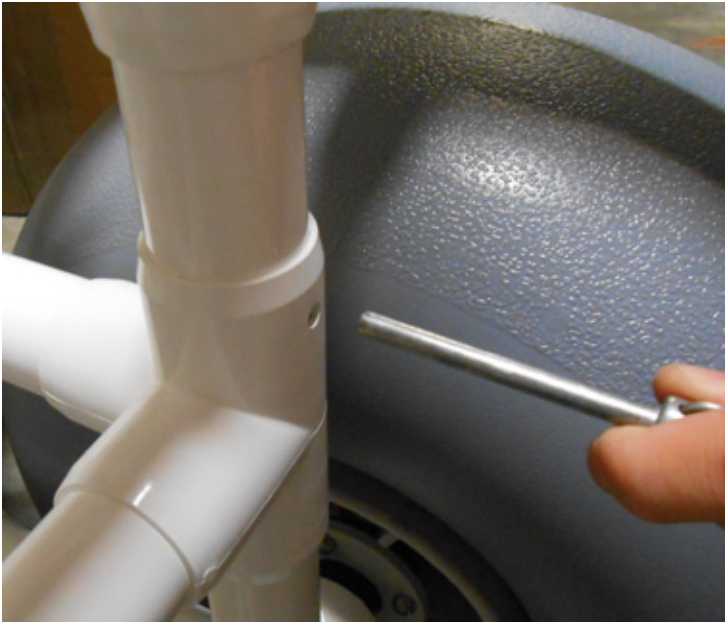


The backrest netting needs to be secured to the frame. To do this, stretch the netting up and over the top bar and attach the buttons on the fabric to the snaps on the under side of the top bar. See pictures below for example.

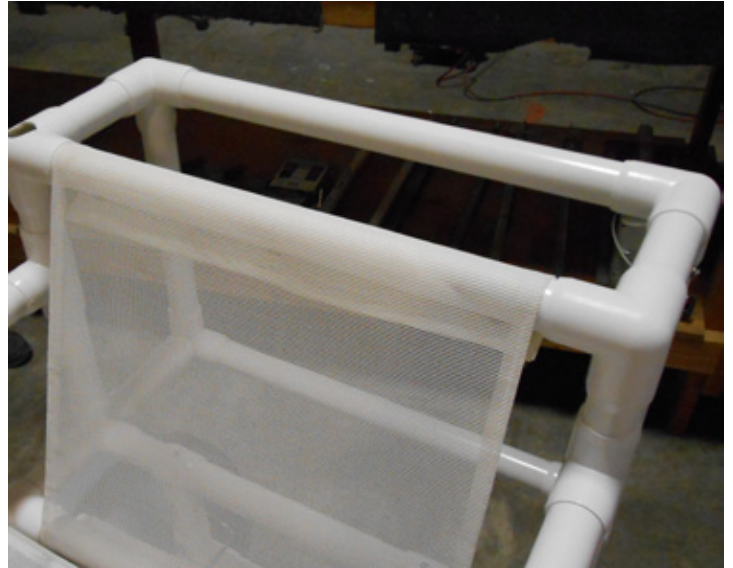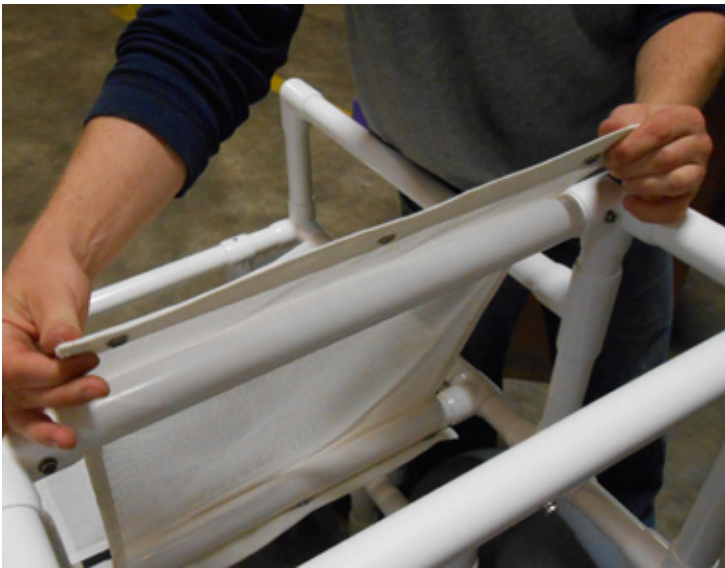

Assembly Instructions

Once the backrest fabric is secure, the seat and backrest cushion may be placed on the chair. To secure in place, the backrest has straps that wrap around the back of the fabric and Velcro™ together. Refer to the pictures below.

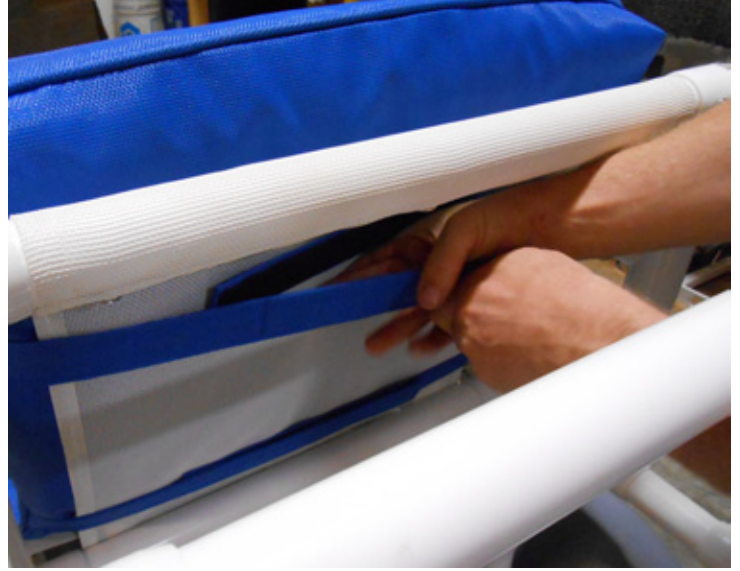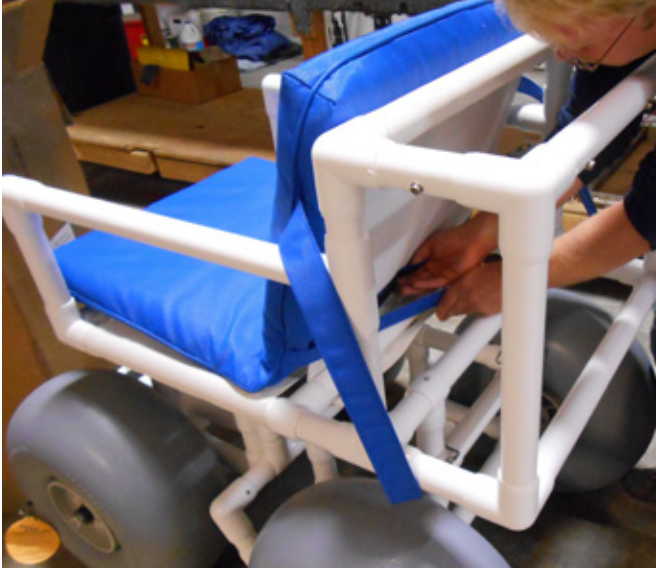

Set the storage bag in the back compartment and secure in place using the buttons on the bag and the snaps on the frame. See pictures below and the first two pictures on the next page.



Assembly Instructions



Lastly, install the seat belt in place by looping it through the chair and securing using the snap buttons. See photo in bottom left. The Beach Access Chair is now ready for use! Finished assembly should look like the picture to the bottom right.



Operating Instructions

The Beach Access Chair operates in an assisted mode; one person rides in the chair while another pushes it from behind. **ALWAYS USE THE SAFETY BELT WHEN RIDING IN THE BEACH ACCESS CHAIR.**

Using the Parking Brake

When the Beach Access Chair is stopped the parking brake should always be engaged. To engage the parking brake, rotate the brake lever back and insert the stainless steel nut into the hole in the side tube as shown below.



Engaging the Parking Brake



Parking Brake Engaged



To release the brake, pull the brake lever nut out of the hole (sideways) and rotate the brake handle forward as shown below.



Parking Brake Disengaged

Umbrella Assembly & Installation

The umbrella comes in a separate package and needs to be assembled before use. First, attach the clamp to the chair frame. Then, insert umbrella pole into the top of the clamp, making sure the button is aligned with the button hole. See pictures below.

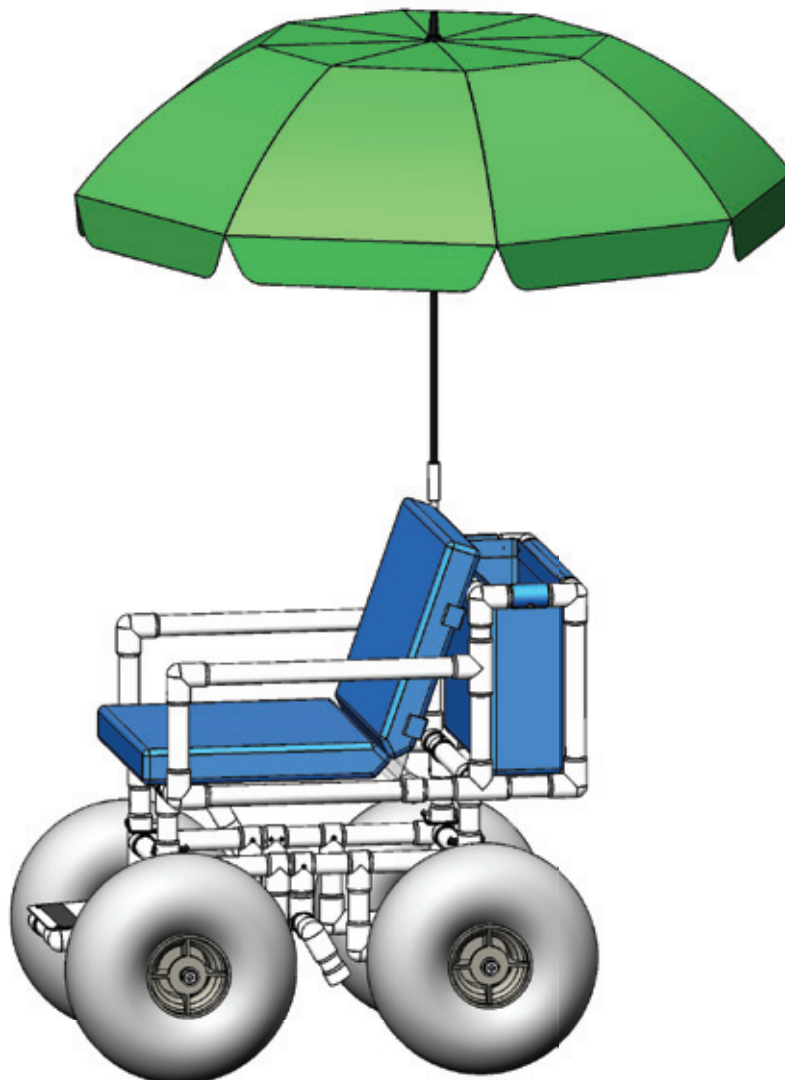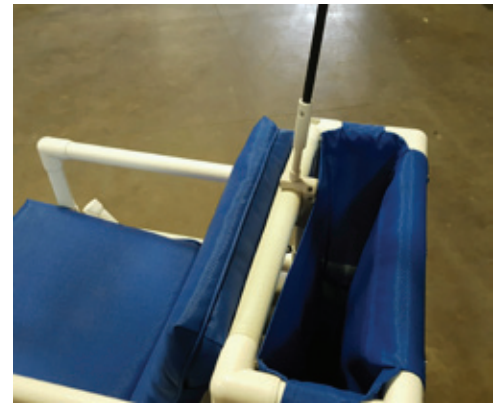

AQUA CREEK PRODUCTS, LLC LIMITED ONE (1) YEAR WARRANTY:

BEACH ACCESS CHAIRS

(ITEM #'s F-013BAC, F-014BAC, F-015RBAC, F-016BS, F-6001)

Aqua Creek Products, LLC (a.k.a. Aqua Creek) warrants to the original end user purchaser that products manufactured by Aqua Creek, when properly installed in accordance with assembly and installation instructions and properly used and maintained, shall be free from defects in material and workmanship for a period of **one (1) year from the date of original purchase**, provided that Aqua Creek receives prompt notice in writing of any defect or failure and satisfactory proof thereof, with the following exceptions:

Exceptions:

- This warranty specifically excludes reimbursement for labor to remove, repair, or install the product and any freight charges.
- This warranty does not cover any damages due to accident, force majeure, act of nature, misuse, abuse, negligence or failure to properly maintain any products, or normal wear and tear from day to day operations.
- In the event that any products are altered, repaired, or improperly installed by anyone without the prior written approval by Aqua Creek, all warranties are void.

IMPORTANT: User is not to exceed the rated weight capacity of this product. It is the responsibility of the product owner to verify the weight of the patron for warranty claim purposes and to ensure that the product is not overloaded.

To initiate a warranty claim, the owner of an Aqua Creek product must provide the place of purchase, in writing, with a full description of the product, its serial number, the dates of purchase and installation, and the exact nature of the defect. Within thirty (30) days after receipt of a written warranty claim by Aqua Creek, and barring any unforeseen delays, the place of purchase will be notified of Aqua Creek's decision regarding the claim. All warranty calculations are from date of original purchase at manufacturer's suggested retail price. Non-payment for product to Aqua Creek may void warranty.

If requested by Aqua Creek, any defective product must be returned, freight prepaid by customer, to Aqua Creek's designated factory location or duly appointed distributor for inspection and/or repair. Aqua Creek will, at its option, repair or replace the failed or defective item, and deliver the repaired product or replacement to the customer. All freight charges are the responsibility of the customer. Products returned to Aqua Creek for which Aqua Creek provides replacement under this limited warranty shall become the property of Aqua Creek. A new warranty period shall **NOT** be established for the repaired or replaced products. Such products shall remain under warranty only for the remainder of the original warranty period on the original products purchased.

This written limited warranty constitutes the final, complete and exclusive statement of warranty terms. No person or organization is authorized to make any other specific or implied warranties or representations on behalf of Aqua Creek.

THE WARRANTIES SET FORTH HEREIN ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED, INCLUDING WITHOUT LIMITATION ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE.

THE SOLE AND EXCLUSIVE REMEDIES FOR BREACH OF ANY AND ALL WARRANTIES WITH RESPECT TO THE PRODUCTS SHALL BE LIMITED TO REPAIR OR REPLACEMENT AT AQUA CREEK'S DESIGNATED FACTORY LOCATION, OR DULY APPOINTED DISTRIBUTOR, OR IN PLACE AT AQUA CREEK'S OPTION. IN NO EVENT SHALL AQUA CREEK'S LIABILITY EXCEED THE ENTIRE AMOUNT PAID TO AQUA CREEK BY THE ORIGINAL PURCHASER FOR THE FAILED OR DEFECTIVE PRODUCT.

IN NO EVENT SHALL AQUA CREEK PRODUCTS, LLC BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL, INDIRECT, PUNITIVE OR EXEMPLARY DAMAGES OR LOST PROFITS FROM ANY BREACH OF THIS LIMITED WARRANTY OR OTHERWISE.

Aqua Creek Products, LLC
9889 Garrymore Lane
Missoula, MT 59808
Toll Free: (888) 687-3552
Local/Intnl: (406) 549-0769
www.aquacreek.com